



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory
1306 W. Miracle Mile, Tucson, AZ 85705

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-000 331 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

WLL
5/6/15

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 4.23.2015 Cal 1 0.096 AC Cal 2 0.096 AC

Between the dates of 3.26.2015 and 4.23.2015 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on —.
The instrument was returned to service on —.

Terry Gallegos

Terry S. Gallegos

5-5-2015

ADPS QAS Permit # 33169

SUBSCRIBED AND SWORN to before me this 5th, day of May, 2015.

Elma Escarcega

Notary Public

*Includes documents specified in A.A.C. R13-10-104.A.5

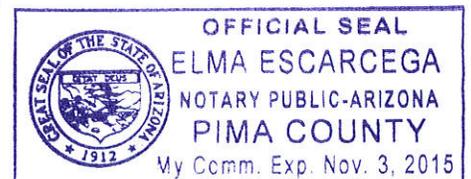


EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

10 of 7
TS4

WLL
5/6/15

QA SPECIALIST Terry Gallegos 28847 AGENCY Tucson Police Department

DATE 4.23.2015 TIME 1143

INTOXILYZER SERIAL # 80-000331 LOCATION DUI-HQ

() 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 096 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE 

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-000331
Core Version: 8105.48
04/23/2015

11:46:51

2 of 7
TS4
WU

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 11:48:00 |
| Diagnostic Test | Pass | 11:48:34 |
| Air Blank | 0.000 | 11:49:01 |
| 0.100 Cal Check | 0.097 | 11:49:22 |
| Air Blank | 0.000 | 11:49:52 |
| Subject Test | 0.000 | 11:50:06 |
| Air Blank | 0.000 | 11:50:33 |
| Wait | ABT* | |
| Air Blank | 0.000 | 11:51:07 |

*Sequence Aborted

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-000331
Core Version: 8105.48
04/23/2015

11:52:41

3 of 7
TS4
WLL

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 11:54:06 |
| Diagnostic Test | Pass | 11:54:40 |
| Air Blank | 0.000 | 11:55:08 |
| 0.100 Cal Check | RFI* | 11:55:23 |
| Air Blank | IPS** | 11:55:37 |

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-000331
Core Version: 8105.48
04/23/2015

11:56:16

4 of 7
TSh
WLL

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 11:57:03 |
| Diagnostic Test | Pass | 11:57:37 |
| Air Blank | 0.000 | 11:58:04 |
| 0.100 Cal Check | 0.097 | 11:58:25 |
| Air Blank | 0.000 | 11:58:55 |
| Subject Test | 0.109* | 12:02:02 |
| Air Blank | 0.000 | 12:02:35 |
| Air Blank | 0.000 | 12:03:02 |
| Subject Test | XXX** | 12:03:26 |
| Air Blank | 0.000 | 12:04:03 |

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-000331
Core Version: 8105.48
04/23/2015

12:04:29

507
B4
WLL

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 12:04:58 |
| 0.100 Cal Check | 0.096 | 12:05:19 |
| Air Blank | 0.000 | 12:05:49 |

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6 of 7
TS4
WLL
5/6/15

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Terry Gallegos 28847 AGENCY Tucson Police Department
DATE 4.23.2015 TIME 1143
Intoxilyzer Serial # 80-000331 Location DUI-HQ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 096 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

DPS Form Exh G-2 (Rev. 05-01)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-000331
Core Version: 8105.48
04/23/2015 11:43:27

7 of 7
TS4
WLL

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 11:43:56 |
| 0.100 Cal Check | 0.096 | 11:44:16 |
| Air Blank | 0.000 | 11:44:46 |

Preventative Maintenance
Performed -- Timer Reset