



CERTIFICATION by CUSTODIAN of RECORDS  
Tucson Police Department Crime Laboratory

By  
6-6-2008

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-000332 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 6-5-08 Cal 1 0.099 AC Cal 2 0.099 AC

Between the dates of 5-8-08 and 6-5-08 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on N/A.  
The instrument was returned to service on N/A.


Clare Conley

Clare L. Conley

6-6-08

ADPS QAS Permit # 33314

SUBSCRIBED AND SWORN to before me this 6, day of June, 2008.

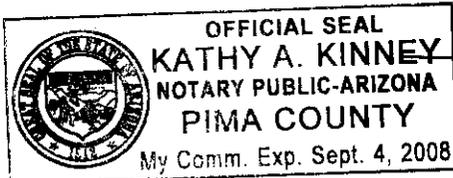

Kathy A Kinney  
Notary Public

\*Includes documents specified in A.A.C. R13-10-104.A.5

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD QUALITY ASSURANCE PROCEDURE

1 of 7  
an  
p4

QA SPECIALIST Clare Conley AGENCY Tucson Police Department  
DATE 6-5-08 TIME 0926  
INTOXILYZER SERIAL # 80-000332 LOCATION DUI HQ ODW

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 099 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS None.  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE Clare Conley

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-000332  
06/05/2008

09:28:38

2 of 7  
CW  
BH

---

Standard Lot#: 735401G  
Last Changed By: T. O HARA #38876

---

QAS: C. CONLEY #34517  
TUCSON PD

---

Operator: C. CONLEY #34517  
TUCSON PD

---

Subject: TEST, SQAP  
DOB: 11/11/1111  
Sex: M Weight: 111

---

15 Minute Deprivation Period? Yes

---

| Test            | g/210L | Time     |
|-----------------|--------|----------|
| Air Blank       | 0.000  | 09:29:31 |
| Diagnostic Test | Pass   | 09:29:58 |
| Air Blank       | 0.000  | 09:30:26 |
| 0.100 Cal Check | 0.099  | 09:30:45 |
| Air Blank       | 0.000  | 09:31:14 |
| Subject Test    | 0.000  | 09:31:34 |
| Air Blank       | ABT*   | 09:31:45 |
| Air Blank       | 0.000  | 09:32:14 |

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-000332  
06/05/2008

09:33:56

---

Standard Lot#: 735401G  
Last Changed By: T. O HARA #38876

---

QAS: C. CONLEY #34517  
TUCSON PD

---

Operator: C. CONLEY #34517  
TUCSON PD

---

Subject: TEST, SQAP  
DOB: 11/11/1111  
Sex: M Weight: 111

---

15 Minute Deprivation Period? Yes

---

| Test      | g/210L | Time     |
|-----------|--------|----------|
| Air Blank | RFI*   | 09:35:01 |
| Air Blank | IPS**  | 09:35:13 |

\*RFI Detect

\*\*Improper Sample

3567  
cu  
P4

**Not a Successfully Completed Test Sequence**

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-000332  
06/05/2008 09:35:30

4 of 7  
CM  
TS

Standard Lot#: 735401G  
Last Changed By: T. O HARA #38876

QAS: C. CONLEY #34517  
TUCSON PD

Operator: C. CONLEY #34517  
TUCSON PD

Subject: TEST, SQAS  
DOB: 11/11/1111  
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

| Test            | g/210L | Time     |
|-----------------|--------|----------|
| Air Blank       | 0.000  | 09:36:46 |
| Diagnostic Test | Pass   | 09:37:13 |
| Air Blank       | 0.000  | 09:37:41 |
| 0.100 Cal Check | 0.098  | 09:37:59 |
| Air Blank       | 0.000  | 09:38:29 |
| Subject Test    | 0.051* | 09:41:37 |
| Air Blank       | 0.000  | 09:42:09 |
| Air Blank       | 0.000  | 09:42:36 |
| Subject Test    | XXX**  | 09:42:52 |
| Air Blank       | 0.000  | 09:43:26 |

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-000332  
06/05/2008 09:44:40

5 of 7  
cu

-----  
Standard Lot#: 735401G  
Last Changed By: T. O HARA #38876  
-----

BH

-----  
QAS: C. CONLEY #34517  
TUCSON PD  
-----

| Test      | g/210L | Time     |
|-----------|--------|----------|
| Air Blank | 0.000  | 09:45:08 |
| Cal Check | 0.099  | 09:45:26 |
| Air Blank | 0.000  | 09:45:56 |

*Clare Conley*

\_\_\_\_\_  
Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

6/6/08  
cu

Tsh  
6.6.08

QA Specialist Clare Conley AGENCY Tucson Police Department  
DATE 6-5-08 TIME 0926  
Intoxilyzer Serial # 80-000332 Location ODW DUI HQ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 0.99 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Clare Conley

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-000332  
06/05/2008 09:26:22

767  
cu

-----  
Standard Lot#: 735401G  
Last Changed By: T. O HARA #38876  
-----

TS4

QAS: C. CONLEY #34517  
TUCSON PD  
-----

| Test      | g/210L | Time     |
|-----------|--------|----------|
| Air Blank | 0.000  | 09:26:50 |
| Cal Check | 0.099  | 09:27:08 |
| Air Blank | 0.000  | 09:27:36 |

Clare Conley  
Operator's Signature



TUCSON POLICE DEPARTMENT  
CRIME LABORATORY

1 of 2  
ew

INTOXILYZER SERVICE RECORD  
FOR  
CALIBRATION GAS TANK CHANGE

Date: 5-28-08 PH 6-6-08  
 QAS: O'Hara 38876  
 Intoxilyzer Serial Number: 80-000 332  
 New Lot Number: 7354016  
 New Expiration Date: 28 DEC 2009

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change." Select "D" for dry gas and enter the following information:  
 Enter Target Value: 0.100  
 Enter Control Lot #: From the tank label  
 Enter Exp Date: From the tank label (mm/dd/yyyy)  
 Edit info Y/N? Y or N  
 Update Std Info? Y ENTER  
 "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing." Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 100 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove and sign printed record. Attach the record to the completed checklist.

SIGNATURE O'Hara #38876

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

2 of 2  
cu

134

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-000332  
05/28/2008 01:29:44

-----  
Standard Lot#: 7354016  
Last Changed By: T. O HARA #38876

-----  
DAS: T. O HARA #38876  
TUCSON PD

| Test      | g/210L | Time     |
|-----------|--------|----------|
| Air Blank | 0.000  | 01:30:11 |
| Cal Check | 0.100  | 01:30:30 |
| Air Blank | 0.000  | 01:31:00 |

-----  
*O'Hara #38876*  
Operator's Signature