



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory  
1306 W. Miracle Mile, Tucson, AZ 85705

TS4  
6.7.2016

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-000334 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 5-27-16 Cal 1 0.099 AC Cal 2 0.097 AC

Between the dates of 4-29-16 and 5-27-16 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on 5-27-16.  
The instrument was returned to service on \_\_\_\_\_.


Amy L. Autret

6-7-16

ADPS QAS Permit # 33718

SUBSCRIBED AND SWORN to before me this 7, day of June, 20 16.

Notary Public

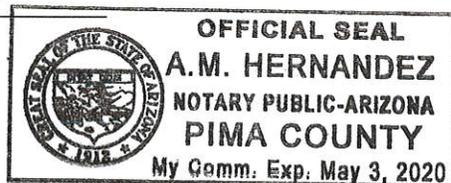

\*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD QUALITY ASSURANCE PROCEDURE

1 of 7  
TS4  
6.6.2016

QA SPECIALIST Amy Autret 53546 AGENCY Tucson Police Department

DATE 5-27-16 TIME 1123

INTOXILYZER SERIAL # 80-000334 LOCATION Lab

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 097 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS

---



---




---



---

SIGNATURE

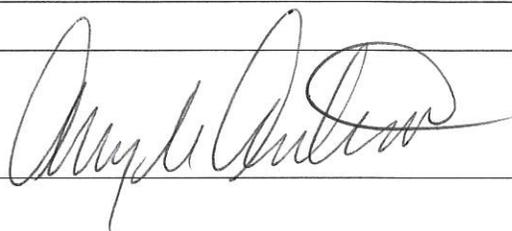

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-000334  
Core Version: 8105.48  
05/27/2016

11:25:03

2017  
JLD  
TSH

---

Standard Lot#: 06616100A1  
Last Changed By: A. AUTRET #53546

---

QAS: A. AUTRET #53546  
TUCSON PD

---

Operator: A. AUTRET #53546  
TUCSON PD

---

Subject: SQAP, SQAP  
DOB: 11/11/1987  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	11:25:58
Diagnostic Test	Pass	11:26:32
Air Blank	0.000	11:26:59
0.100 Cal Check	0.099	11:27:20
Air Blank	0.000	11:27:50
Subject Test	0.000	11:28:06
Air Blank	0.000	11:28:33
Wait	ABT*	
Air Blank	0.000	11:29:08

\*Sequence Aborted

### Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-000334  
Core Version: 8105.48  
05/27/2016

11:29:12

30F7  
JLD  
TSG

---

Standard Lot#: 06616100A1  
Last Changed By: A. AUTRET #53546

---

QAS: A. AUTRET #53546  
TUCSON PD

---

Operator: A. AUTRET #53546  
TUCSON PD

---

Subject: SQAP, SQAP  
DOB: 11/11/1987  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	11:30:09
Diagnostic Test	Pass	11:30:43
Air Blank	0.000	11:31:11
0.100 Cal Check	RFI*	11:31:24
Air Blank	IPS**	11:31:33

\*RFI Detect

\*\*Improper Sample

### Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-000334  
Core Version: 8105.48  
05/27/2016

11:31:41

4067  
dld  
Bh

Standard Lot#: 06616100A1  
Last Changed By: A. AUTRET #53546

QAS: A. AUTRET #53546  
TUCSON PD

Operator: A. AUTRET #53546  
TUCSON PD

Subject: SQAP, SQAP  
DOB: 11/11/1987  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:32:43
Diagnostic Test	Pass	11:33:17
Air Blank	0.000	11:33:44
0.100 Cal Check	0.098	11:34:05
Air Blank	0.000	11:34:34
Subject Test	0.338*	11:37:42
Air Blank	0.000	11:38:15
Air Blank	0.000	11:38:42
Subject Test	XXX**	11:39:13
Air Blank	0.000	11:39:47

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

### Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-000334  
Core Version: 8105.48  
05/27/2016 11:40:04

5 of 7  
JLD  
TSL

-----  
Standard Lot#: 06616100A1  
Last Changed By: A. AUTRET #53546  
-----

QAS: A. AUTRET #53546  
TUCSON PD  
-----

Operator: A. AUTRET #53546  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	11:40:33
0.100 Cal Check	0.097	11:40:54
Air Blank	0.000	11:41:24

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

6 of 7  
dit  
BSH  
6-6-2016

QA Specialist Amy Antret 53546 AGENCY Tucson Police Department  
DATE 5-27-16 TIME 1123  
Intoxilyzer Serial # 80-000334 Location Lab

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-000334  
Core Version: 8105.48  
05/27/2016 11:23:35

7067  
JL  
TS4

-----  
Standard Lot#: 06616100A1  
Last Changed By: A. AUTRET #53546  
-----

QAS: A. AUTRET #53546  
TUCSON PD  
-----

Operator: A. AUTRET #53546  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	11:24:04
0.100 Cal Check	0.099	11:24:25
Air Blank	0.000	11:24:54

Preventative Maintenance  
Performed -- Timer Reset



TUCSON POLICE DEPARTMENT  
CRIME LABORATORY

10F1  
AS

INTOXILYZER SERVICE RECORD

TJG  
6-7-2016

Date: 5-27-16

QAS: Amy Autret

Intoxilyzer Serial Number: 80-000334

Reason For Service: Able to download instrument, but  
unable to write data to COBRA due to  
checksum error.

Date Out of Service: 5-27-16

Calibration Check completed prior to removal from service

Date: 5-27-16

SQAP completed prior to removal from service

Date: 5-27-16

Corrective Action:  Sent to CMI, Inc.

Contact CMI