

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

1 of 7 SC
134

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 11/21/07 TIME 13:36

INTOXILYZER SERIAL # 80-001108 LOCATION Team 3

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 097 AC

Instrument is operating properly and accurately. YES NO

COMMENTS none

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001108
11/21/2007

13:38:40

2 of 7sc
104

Standard Lot#: 719801I
Last Changed By: C. CONLEY #34517

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP,TET
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:39:37
Diagnostic Test	Pass	13:40:04
Air Blank	0.000	13:40:32
0.100 Cal Check	0.100	13:40:51
Air Blank	0.000	13:41:20
Subject Test	0.000	13:41:39
Air Blank	0.000	13:42:07
Five Minute Wait	ABT*	
Air Blank	0.000	13:42:42

*Sequence Aborted

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001108
11/21/2007

13:42:56

3 of 75C

BY

Standard Lot#: 719801I
Last Changed By: C. CONLEY #34517

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP,TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	13:43:37
Air Blank	IPS**	13:43:47

*RFI Detect
**Improper Sample

Not a successfully completed Test Sequence

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001108
11/21/2007

13:44:02

4 of 75C
B4

Standard Lot#: 719801I
Last Changed By: C. CONLEY #34517

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:44:50
Diagnostic Test	Pass	13:45:18
Air Blank	0.000	13:45:45
0.100 Cal Check	0.099	13:46:04
Air Blank	0.000	13:46:33
Subject Test	0.149*	13:49:42
Air Blank	0.000	13:50:14
Air Blank	0.000	13:50:41
Subject Test	XXX**	13:51:04
Air Blank	0.000	13:51:39

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-001108
11/21/2007

13:51:57

5 of 7sc
154

Standard Lot#: 719801I
Last Changed By: C. CONLEY #34517

QAS: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	13:52:24
Cal Check	0.097	13:52:43
Air Blank	0.000	13:53:13

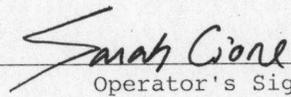

Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6 of 7sc

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

By

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Sarah Cione AGENCY Tucson Police Department
DATE 11/21/07 TIME 13:36
Intoxilyzer Serial # 80-001108 Location Team 3

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-001108
11/21/2007 13:36:58

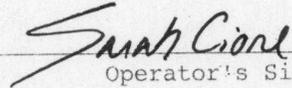
7 of 75C

Standard Lot#: 719801I
Last Changed By: C. CONLEY #34517

B4

QAS: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	13:37:26
Cal Check	0.100	13:37:44
Air Blank	0.000	13:38:14


Operator's Signature