



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory
1306 W. Miracle Mile, Tucson, AZ 85705

TS4
1-28-2013

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-001108 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 1/3/13 Cal 1 0.101 AC Cal 2 0.101 AC

Between the dates of 12/6/12 and 1/3/13 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on -.
The instrument was returned to service on -.

Sarah Cione

Sarah L. Cione


1/7/13

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 7th day of January, 2013.

Edma Escarcega

Notary Public

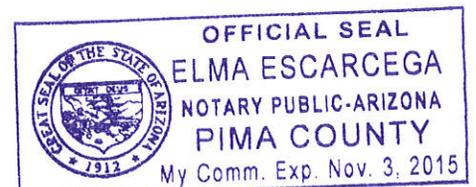

*Includes documents specified in A.A.C. R13-10-104.A.5

1 of 7 sc
TS4
1-28-2013

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 1/3/13 TIME 1506

INTOXILYZER SERIAL # 80-001108 LOCATION Vehicle #437 (ODW parking lot)

() 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 101 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Sarah Cione

2 of 7sc
B4

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
01/03/2013 15:07:50

Standard Lot#: 09312100A1
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SOAP, SOAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:08:44
Diagnostic Test	Pass	15:09:22
Air Blank	0.000	15:09:49
0.100 Cal Check	0.101	15:10:10
Air Blank	0.000	15:10:39
Subject Test	0.000	15:11:03
Air Blank	0.000	15:11:30
Wait	ABT*	
Air Blank	0.000	15:12:06

*Sequence Aborted

Not a Successfully
Completed
Test Sequence

3 of 7sc
B4

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
01/03/2013 15:12:16

Standard Lot#: 09312100A1
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:13:12
Diagnostic Test	Pass	15:13:49
Air Blank	0.000	15:14:16
0.100 Cal Check	0.101	15:14:37
Air Blank	0.000	15:15:07
Subject Test	RF1*	15:15:28
Air Blank	IPS**	15:15:36

*RFI Detect
**Improper Sample

Not a Successfully
Completed
Test Sequence

4 of 7 sc
Bh

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
01/03/2013 15:15:46

Standard Lot#: 09312100A1
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SOAP, SOAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:16:42
Diagnostic Test	Pass	15:17:19
Air Blank	0.000	15:17:46
0.100 Cal Check	0.100	15:18:07
Air Blank	0.000	15:18:36
Subject Test	0.017*	15:21:44
Air Blank	0.000	15:22:14
Air Blank	0.000	15:22:41
Subject Test	XXX**	15:23:06
Air Blank	0.000	15:23:39

*Deficient Sample
**Invalid Sample
Begin new deprivation period

Not a Successfully
Completed
Test Sequence

5 of 7 sc
B4

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
01/03/2013 15:24:56

Standard Lot#: 09312100A1
Last Changed By: S. CIONE #50616

OAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	15:25:25
0.100 Cal Check	0.101	15:25:45
Air Blank	0.000	15:26:15

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6 of 7 sc
TSY
1-28-2013

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Sarah Cione AGENCY Tucson Police Department

DATE 1/3/13 TIME 1506

Intoxilyzer Serial # 80-001108 Location vehicle #437 (ODW parking lot)

(✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.

OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C

(✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

(✓) 4. Air blank completed.

(✓) 5. Calibration check completed. Test results 0. 101 AC.

(✓) 6. Air blank completed.

(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

7 of 7sc
Bh

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
01/03/2013 15:06:16

Standard Lot#: 09312100A1
Last Changed By: S. CIONE #50616

OAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	15:06:45
0.100 Cal Check	0.101	15:07:06
Air Blank	0.000	15:07:35

Preventative Maintenance
Performed -- Timer Reset