



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory
1306 W. Miracle Mile, Tucson, AZ 85705

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-001108 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

sc
11/9/13

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 10.17.2013 Cal 1 0.102 AC Cal 2 0.101 AC

Between the dates of 9.18.2013 and 10.17.2013 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on —.
The instrument was returned to service on —.

Terry Gallegos
Terry S. Gallegos


11.7.2013

ADPS QAS Permit # 33169

SUBSCRIBED AND SWORN to before me this 7, day of November, 2013.

A. M. Hernandez

Notary Public

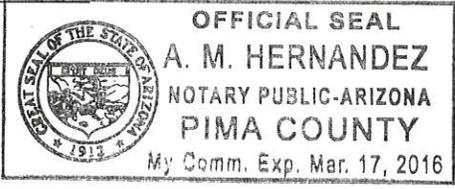

*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

10 of 7
B4
SC
11/19/13

QA SPECIALIST Terry Gallegos 28847 AGENCY Tucson Police Department

DATE 10-17-2013 TIME 1041

INTOXILYZER SERIAL # 80-001108 LOCATION ODD

() 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 101 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Terry Gallegos

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
10/17/2013

10:41:02

20 of 7
Bh SC

Standard Lot#: 09312100A1
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:42:03
Diagnostic Test	Pass	10:42:38
Air Blank	0.000	10:43:05
0.100 Cal Check	0.100	10:43:26
Air Blank	0.000	10:43:56
Subject Test	0.000	10:44:56
Air Blank	0.000	10:45:23
Wait	ABT*	
Air Blank	0.000	10:45:59

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
10/17/2013

10:46:24

3057
D4
5c

Standard Lot#: 09312100A1
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:47:41
Diagnostic Test	Pass	10:48:16
Air Blank	0.000	10:48:43
0.100 Cal Check	0.101	10:49:04
Air Blank	RFI*	10:49:26
Air Blank	IPS**	10:49:34

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
10/17/2013 10:49:39

4 of 7
By

50

Standard Lot#: 09312100A1
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:50:31
Diagnostic Test	Pass	10:51:06
Air Blank	0.000	10:51:33
0.100 Cal Check	0.101	10:51:54
Air Blank	0.000	10:52:24
Subject Test	0.136*	10:55:31
Air Blank	0.000	10:56:03
Air Blank	0.000	10:56:31
Subject Test	XXX**	10:56:52
Air Blank	0.000	10:57:25

*Deficient Sample
**Invalid Sample
Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
10/17/2013 10:57:46

507
B4
SC

Standard Lot#: 09312100A1
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	10:58:16
0.100 Cal Check	0.101	10:58:37
Air Blank	0.000	10:59:07

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

6 of 7
TSK SC
11/19/13

QA Specialist Terry Gallegos 28847 AGENCY Tucson Police Department
DATE 10.17.2013 TIME 1041
Intoxilyzer Serial # 80-001108 Location ODD

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 102 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

DPS Form Exh G-2 (Rev. 05-01)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
10/17/2013 10:41:05

7 of 7
TS4
SC

Standard Lot#: 09312100A1
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Test	g/210L	Time
Air Blank	0.000	10:41:33
0.100 Cal Check	0.102	10:41:54
Air Blank	0.000	10:42:23



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

INTOXILYZER SERVICE RECORD
FOR
CALIBRATION GAS TANK CHANGE

10 of 2
P4

Date: 10-14-2013
QAS: Terry Gallegos 28847
Intoxilyzer Serial Number: 80-001108
New Lot Number: 09312100A1
New Expiration Date: 06-01-2014

SC
11/19/13

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."
Select "D" for dry gas and enter the following information:
 - Enter Target Value: 0.100
 - Enter Control Lot #: From the tank label
 - Enter Exp Date: From the tank label (mm/dd/yyyy)
 - Edit info Y/N? Y or N
 - Update Std Info? Y ENTER
 - "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."
Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 100 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Terry Gallegos

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
10/14/2013 18:05:56

2 of 2
TS4
SC

Standard Lot#: 09312100A1
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Test	g/210L	Time
Air Blank	0.000	18:06:23
0.100 Cal Check	0.100	18:06:44
Air Blank	0.000	18:07:14



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

INTOXILYZER SERVICE RECORD
FOR
CALIBRATION GAS TANK CHANGE

10 of 3
TS4
sc
11/19/13

Date: 10.17.2013
 QAS: Terry Gallegos 28847
 Intoxilyzer Serial Number: 80-001108
 New Lot Number: 30412100A1
 New Expiration Date: 12.1.2014

Tank change due to a partial tank installed on 10/14/2013

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."
 Select "D" for dry gas and enter the following information:
 Enter Target Value: 0.100
 Enter Control Lot #: From the tank label
 Enter Exp Date: From the tank label (mm/dd/yyyy)
 Edit info Y/N? Y or N
 Update Std Info? Y ENTER
 "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."
 Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 099 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove printed record. Attach the record to the completed checklist.

SIGNATURE *Terry Gallegos*

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
10/17/2013 11:09:11

2 of 3
TJ sc

Standard Lot#: 09312100A1
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	11:09:42
0.100 Cal Check	0.099	11:10:02
Air Blank	0.000	11:10:32

Preventative Maintenance NOT
Performed -- Timer NOT Reset

Before tank change
TJ 28847

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
10/17/2013 11:11:52

30f3
154
SC

Standard Lot#: 30412100A1
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	11:12:22
0.100 Cal Check	0.099	11:12:43
Air Blank	0.000	11:13:13

Preventative Maintenance NOT
Performed -- Timer NOT Reset

After tank change By 28847