



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory
1306 W. Miracle Mile, Tucson, AZ 85705

SC
12/12/14

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-001108 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 11.13.2014 Cal 1 0.097 AC Cal 2 0.096 AC

Between the dates of 10.15.2014 and 11.13.2014 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on —.
The instrument was returned to service on —.

Terry S. Gallegos

Terry S. Gallegos

12-12-2014

ADPS QAS Permit # 33169

SUBSCRIBED AND SWORN to before me this 12th, day of December, 2014.

Elma Escarcega

Notary Public

*Includes documents specified in A.A.C. R13-10-104.A.5

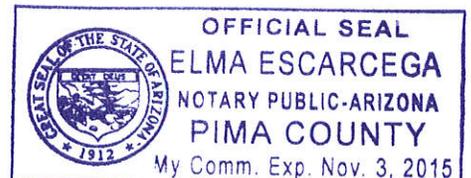


EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

1 of 7
TS4

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

SC
12/12/14

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Terry Gallegos 28847 AGENCY Tucson Police Department

DATE 11-13-2014 TIME 1212

INTOXILYZER SERIAL # 80-001108 LOCATION ODM-Sgt's Office

() 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 096 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE 

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

2077
154

5C

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
11/13/2014 12:14:10

Standard Lot #: 25913100A2
Last Changed By: S. CIONE #50616

DAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SOAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Depriuation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 12:15:38 |
| Diagnostic Test | Pass | 12:16:16 |
| Air Blank | 0.000 | 12:16:43 |
| 0.100 Cal Check | 0.098 | 12:17:04 |
| Air Blank | 0.000 | 12:17:34 |
| Subject Test | 0.000 | 12:17:51 |
| Air Blank | 0.000 | 12:18:18 |
| Wait | ABT* | |
| Air Blank | 0.000 | 12:18:53 |

*Sequence Aborted

Not a Successfully
Completed
Test Sequence

3 of 7
TS4
sc

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
11/13/2014 12:19:02

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

OAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SOAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 12:20:07 |
| Diagnostic Test | Pass | 12:20:44 |
| Air Blank | 0.000 | 12:21:11 |
| 0.100 Cal Check | RFI* | 12:21:26 |
| Air Blank | IPS** | 12:21:36 |

*RFI Detect
**Improper Sample

Not a Successfully
Completed
Test Sequence

4 of 7
TS4
SC

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
11/13/2014 12:22:10

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST,SOAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 12:23:01 |
| Diagnostic Test | Pass | 12:23:38 |
| Air Blank | 0.000 | 12:24:06 |
| 0.100 Cal Check | 0.097 | 12:24:26 |
| Air Blank | 0.000 | 12:24:56 |
| Subject Test | 0.222* | 12:28:03 |
| Air Blank | 0.000 | 12:28:36 |
| Air Blank | 0.000 | 12:29:04 |
| Subject Test | XXX** | 12:29:22 |
| Air Blank | 0.000 | 12:29:55 |

*Deficient Sample
**Invalid Sample
Begin new deprivation period

Not a Successfully
Completed
Test Sequence

50f7
TS4

SC
12/12/14

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
11/13/2014 12:30:30

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

DAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 12:31:02 |
| 0.100 Cal Check | 0.096 | 12:31:23 |
| Air Blank | 0.000 | 12:31:52 |

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

60F7
TS4

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

5C
12/12/14

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Terry Gallegos 28847 AGENCY Tucson Police Department
DATE 11.13.2014 TIME 1212
Intoxilyzer Serial # 80-001108 Location ODM-Sgt's Office

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 097 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Terry Gallegos

DPS Form Exh G-2 (Rev. 05-01)

Fof7
TS4

SC

INTOXILYZER 8000
Instrument Initialization
11:35:41 11/13/2014

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
11/13/2014 12:12:18

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 12:12:48 |
| 0.100 Cal Check | 0.097 | 12:13:09 |
| Air Blank | 0.000 | 12:13:39 |

Preventative Maintenance
Performed -- Timer Reset