



CERTIFICATION by CUSTODIAN of RECORDS
 Tucson Police Department Crime Laboratory
 1306 W. Miracle Mile, Tucson, AZ 85705

TS4
 7.2.2015

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-001108 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 6-25-2015 Cal 1 0.097 AC Cal 2 0.096 AC

Between the dates of 5-28-2015 and 6-25-2015 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on _____.
 The instrument was returned to service on _____.

Amy L. Autret
 Amy L. Autret


7/2/15

ADPS QAS Permit # 33718

SUBSCRIBED AND SWORN to before me this 2nd, day of July, 2015.

Elma Escarcega

Notary Public

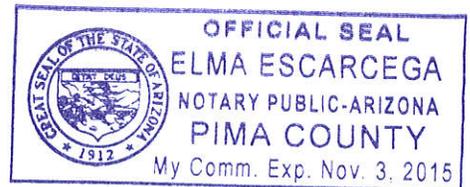

*Includes documents specified in A.A.C. R13-10-104.A.5

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

1087
TS4
7-2-2015

QA SPECIALIST Amy Autret 53546 AGENCY Tucson Police Department

DATE 6-25-2015 TIME 1345

INTOXILYZER SERIAL # 80-001108 LOCATION Crime Laboratory

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.


OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 096 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE

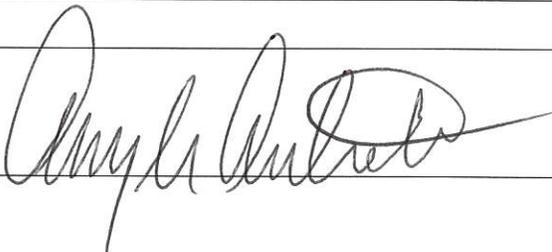

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
06/25/2015

13:47:20

2 of 7
JL
TS4

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:48:15
Diagnostic Test	Pass	13:48:50
Air Blank	0.000	13:49:17
0.100 Cal Check	0.097	13:49:38
Air Blank	0.000	13:50:08
Subject Test	0.000	13:50:22
Air Blank	0.000	13:50:50
Wait	ABT*	
Air Blank	0.000	13:51:24

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
06/25/2015

13:51:58

3 of 7
JL
B4

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:52:50
Diagnostic Test	Pass	13:53:25
Air Blank	0.000	13:53:52
0.100 Cal Check	RFI*	13:54:05
Air Blank	IPS**	13:54:13

*RFI Detect

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
06/25/2015

13:55:33

4 of 7
[Signature]
B4

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:56:31
Diagnostic Test	Pass	13:57:06
Air Blank	0.000	13:57:33
0.100 Cal Check	0.097	13:57:54
Air Blank	0.000	13:58:24
Subject Test	0.190*	14:01:31
Air Blank	0.000	14:02:03
Air Blank	0.000	14:02:31
Subject Test	XXX**	14:02:51
Air Blank	0.000	14:03:25

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
06/25/2015 14:03:53

5 of 7
JLD
T34

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	14:04:22
0.100 Cal Check	0.096	14:04:43
Air Blank	0.000	14:05:13

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

*W of J
JLD*

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

*TBJ
7.2.2015*

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Amy Autret 53546 AGENCY Tucson Police Department

DATE 6-25-2015 TIME 1345

Intoxilyzer Serial # 80-001108 Location Crime Laboratory

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 097 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

DPS Form Exh G-2 (Rev. 05-01)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-001108
Core Version: 8105.48
06/25/2015 13:45:22

2 of 7
JLB
TS4

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	13:45:53
0.100 Cal Check	0.097	13:46:14
Air Blank	0.000	13:46:43

Preventative Maintenance
Performed -- Timer Reset