



2/27

CERTIFICATION by CUSTODIAN of RECORDS
Tucson Police Department Crime Laboratory

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-001163 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 1.26.2007 Cal 1 0.097 AC Cal 2 0.097 AC

Between the dates of 12.28.2006 and 1.26.2007 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on —.
The instrument was returned to service on —.


Terry S. Gallegos
Terry S. Gallegos

2.27.2007

ADPS QAS Permit # 33169

SUBSCRIBED AND SWORN to before me this 27, day of

February, 2007.

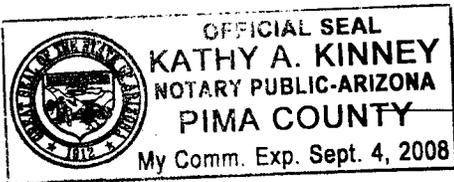

Kathy A Kinney
Notary Public

*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

1 of 10
BY [signature]

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Terry Gallegos 28847 AGENCY Tucson Police Department

DATE 1-26-2007 TIME 1515

INTOXILYZER SERIAL # 80-00 1163 LOCATION DUI HQ

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0.000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed ✓✓✓
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed ✓
Deficient sample printed. ✓
- (✓) 4. Standard Calibration Check standard 0.100 AC
Result: 0.097 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE [Signature]

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001163
01/26/2007 15:17:45

2 of 10
Dy mm

Standard Lot#: 502602I
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Operator: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Subject: SQAP, SQAP
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:18:45
Diagnostic Test	Pass	15:19:13
Air Blank	0.000	15:19:40
0.100 Cal Check	0.097	15:19:59
Air Blank	0.000	15:20:29
Subject Test	0.000	15:21:19
Air Blank	0.000	15:21:46
Five Minute Wait	ABT*	
Air Blank	0.000	15:22:25

*Sequence Aborted

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001163
01/26/2007

15:22:55

7ms
3 of 10
TSL

Standard Lot#: 502602I
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Operator: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Subject: SQAP, SQAP
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:23:51
Diagnostic Test	Pass	15:24:18
Air Blank	RFI*	15:24:28
Air Blank	IPS**	15:24:44

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001163
01/26/2007 15:26:26

7MT
4 of 10
TOL

Standard Lot#: 502602I
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Operator: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Subject: SQAP, SQAP
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:27:22
Diagnostic Test	Pass	15:27:49
Air Blank	0.000	15:28:17
0.100 Cal Check	0.097	15:28:35
Air Blank	0.000	15:29:05
Subject Test	XXX*	15:29:31
Air Blank	0.000	15:30:04

*Invalid Sample
Begin new deprivation period

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001163
01/26/2007 15:30:11

744
50f 10
TS4

Standard Lot#: 502602I
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Operator: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Subject: SQAP, SQAP
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:31:05
Diagnostic Test	Pass	15:31:33
Air Blank	0.000	15:32:00
0.100 Cal Check	0.097	15:32:19
Air Blank	0.000	15:32:48
Subject Test	0.029*	15:35:57
Air Blank	0.000	15:36:29
Air Blank	0.000	15:36:56
Subject Test	DEF**	15:40:05
Air Blank	0.000	15:40:33

*Deficient Sample
**Deficient Sample

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

*Fof 10th MS
TS*

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Terry Gallegos 28847 AGENCY Tucson Police Department
DATE 1-26-2007 TIME 1515
Intoxilyzer Serial # 80-001163 Location DUI HQ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 097 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE *Terry Gallegos*

DPS Form Exh G-2 (Rev. 05-01)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

9 of 10
TS4

INTOXILYZER SERVICE RECORD
FOR
CALIBRATION GAS TANK CHANGE

Date: 1-26-2007
 QAS: Terry Gallegos 28847
 Intoxilyzer Serial Number: 80-001163
 New Lot Number: 507501G
 New Expiration Date: 3-18-2007

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."
 Select "D" for dry gas and enter the following information:
 Enter Target Value: 0.100
 Enter Control Lot #: From the tank label
 Enter Exp Date: From the tank label (mm/dd/yyyy)
 Edit info Y/N? Y or N
 Update Std Info? Y ENTER
 "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."
 Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 097 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove and sign printed record. Attach the record to the completed checklist.

SIGNATURE Terry Gallegos

INTOXILYZER 8000

Location: TUCSON POLICE DEPT

Serial Number: 80-001163

01/26/2007

15:48:28

10 of 10
TJG

Standard Lot#: 507501G

Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847

TUCSON PD CRIME LAB

Test	g/210L	Time
Air Blank	0.000	15:48:55
Cal Check	0.097	15:49:14
Air Blank	0.000	15:49:43


Operator's Signature