



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory  
1306 W. Miracle Mile, Tucson, AZ 85705

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-001163 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

TS4  
5.7.2015

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 4/30/2015 Cal 1 0.097 AC Cal 2 0.097 AC

Between the dates of 4/1/2015 and 4/30/2015 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on —.  
The instrument was returned to service on —.

[Signature]  
Amy L. Autret


5-5-2015

ADPS QAS Permit # 33718

SUBSCRIBED AND SWORN to before me this 5<sup>th</sup>, day of May, 20 15.

[Signature]

Notary Public

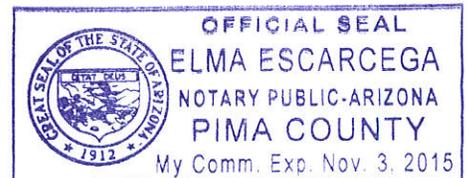

\*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

10F7  
ML  
TSY  
5.7.2015

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Amy Autret 53546 AGENCY Tucson Police Department

DATE 4/30/2015 TIME 1401

INTOXILYZER SERIAL # 80-001163 LOCATION Crime Lab

(  ) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.
- (  ) 2. Date check.

OPERATIONAL TESTS


- (  ) 1. Alcohol-free subject test result 0. 000 AC.
- (  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (  ) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 097 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE

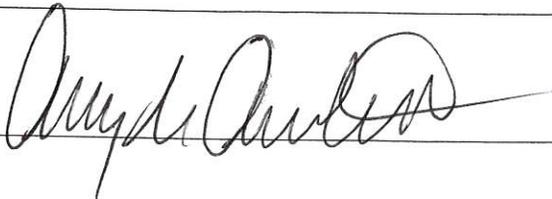

2017  
JLH  
TS4

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-001163  
Core Version: 8105.48  
04/30/2015 14:04:44

Standard Lot#: 11514100A2  
Last Changed By: T. GALLEGOS #28847

QAS: A. AUTRET #53546  
TUCSON PD

Operator: A. AUTRET #53546  
TUCSON PD

Subject: SOAP, SOAP  
DOB: 11/11/1987  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	14:05:55
Diagnostic Test	Pass	14:06:32
Air Blank	0.000	14:06:59
0.100 Cal Check	0.097	14:07:20
Air Blank	0.000	14:07:50
Subject Test	0.000	14:08:15
Air Blank	0.000	14:08:42
Wait	ABT*	
Air Blank	0.000	14:09:25

\*Sequence Aborted

Not a Successfully  
Completed  
Test Sequence

3 of 7  
Alt  
TS4

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-001163  
Core Version: 8105.48  
04/30/2015 14:10:40

-----  
Standard Lot#: 11514100A2  
Last Changed By: T. GALLEGOS #28847  
-----

DAS: A. AUTRET #53546  
TUCSON PD  
-----

Operator: A. AUTRET #53546  
TUCSON PD  
-----

Subject: SQAP,SQAP  
DOB: 11/11/1987  
Sex: F Weight: 111  
-----

15 Minute Deprivation Period? Yes  
-----

Test	g/210L	Time
Air Blank	0.000	14:11:45
Diagnostic Test	Pass	14:12:23
Air Blank	0.000	14:12:50
0.100 Cal Check	RFI*	14:13:03
Air Blank	IPS**	14:13:11

\*RFI Detect  
\*\*Improper Sample

Not a Successfully  
Completed  
Test Sequence

4 of 7  
MS  
TSh

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-001163  
Core Version: 8105.48  
04/30/2015 14:14:16

-----  
Standard Lot#: 11514100A2  
Last Changed By: T. GALLEGOS #28847  
-----

OAS: A. AUTRET #53546  
TUCSON PD  
-----

Operator: A. AUTRET #53546  
TUCSON PD  
-----

Subject: SOAP, SOAP  
DOB: 11/11/1987  
Sex: F Weight: 111  
-----

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	14:15:29
Diagnostic Test	Pass	14:16:06
Air Blank	0.000	14:16:34
0.100 Cal Check	0.097	14:16:54
Air Blank	0.000	14:17:24
Subject Test	0.093*	14:20:31
Air Blank	0.000	14:21:03
Air Blank	0.000	14:21:31
Subject Test	XXX**	14:21:56
Air Blank	0.000	14:22:29

\*Deficient Sample  
\*\*Invalid Sample  
Begin new deprivation period

Not a Successfully  
Completed  
Test Sequence

5 of 7  
ML  
TS4

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-001163  
Core Version: 8105.48  
04/30/2015 14:23:32

-----  
Standard Lot#: 11514100A2  
Last Changed By: T. GALLEGOS #28847  
-----

QAS: A. AUTRET #53546  
TUCSON PD  
-----

Operator: A. AUTRET #53546  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	14:24:03
0.100 Cal Check	0.097	14:24:24
Air Blank	0.000	14:24:53

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

leaf 7  
MA

ARIZONA DEPARTMENT OF PUBLIC SAFETY


STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

TJ  
5.7.2015

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Amy Axtret 53546 AGENCY Tucson Police Department  
DATE 4/30/2015 TIME 1401  
Intoxilyzer Serial # 80-001163 Location Crime Lab

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 097 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

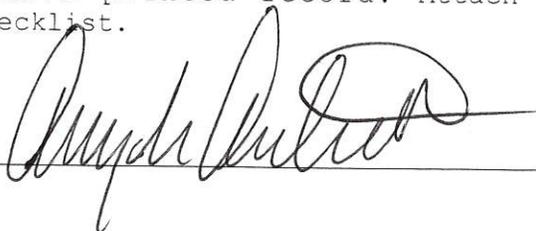SIGNATURE 

DPS Form Exh G-2 (Rev. 05-01)

2 of 7  
MA  
TJL

INTOXILYZER 8000  
Instrument Initialization  
13:37:35 04/30/2015

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-001163  
Core Version: 8105.48  
04/30/2015 14:01:42

-----  
Standard Lot#: 11514100A2  
Last Changed By: T. GALLEGOS #28847  
-----

QAS: A. AUTRET #53546  
TUCSON PD  
-----

Operator: A. AUTRET #53546  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	14:02:13
0.100 Cal Check	0.097	14:02:34
Air Blank	0.000	14:03:03

Preventative Maintenance  
Performed -- Timer Reset