

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

1 of 7
[Signature]

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

TJH
8-25-2015

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Amy Autret 53546 AGENCY Tucson Police Department

DATE 8/20/15 TIME 1252

INTOXILYZER SERIAL # 80-001163 LOCATION crime lab

() 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 095 AC

Instrument is operating properly and accurately. YES NO

COMMENTS _____

SIGNATURE [Signature]

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

2 of 7
JL
TSh

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-001163
Core Version: 8105.48
08/20/2015 12:55:55

Standard Lot#: 11514100A2
Last Changed By: T. GALLEGOS #28847

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Subject: SOAP, SOAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	12:57:14
Diagnostic Test	Pass	12:57:51
Air Blank	0.000	12:58:19
0.100 Cal Check	0.094	12:58:40
Air Blank	0.000	12:59:09
Subject Test	0.000	12:59:53
Air Blank	0.000	13:00:20
Wait	ABT*	
Air Blank	0.000	13:00:55

*Sequence Aborted

Not a Successfully
Completed
Test Sequence

3 of 7
AD
TBH

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-001163
Core Version: 8105.48
08/20/2015 13:02:13

Standard Lot#: 11514100A2
Last Changed By: T. GALLEGOS #28847

OAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:03:40
Diagnostic Test	Pass	13:04:17
Air Blank	0.000	13:04:45
Reference	RFI*	13:04:52
Air Blank	IPS**	13:05:09

*RFI Detect
**Improper Sample

Not a Successfully
Completed
Test Sequence

4 of 7
Ad
B4

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-001163
Core Version: 8105.48
08/20/2015 13:07:00

Standard Lot#: 11514100A2
Last Changed By: T. GALLEGOS #28847

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Subject: SOAP, SOAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:08:22
Diagnostic Test	Pass	13:08:59
Air Blank	0.000	13:09:26
0.100 Cal Check	0.095	13:09:47
Air Blank	0.000	13:10:17
Subject Test	0.097*	13:13:24
Air Blank	0.000	13:13:56
Air Blank	0.000	13:14:23
Subject Test	XXX**	13:14:58
Air Blank	0.000	13:15:32

*Deficient Sample
**Invalid Sample
Begin new deprivation period

Test Sequence

5047
JL
TS

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-001163
Core Version: 8105.48
08/20/2015 13:17:49

Standard Lot#: 11514100A2
Last Changed By: T. GALLEGOS #28847

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	13:18:19
0.100 Cal Check	0.095	13:18:40
Air Blank	0.000	13:19:10

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

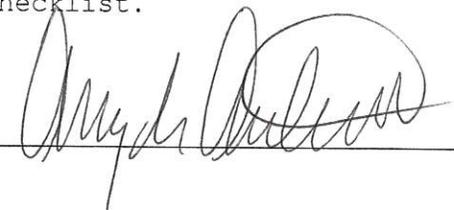
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

leaf 7
Bh
8.25.2015

QA Specialist Amy A. Trot 53546 AGENCY Tucson Police Department
DATE 8/20/2015 TIME 1252
Intoxilyzer Serial # 80-001163 Location Crim Labs

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 095 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

2 of 7
[Signature]
TS4

INTOXILYZER 8000
Instrument Initialization
09:39:12 08/20/2015

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-001163
Core Version: 8105.48
08/20/2015 12:52:20

Standard Lot#: 11514100A2
Last Changed By: T. GALLEGOS #28847

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	12:53:07
0.100 Cal Check	0.095	12:53:27
Air Blank	0.000	12:53:57

Preventative Maintenance
Performed -- Timer Reset