



CERTIFICATION by CUSTODIAN of RECORDS  
 Tucson Police Department Crime Laboratory  
 270 S. Stone Avenue Tucson, AZ 85701-1917

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-001165 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

TSY  
 9-27-2011

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 08/25/11 Cal 1 0.096 AC Cal 2 0.096 AC

Between the dates of 07/28/11 and 08/25/11 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on N/A.  
 The instrument was returned to service on N/A.

Cheryl K. Straub

Cheryl K. Straub


09/08/2011

ADPS QAS Permit # 33408

SUBSCRIBED AND SWORN to before me this 8 day of Sept, 2011.

Ann L. Brecroft

Notary Public

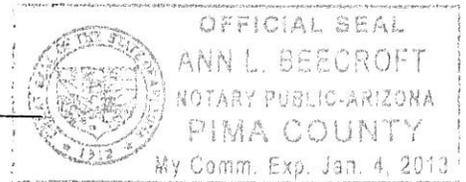

\*Includes documents specified in A.A.C. R13-10-104.A.5

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

188  
CS

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

TSY  
9-27-2011

QA SPECIALIST Cheryl STRAUB AGENCY Tucson Police Department

DATE 08/25/2011 TIME 08:50

INTOXILYZER SERIAL # 80-001165 LOCATION TEAM 1

(  ) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.
- (  ) 2. Date check.

OPERATIONAL TESTS

- (  ) 1. Alcohol-free subject test result 0. 000 AC.
- (  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (  ) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 096 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS INVALID Sample test WAS REPEATED, FIRST TEST  
WAS NOT SUCCESSFUL.  
DEFICIENT Sample WAS NOT PERFORMED

SIGNATURE Cheryl Straub

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

288  
CS

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-001165  
08/25/2011

BM

08:51:48

Standard Lot#: 04911100A1  
Last Changed By: C. STRAUB #50772

QAS: C. STRAUB #50772  
TUCSON PD

Operator: C. STRAUB #50772  
TUCSON PD

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:52:35
Diagnostic Test	Pass	08:53:02
Air Blank	0.000	08:53:30
0.100 Cal Check	0.096	08:53:48
Air Blank	0.000	08:54:19
Subject Test	0.000	08:54:42
Air Blank	ABT*	08:54:49
Air Blank	0.000	08:55:17

\*Sequence Aborted

### Not a Successfully Completed Test Sequence

378  
CS

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-001165  
08/25/2011 08:55:25

BSH

Standard Lot#: 04911100A1  
Last Changed By: C. STRAUB #50772

QAS: C. STRAUB #50772  
TUCSON PD

Operator: C. STRAUB #50772  
TUCSON PD

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	08:55:57
Air Blank	IPS**	08:56:06

\*RFI Detect  
\*\*Improper Sample

### Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

498  
CS

Bh

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-001165  
08/25/2011 08:56:15

Standard Lot#: 04911100A1  
Last Changed By: C. STRAUB #50772

QAS: C. STRAUB #50772  
TUCSON PD

Operator: C. STRAUB #50772  
TUCSON PD

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:57:05
Diagnostic Test	Pass	08:57:33
Air Blank	0.000	08:58:00
0.100 Cal Check	0.096	08:58:19
Air Blank	0.000	08:58:49
Subject Test	0.141	08:59:23
Air Blank	0.000	08:59:56
Five Minute Wait	ABT*	
Air Blank	0.000	09:00:45

\*Sequence Aborted

### Not a Successfully Completed Test Sequence

598  
CS

TS4

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-001165  
08/25/2011 09:01:06

Standard Lot#: 04911100A1  
Last Changed By: C. STRAUB #50772

QAS: C. STRAUB #50772  
TUCSON PD

Operator: C. STRAUB #50772  
TUCSON PD

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:01:55
Diagnostic Test	Pass	09:02:23
Air Blank	0.000	09:02:50
0.100 Cal Check	0.095	09:03:09
Air Blank	0.000	09:03:39
Subject Test	XXX*	09:03:57
Air Blank	0.000	09:04:31

\*Invalid Sample  
Begin new deprivation period

### Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

678  
CS

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-001165  
08/25/2011 09:04:51

TS4

-----  
Standard Lot#: 04911100A1  
Last Changed By: C. STRAUB #50772  
-----

QAS: C. STRAUB #50772  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	09:05:19
Cal Check	0.096	09:05:38
Air Blank	0.000	09:06:08



Operator's Signature

790cs

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

B4  
9-27-2011

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Cheryl STRAUB AGENCY Tucson Police Department  
DATE 08/25/2011 TIME 08:50  
Intoxilyzer Serial # 80-001165 Location TEAM 1

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution N/A AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 096 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Cheryl Straub

DPS Form Exh G-2 (Rev. 05-01)

888  
CS


INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-001165  
08/25/2011 08:50:14

-----  
Standard Lot#: 04911100A1  
Last Changed By: C. STRAUB #50772  
-----

TS4

-----  
QAS: C. STRAUB #50772  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	08:50:41
Cal Check	0.096	08:51:00
Air Blank	0.000	08:51:30

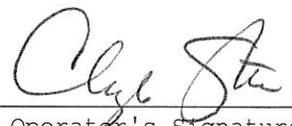

Operator's Signature



TUCSON POLICE DEPARTMENT  
CRIME LABORATORY

INTOXILYZER SERVICE RECORD  
FOR  
CALIBRATION GAS TANK CHANGE

BM  
9.27.2011

Date: 08/25/2011  
 QAS: Cheryl STRAUB  
 Intoxilyzer Serial Number: 80-001165  
 New Lot Number: 0491100A1  
 New Expiration Date: 03/01/2013

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."  
 Select "D" for dry gas and enter the following information:  
     Enter Target Value: 0.100  
     Enter Control Lot #: From the tank label  
     Enter Exp Date: From the tank label (mm/dd/yyyy)  
     Edit info Y/N? Y or N  
     Update Std Info? Y ENTER  
     "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "Control Testing."  
 Select "D" for dry control test. After selection is made press  
 ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 096 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Cheryl Straub

INTOXILYZER 8000

Location: TUCSON POLICE DEPT

Serial Number: 80-001165

08/25/2011

08:48:35

*TSK*


-----  
Standard Lot#: 04911100A1

Last Changed By: C. STRAUB #50772  
-----

QAS: C. STRAUB #50772

TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	08:49:02
Cal Check	0.096	08:49:21
Air Blank	0.000	08:49:51

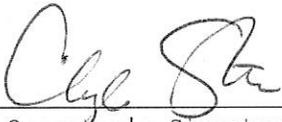

Operator's Signature