



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory  
1306 W. Miracle Mile, Tucson, AZ 85705

T34  
12-27-2013

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-001165 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 12/5/13 Cal 1 0.098 AC Cal 2 0.099 AC

Between the dates of 11/8/13 and 12/5/13 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on -.  
The instrument was returned to service on -.

Sarah Cione

Sarah L. Cione


12/19/13

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 19th, day of December, 2013.

Edma Escarcega

Notary Public

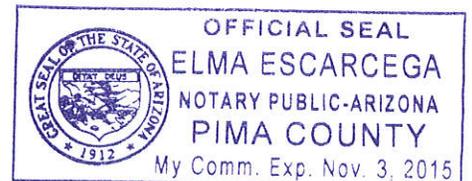

\*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

1 of 9 sc  
TS4  
12-27-2013

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 12/5/13 TIME 1057

INTOXILYZER SERIAL # 80-001165 LOCATION DUI HQ

(  ) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.
- (  ) 2. Date check.

OPERATIONAL TESTS

- (  ) 1. Alcohol-free subject test result 0. 000 AC.
- (  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (  ) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 099 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-001165  
Core Version: 8105.48  
12/05/2013 10:59:17

2 of 9 sc  
TS4

---

Standard Lot#: 30412100A1  
Last Changed By: S. CIONE #50616

---

QAS: S. CIONE #50616  
TUCSON PD

---

Operator: S. CIONE #50616  
TUCSON PD

---

Subject: SQAP, SQAP  
DOB: 11/11/1980  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	11:00:05
Diagnostic Test	Pass	11:00:40
Air Blank	0.000	11:01:08
0.100 Cal Check	0.099	11:01:29
Air Blank	0.000	11:01:58
Subject Test	0.000	11:02:39
Air Blank	0.000	11:03:06
Wait	ABT*	
Air Blank	0.000	11:04:23

\*Sequence Aborted

### Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-001165  
Core Version: 8105.48  
12/05/2013

11:04:33

39<sub>5c</sub>  
154

---

Standard Lot#: 30412100A1  
Last Changed By: S. CIONE #50616

---

QAS: S. CIONE #50616  
TUCSON PD

---

Operator: S. CIONE #50616  
TUCSON PD

---

Subject: SQAP,SQAP  
DOB: 11/11/1980  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	11:05:37
Diagnostic Test	Pass	11:06:12
Air Blank	0.000	11:06:39
0.100 Cal Check	0.098	11:07:00
Air Blank	0.000	11:07:29
Subject Test	RFI*	11:08:02
Air Blank	IPS**	11:08:10

\*RFI Detect

\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-001165  
Core Version: 8105.48  
12/05/2013

11:12:12

4 of 9  
sc  
TS4

---

Standard Lot#: 30412100A1  
Last Changed By: S. CIONE #50616

---

QAS: S. CIONE #50616  
TUCSON PD

---

Operator: S. CIONE #50616  
TUCSON PD

---

Subject: SQAP, SQAP  
DOB: 11/11/1980  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	11:13:04
Diagnostic Test	Pass	11:13:40
Air Blank	0.000	11:14:07
0.100 Cal Check	0.099	11:14:28
Air Blank	0.000	11:14:57
Subject Test	0.146*	11:18:05
Air Blank	0.000	11:18:36
Air Blank	0.000	11:19:04
Subject Test	XXX**	11:19:26
Air Blank	0.000	11:20:00

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: TUCSON POLICE DEPT

Serial Number: 80-001165

Core Version: 8105.48

12/05/2013

11:20:52

5 of 9 sc  
TS4

-----  
Standard Lot#: 30412100A1

Last Changed By: S. CIONE #50616  
-----

QAS: S. CIONE #50616

TUCSON PD  
-----

Operator: S. CIONE #50616

TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	11:21:21
0.100 Cal Check	0.099	11:21:41
Air Blank	0.000	11:22:11

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

6 of 9 sc  
TS4  
12.27.2013

QA Specialist Sarah Cione AGENCY Tucson Police Department  
DATE 12/5/13 TIME 1057  
Intoxilyzer Serial # 80-001165 Location DUI HQ

- ( ✓ ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- ( ✓ ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ( ✓ ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ( ✓ ) 4. Air blank completed.
- ( ✓ ) 5. Calibration check completed. Test results 0. 098 AC.
- ( ✓ ) 6. Air blank completed.
- ( ✓ ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-001165  
Core Version: 8105.48  
12/05/2013 10:57:23

7 of 9 sc  
TS4

-----  
Standard Lot#: 30412100A1  
Last Changed By: S. CIONE #50616  
-----

QAS: S. CIONE #50616  
TUCSON PD  
-----

Operator: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	10:57:52
0.100 Cal Check	0.098	10:58:13
Air Blank	0.000	10:58:42

Preventative Maintenance  
Performed -- Timer Reset



TUCSON POLICE DEPARTMENT  
CRIME LABORATORY

Page 9  
TSK  
12-27-2013

INTOXILYZER SERVICE RECORD  
FOR  
CALIBRATION GAS TANK CHANGE

Date: 12/5/13  
QAS: Sarah Cione  
Intoxilyzer Serial Number: 80-001165  
New Lot Number: 19613100A4  
New Expiration Date: 8/1/15

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."  
Select "D" for dry gas and enter the following information:  
  - Enter Target Value: 0.100
  - Enter Control Lot #: From the tank label
  - Enter Exp Date: From the tank label (mm/dd/yyyy)
  - Edit info Y/N? Y or N
  - Update Std Info? Y ENTER
  - "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."  
Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 100 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

~600psi

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-001165  
Core Version: 8105.48  
12/05/2013 11:24:08

9 of 9 SC  
T34

-----  
Standard Lot#: 19613100A4  
Last Changed By: S. CIONE #50616  
-----

QAS: S. CIONE #50616  
TUCSON PD  
-----

Operator: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	11:24:37
0.100 Cal Check	0.100	11:24:58
Air Blank	0.000	11:25:27

Preventative Maintenance  
Performed -- Timer Reset