



CERTIFICATION by CUSTODIAN of RECORDS  
Tucson Police Department Crime Laboratory

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-001199 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

TS4  
11-3-08

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 10/17/08 Cal 1 0.101 AC Cal 2 0.100 AC

Between the dates of 9/19/08 and 10/17/08 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on -  
The instrument was returned to service on -

Sarah Cione

Sarah L. Cione


10/20/08

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 20<sup>th</sup> day of October, 2008.

A.M. Hernandez

Notary Public

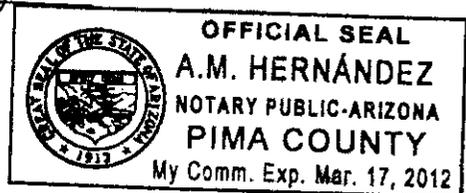

\*Includes documents specified in A.A.C. R13-10-104.A.5

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

1 of 7 sc

TSY  
11.3.08

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department  
DATE 10/17/08 TIME 13:12  
INTOXILYZER SERIAL # 80-001199 LOCATION Crime Lab

- (✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 100 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE Sarah Cione

TS4

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-001199  
10/17/2008 13:15:02

Standard Lot#: 801002I  
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616  
TUCSON PD

Operator: S. CIONE #50616  
TUCSON PD

Subject: SQAP, TEST  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:15:53
Diagnostic Test	Pass	13:16:21
Air Blank	0.000	13:16:48
0.100 Cal Check	0.100	13:17:07
Air Blank	0.000	13:17:36
Subject Test	0.000	13:17:51
Air Blank	0.000	13:18:18
Five Minute Wait	ABT*	
Air Blank	0.000	13:19:02

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

3 of 7 sc

B4

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-001199  
10/17/2008

13:19:33

---

Standard Lot#: 801002I  
Last Changed By: S. CIONE #50616

---

QAS: S. CIONE #50616  
TUCSON PD

---

Operator: S. CIONE #50616  
TUCSON PD

---

Subject: SQAP, TEST  
DOB: 11/11/1111  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	13:20:40
Air Blank	IPS**	13:20:51

\*RFI Detect  
\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

*Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.*

4 of 7  
SC

TS

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-001199  
10/17/2008 13:21:57

Standard Lot#: 801002I  
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616  
TUCSON PD

Operator: S. CIONE #50616  
TUCSON PD

Subject: SQAP, TEST  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:23:01
Diagnostic Test	Pass	13:23:29
Air Blank	0.000	13:23:56
0.100 Cal Check	0.102	13:24:15
Air Blank	0.000	13:24:44
Subject Test	0.154*	13:27:52
Air Blank	0.000	13:28:24
Air Blank	0.000	13:28:52
Subject Test	XXX**	13:29:42
Air Blank	0.000	13:30:15

\*Deficient Sample  
\*\*Invalid Sample  
Begin new deprivation period

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-001199  
10/17/2008 13:30:40

5 of 7 SC  
PH

-----  
Standard Lot#: 801002I  
Last Changed By: S. CIONE #50616  
-----

QAS: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	13:31:08
Cal Check	0.100	13:31:26
Air Blank	0.000	13:31:56

Sarah Cione  
Operator's Signature

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

6 of 75c

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

TS4  
11.3.08

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Sarah Cione AGENCY Tucson Police Department  
DATE 10/17/08 TIME 13:12  
Intoxilyzer Serial # 80-001199 Location Crime Lab

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0.101 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-001199  
10/17/2008 13:12:33

7 of 7  
SC

134

-----  
Standard Lot#: 801002I  
Last Changed By: S. CIONE #50616  
-----

QAS: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	13:13:00
Cal Check	0.101	13:13:19
Air Blank	0.000	13:13:49

*Sarah Cione*

Operator's Signature