



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory
270 S. Stone Avenue Tucson, AZ 85701-1917

TS4
8-22-2011

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 50-001199 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 8-18-11 Cal 1 0.098 AC Cal 2 0.098 AC

Between the dates of 7-21-11 and 8-18-11 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on N/A.
The instrument was returned to service on N/A.

Clare Conley

Clare L. Conley


8-18-11

ADPS QAS Permit # 33314

SUBSCRIBED AND SWORN to before me this 18th, day of August, 20 11.

Susan A. Hitchye

Notary Public

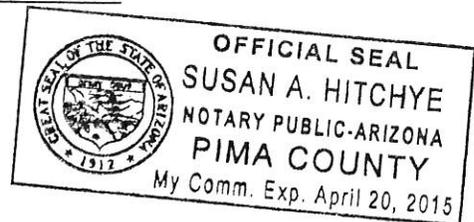

*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

1 of 7
cu
TSH
8.22.2011

QA SPECIALIST Mare Conley AGENCY Tucson Police Department

DATE 8-18-11 TIME 0811

INTOXILYZER SERIAL # SD-001199 LOCATION ODW DUI HQ

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

(✓) 1. Clock time check.

(✓) 2. Date check.

OPERATIONAL TESTS

(✓) 1. Alcohol-free subject test result 0. 000 AC.

(✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed

(✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.

(✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 098 AC

Instrument is operating properly and accurately. YES NO

COMMENTS None

SIGNATURE Mare Conley

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001199
08/18/2011

08:13:20

267
cu
BY

Standard Lot#: 20910100A1
Last Changed By: S. CIONE #50616

QAS: C. CONLEY #34517
TUCSON PD

Operator: C. CONLEY #34517
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1111
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:14:06
Diagnostic Test	Pass	08:14:34
Air Blank	0.000	08:15:01
0.100 Cal Check	0.098	08:15:20
Air Blank	0.000	08:15:49
Subject Test	0.000	08:16:37
Air Blank	ABT*	08:16:43
Air Blank	0.000	08:17:12

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001199
08/18/2011 08:18:21

3567
C
784

Standard Lot#: 20910100A1
Last Changed By: S. CIONE #50616

QAS: C. CONLEY #34517
TUCSON PD

Operator: C. CONLEY #34517
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1111
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	08:18:51
Air Blank	IPS**	08:19:04

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001199
09/19/2011

08:19:49

Standard Lot#: 20910100A1
Last Changed By: S. CIONE #50616

QAS: C. CONLEY #34517
TUCSON PD

Operator: C. CONLEY #34517
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1111
Sex: M Weight: 11

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:20:48
Diagnostic Test	Pass	08:21:16
Air Blank	0.000	08:21:43
0.100 Cal Check	0.098	08:22:02
Air Blank	0.000	08:22:31
Subject Test	0.137*	08:25:39
Air Blank	0.000	08:26:11
Air Blank	0.000	08:26:39
Subject Test	XXX**	08:27:17
Air Blank	0.000	08:27:50

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-001199
08/18/2011 08:28:18

567
CW

Standard Lot#: 20910100A1
Last Changed By: S. CIONE #50616

124

QAS: C. CONLEY #34517
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	08:28:45
Cal Check	0.098	08:29:04
Air Blank	0.000	08:29:33

Clare Conley

Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6-67
CW

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

T54
8.22.2011

QA Specialist Clare Conley AGENCY Tucson Police Department
DATE 8-18-11 TIME 0811
Intoxilyzer Serial # 50-501199 Location ODW DUI HQ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 10 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 098 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Clare Conley

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-001199
08/18/2011 08:11:55

7587
Cu

Standard Lot#: 20910100A1
Last Changed By: S. CIONE #50616

134

QAS: C. CONLEY #34517
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	08:12:23
Cal Check	0.098	08:12:41
Air Blank	0.000	08:13:11



Operator's Signature