



THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

CU  
8.21.07  
10 of 7  
TS4

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Terry Gallegos 28847 AGENCY Tucson Police Department

DATE 7.20.2007 TIME 1052

INTOXILYZER SERIAL # 30-002201 LOCATION Lab

(  ) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.
- (  ) 2. Date check.

OPERATIONAL TESTS

- (  ) 1. Alcohol-free subject test result 0. 000 AC.
- (  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed ✓✓
- (  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (  ) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 098 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS

---



---



---



---

SIGNATURE

Terry Gallegos

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-002201  
07/20/2007

10:59:44

2 of 7  
TB

---

Standard Lot#: 627101G  
Last Changed By: J. OLIVARES #36552

---

QAS: T. GALLEGOS #28847  
TUCSON PD CRIME LAB

---

Operator: T. GALLEGOS #28847  
TUCSON PD CRIME LAB

---

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

| Test             | g/210L | Time     |
|------------------|--------|----------|
| Air Blank        | 0.000  | 11:00:33 |
| Diagnostic Test  | Pass   | 11:01:01 |
| Air Blank        | 0.000  | 11:01:28 |
| 0.100 Cal Check  | 0.099  | 11:01:47 |
| Air Blank        | 0.000  | 11:02:16 |
| Subject Test     | 0.000  | 11:02:47 |
| Air Blank        | 0.000  | 11:03:14 |
| Five Minute Wait | ABT*   |          |
| Air Blank        | 0.000  | 11:03:48 |

\*Sequence Aborted

### Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-002201  
07/20/2007

OK

11:04:00

3087  
TS4

---

Standard Lot#: 627101G  
Last Changed By: J. OLIVARES #36552

---

QAS: T. GALLEGOS #28847  
TUCSON PD CRIME LAB

---

Operator: T. GALLEGOS #28847  
TUCSON PD CRIME LAB

---

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

| Test            | g/210L | Time     |
|-----------------|--------|----------|
| Air Blank       | 0.000  | 11:04:46 |
| Diagnostic Test | Pass   | 11:05:14 |
| Air Blank       | 0.000  | 11:05:41 |
| 0.100 Cal Check | 0.098  | 11:06:00 |
| Air Blank       | RFI*   | 11:06:09 |
| Air Blank       | IPS**  | 11:06:18 |

\*RFI Detect  
\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-002201  
07/20/2007

em

11:06:35

40f7  
TS6

Standard Lot#: 627101G  
Last Changed By: J. OLIVARES #36552

QAS: T. GALLEGOS #28847  
TUCSON PD CRIME LAB

Operator: T. GALLEGOS #28847  
TUCSON PD CRIME LAB

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test            | g/210L | Time     |
|-----------------|--------|----------|
| Air Blank       | 0.000  | 11:07:28 |
| Diagnostic Test | Pass   | 11:07:55 |
| Air Blank       | 0.000  | 11:08:23 |
| 0.100 Cal Check | 0.099  | 11:08:41 |
| Air Blank       | 0.000  | 11:09:11 |
| Subject Test    | 0.070* | 11:12:19 |
| Air Blank       | 0.000  | 11:12:52 |
| Air Blank       | 0.000  | 11:13:19 |
| Subject Test    | XXX**  | 11:13:35 |
| Air Blank       | 0.000  | 11:14:09 |

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

### Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000

Location: TUCSON POLICE DEPT

Serial Number: 80-002201

07/20/2007

11:14:26

ck

5077  
TS4

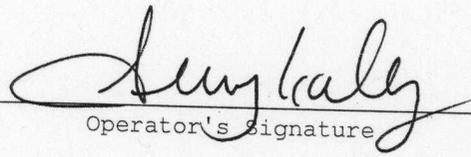
-----  
Standard Lot#: 627101G

Last Changed By: J. OLIVARES #36552

-----  
QAS: T. GALLEGOS #28847

TUCSON PD CRIME LAB

-----  
Test g/210L Time  
-----  
Air Blank 0.000 11:14:53  
Cal Check 0.098 11:15:12  
Air Blank 0.000 11:15:41

  
Operator's Signature

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

*OK 8.21.07*

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

*60 of 7  
184*

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Terry Gallegos 28847 AGENCY Tucson Police Department  
DATE 7-20-2007 TIME 1052  
Intoxilyzer Serial # 80-002201 Location Lab

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results 0. 099 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE *Terry Gallegos*

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002201  
07/20/2007

AM

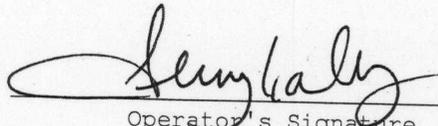
10:52:16

7077  
TS6

-----  
Standard Lot#: 627101G  
Last Changed By: J. OLIVARES #36552  
-----

QAS: T. GALLEGOS #28847  
TUCSON PD CRIME LAB  
-----

| Test      | g/210L | Time     |
|-----------|--------|----------|
| Air Blank | 0.000  | 10:52:43 |
| Cal Check | 0.099  | 10:53:02 |
| Air Blank | 0.000  | 10:53:31 |

  
Operator's Signature



TUCSON POLICE DEPARTMENT  
CRIME LABORATORY

OK 8.21.07

INTOXILYZER SERVICE RECORD  
FOR  
CALIBRATION GAS TANK CHANGE

10f2  
JS4

Date: 07-08-07  
QAS: OLIVARES #36552  
Intoxilyzer Serial Number: 80-002201  
New Lot Number: 6271016  
New Expiration Date: 10-03-2008

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- ( ) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."  
Select "D" for dry gas and enter the following information:  
Enter Target Value: 0.100  
Enter Control Lot #: From the tank label  
Enter Exp Date: From the tank label (mm/dd/yyyy)  
Edit info Y/N? Y or N  
Update Std Info? Y ENTER  
"Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."  
Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 098 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove and sign printed record. Attach the record to the completed checklist.

SIGNATURE  36552

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

90  
4  
760

DM  
2 of 2  
JSH

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002201  
07/08/2007 21:53:35

Standard Lot#: 6271010  
Last Changed By: J. OLIVARES #36552

QAS: J. OLIVARES #36552  
TUCSON PD

| Test      | g/210L | Time     |
|-----------|--------|----------|
| Air Blank | 0.000  | 21:53:14 |
| Cal Check | 0.096  | 21:53:22 |
| Air Blank | 0.000  | 21:53:52 |

\_\_\_\_\_  
Operator's Signature



TUCSON POLICE DEPARTMENT  
 CRIME LABORATORY 8.21-07  
 CW

INTOXILYZER SERVICE RECORD  
 FOR  
 CALIBRATION GAS TANK CHANGE

10 of 2  
 TS4

Date: 08-12-07  
 QAS: OLUARGS, JOSE A.  
 Intoxilyzer Serial Number: 80-002201  
 New Lot Number: 706602 I  
 New Expiration Date: 11 MAR 2009

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."  
 Select "D" for dry gas and enter the following information:  
     Enter Target Value: 0.100  
     Enter Control Lot #: From the tank label  
     Enter Exp Date: From the tank label (mm/dd/yyyy)  
     Edit info Y/N? Y or N  
     Update Std Info? Y ENTER  
     "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."  
 Select "D" for dry control test. After selection is made press  
 ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0.099 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove and sign printed record. Attach the record to the completed checklist.

SIGNATURE

*Jose A. Oluarg*  
 80552

80-002201

2 of 2  
TS4

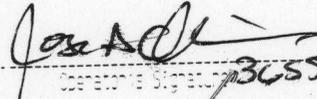
CU

INSTRUMENT 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002201  
10/12/2007 19:35:00

Standard Lot#: 70661021  
Last Charged By: J. OLIVARES #36552

Off: J. OLIVARES #36552  
TUCSON PD

| Test      | g/210L | Time     |
|-----------|--------|----------|
| Air Blank | 0.000  | 19:36:48 |
| Cal Check | 0.099  | 19:37:06 |
| Air Blank | 0.000  | 19:37:36 |

  
Operator's Signature: 36552