



CERTIFICATION by CUSTODIAN of RECORDS
 Tucson Police Department Crime Laboratory
 1306 W. Miracle Mile, Tucson, AZ 85705

Tsh
 7-24-2012

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-002201 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 7/19/12 Cal 1 0.102 AC Cal 2 0.100 AC

Between the dates of 6/21/12 and 7/19/12 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on -.
 The instrument was returned to service on -.

Sarah Cione

Sarah L. Cione


7/20/12

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 20th, day of July, 2012.

Edna Escarcega

Notary Public

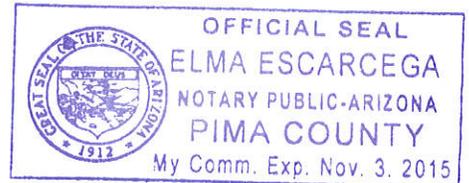

*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

1 of 7_{sc}

TS4
7-24-2012

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 7/19/12 TIME 1015

INTOXILYZER SERIAL # 80-002201 LOCATION DUI HQ

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 100 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Sarah Cione

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002201
Core Version: 8105.48
07/19/2012 10:18:34

2 of 7 sc
T54

Standard Lot#: 23511100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 10:19:17 |
| Diagnostic Test | Pass | 10:19:52 |
| Air Blank | 0.000 | 10:20:20 |
| 0.100 Cal Check | 0.102 | 10:20:41 |
| Air Blank | 0.000 | 10:21:10 |
| Subject Test | 0.000 | 10:21:31 |
| Air Blank | 0.000 | 10:21:59 |
| Wait | ABT* | |
| Air Blank | 0.000 | 10:22:34 |

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002201
Core Version: 8105.48
07/19/2012

10:24:01

3 of 7_{sc}
TSh

Standard Lot#: 23511100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | RFI* | 10:24:26 |
| Air Blank | IPS** | 10:24:33 |

*RFI Detect

**Improper Sample

Not a Successfully Completed Test Sequence

4 of 7 sc
Bh

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002201
Core Version: 8105.48
07/19/2012 10:25:19

Standard Lot#: 23511100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 10:26:11 |
| Diagnostic Test | Pass | 10:26:46 |
| Air Blank | 0.000 | 10:27:13 |
| 0.100 Cal Check | 0.101 | 10:27:34 |
| Air Blank | 0.000 | 10:28:03 |
| Subject Test | 0.055* | 10:31:11 |
| Air Blank | 0.000 | 10:31:43 |
| Air Blank | 0.000 | 10:32:10 |
| Subject Test | XXX** | 10:32:23 |
| Air Blank | 0.000 | 10:32:56 |

*Deficient Sample
**Invalid Sample
Begin new deprivation period

Not a Successfully Completed Test Sequence

5 of 7sc

Bh

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002201
Core Version: 8105.48
07/19/2012 10:33:36

Standard Lot#: 23511100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 10:34:04 |
| 0.100 Cal Check | 0.100 | 10:34:25 |
| Air Blank | 0.000 | 10:34:55 |

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6 of 7_{sc}

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

T34
7-24-2012

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Sarah Cione AGENCY Tucson Police Department
DATE 7/19/12 TIME 1015
Intoxilyzer Serial # 80-002201 Location DUIHQ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 102 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002201
Core Version: 8105.48
07/19/2012 10:15:33

7 of 7 sc
TS4

Standard Lot#: 23511100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 10:16:01 |
| 0.100 Cal Check | 0.102 | 10:16:22 |
| Air Blank | 0.000 | 10:16:51 |

Preventative Maintenance
Performed -- Timer Reset