



CERTIFICATION by CUSTODIAN of RECORDS
 Tucson Police Department Crime Laboratory
 1306 W. Miracle Mile, Tucson, AZ 85705

TL
6.19.2014

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-002201 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 6/19/14 Cal 1 0.099 AC Cal 2 0.098 AC

Between the dates of 5/22/14 and 6/19/14 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on -.
 The instrument was returned to service on -.

Sarah Cione

Sarah L. Cione


6/19/14

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this *19th*, day of *June*, 20 *14*.

Elma Escarcega

Notary Public

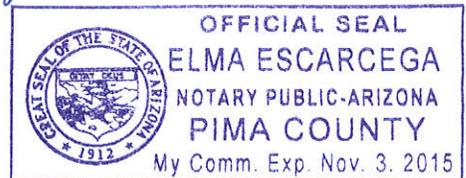

*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

1 of 7 SC
TSY
6-19-2014

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department
DATE 6/19/14 TIME 915
INTOXILYZER SERIAL # 80-002201 LOCATION DUI HQ

() 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 098 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002201
Core Version: 8105.48
06/19/2014

09:18:08

2 of 7
sc
BY

Standard Lot#: 30412100A1
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:19:00
Diagnostic Test	Pass	09:19:35
Air Blank	0.000	09:20:03
0.100 Cal Check	0.099	09:20:23
Air Blank	0.000	09:20:53
Subject Test	0.000	09:21:43
Air Blank	0.000	09:22:10
Wait	ABT*	
Air Blank	0.000	09:23:37

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002201
Core Version: 8105.48
06/19/2014

09:23:54

3 of 7 sc
Bh

Standard Lot#: 30412100A1
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:24:45
Diagnostic Test	Pass	09:25:20
Air Blank	0.000	09:25:47
0.100 Cal Check	0.099	09:26:08
Air Blank	RFI*	09:26:16
Air Blank	IPS**	09:26:25

*RFI Detect

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002201
Core Version: 8105.48
06/19/2014

09:29:44

4 of 7_{sc}

Standard Lot#: 30412100A1
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:30:37
Diagnostic Test	Pass	09:31:12
Air Blank	0.000	09:31:39
0.100 Cal Check	0.100	09:32:00
Air Blank	0.000	09:32:30
Subject Test	0.440*	09:35:37
Air Blank	0.000	09:36:13
Air Blank	0.000	09:36:40
Subject Test	XXX**	09:37:14
Air Blank	0.000	09:37:47

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002201
Core Version: 8105.48
06/19/2014 09:38:05

5 of 7 sc
184

Standard Lot#: 30412100A1
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	09:38:33
0.100 Cal Check	0.098	09:38:54
Air Blank	0.000	09:39:24

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

6 of 7 sc
TS4
6.19.2014

QA Specialist Sarah Cione AGENCY Tucson Police Department
DATE 6/19/14 TIME 915
Intoxilyzer Serial # 80-002201 Location DUI HQ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002201
Core Version: 8105.48
06/19/2014 09:15:01

7 of 7
SC
B4

Standard Lot#: 30412100A1
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	09:15:29
0.100 Cal Check	0.099	09:15:50
Air Blank	0.000	09:16:19

Preventative Maintenance
Performed -- Timer Reset



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

1 of 2 sc
TJH
6-19-2014

INTOXILYZER SERVICE RECORD
FOR
CALIBRATION GAS TANK CHANGE

Date: 6/3/14

QAS: Sarah Cione

Intoxilyzer Serial Number: 80-002201

New Lot Number: 30412100A1

New Expiration Date: 12/1/14

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."
Select "D" for dry gas and enter the following information:
 - Enter Target Value: 0.100
 - Enter Control Lot #: From the tank label
 - Enter Exp Date: From the tank label (mm/dd/yyyy)
 - Edit info Y/N? Y or N
 - Update Std Info? Y ENTER
 - "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."
Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0.100 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove printed record. Attach the record to the completed checklist.

* - Tank info updated/replaced after completing PT.

SIGNATURE Sarah Cione

2 of 2 sc

BSH

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002201
Core Version: 8105.48
06/03/2014 16:58:07

Standard Lot#: 30412100A1
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	16:58:36
0.100 Cal Check	0.100	16:58:57
Air Blank	0.000	16:59:26

Preventative Maintenance NOT
Performed -- Timer NOT Reset