



CERTIFICATION by CUSTODIAN of RECORDS
Tucson Police Department Crime Laboratory

TSY
1-16-2008

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-002261 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 1/16/08 Cal 1 0.100 AC Cal 2 0.100 AC

Between the dates of 12/18/07 and 1/16/08 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on -.
The instrument was returned to service on -.

Sarah L. Cione


Sarah L. Cione

1/16/08

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 16, day of

January, 2008;

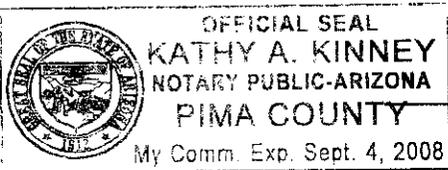

Kathy A Kinney
Notary Public

*Includes documents specified in A.A.C. R13-10-104.A.5

1 of 75c

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

TS4
1.16.2008

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 1/16/08 TIME 10:21AM

INTOXILYZER SERIAL # 80-002261 LOCATION DUI HQ

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 100 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department

2 of 7 sc

TSB

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-002261
01/16/2008 10:23:45

Standard Lot#: 730901
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP,TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|------------------|--------|----------|
| Air Blank | 0.000 | 10:24:35 |
| Diagnostic Test | Pass | 10:25:02 |
| Air Blank | 0.000 | 10:25:29 |
| 0.100 Cal Check | 0.101 | 10:25:48 |
| Air Blank | 0.000 | 10:26:18 |
| Subject Test | 0.000 | 10:26:53 |
| Air Blank | 0.000 | 10:27:20 |
| Five Minute Wait | ABT* | |
| Air Blank | 0.000 | 10:28:08 |

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-002261
01/16/2008

10:29:38

SM

Standard Lot#: 730901
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP,TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 10:30:26 |
| Diagnostic Test | Pass | 10:30:53 |
| Air Blank | RFI* | 10:31:17 |
| Air Blank | IPS** | 10:31:28 |

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

4 of 7 sc

154

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-002261
01/16/2008 10:31:42

Standard Lot#: 730901
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP,TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 10:32:31 |
| Diagnostic Test | Pass | 10:32:58 |
| Air Blank | 0.000 | 10:33:26 |
| 0.100 Cal Check | 0.101 | 10:33:45 |
| Air Blank | 0.000 | 10:34:15 |
| Subject Test | 0.262* | 10:37:23 |
| Air Blank | 0.000 | 10:37:56 |
| Air Blank | 0.000 | 10:38:23 |
| Subject Test | XXX** | 10:38:54 |
| Air Blank | 0.000 | 10:39:28 |

*Deficient Sample
 **Invalid Sample
 Begin new deprivation period

Not a Successfully Completed Test Sequence

5 of 7 sc

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-002261
01/16/2008 10:41:03

TSY

Standard Lot#: 730901
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | 0.000 | 10:41:31 |
| Cal Check | 0.100 | 10:41:49 |
| Air Blank | 0.000 | 10:42:20 |

Sarah Cione
Operator's Signature

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6 of 751

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

By
1.16.2008

QA Specialist Sarah Cione AGENCY Tucson Police Department
DATE 1/16/08 TIME 10:21am
Intoxilyzer Serial # 80-002261 Location DUI HQ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

7 of 7sc

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-002261
01/16/2008 10:21:12

Standard Lot#: 730901
Last Changed By: S. CIONE #50616

BY

QAS: S. CIONE #50616
TUCSON PD

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | 0.000 | 10:21:39 |
| Cal Check | 0.100 | 10:21:58 |
| Air Blank | 0.000 | 10:22:28 |

Sarah Cione

Operator's Signature



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

1 of 2 SC

INTOXILYZER SERVICE RECORD
FOR
CALIBRATION GAS TANK CHANGE

Date: 1/16/08

154

QAS: Sarah Cione

Intoxilyzer Serial Number: 80-002261

New Lot Number: 730901 I ("I" not included in Standard Lot#) SC

New Expiration Date: 11/6/2009

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change." Select "D" for dry gas and enter the following information:
 Enter Target Value: 0.100
 Enter Control Lot #: From the tank label
 Enter Exp Date: From the tank label (mm/dd/yyyy)
 Edit info Y/N? Y or N
 Update Std Info? Y ENTER
 "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing." Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 101 AC.
- (✓) 9. Air blank completed.
- () 10. Remove and sign printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-002261
01/16/2008 10:19:00

154

Standard Lot#: 730901
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | 0.000 | 10:19:28 |
| Cal Check | 0.101 | 10:19:46 |
| Air Blank | 0.000 | 10:20:16 |

Sarah Cione

Operator's Signature



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

INTOXILYZER SERVICE RECORD
FOR
CALIBRATION GAS TANK CHANGE

By
1-22-09

Date: 1/17/08
QAS: NOLAN
Intoxilyzer Serial Number: 80-002261
New Lot Number: 730901I
New Expiration Date: 06 Nov 2009

- 7) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 7) 2. Remove the old gas tank and install the new one.
- 7) 3. Hit the "ESC" key twice and swipe the QAS card.
- 7) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."
Select "D" for dry gas and enter the following information:
Enter Target Value: 0.100
Enter Control Lot #: From the tank label
Enter Exp Date: From the tank label (mm/dd/yyyy)
Edit info Y/N? Y or N
Update Std Info? Y ENTER
"Please wait saving..."
- 7) 5. Hit the "ESC" key twice.
- 7) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."
Select "D" for dry control test. After selection is made press ENTER.
- 7) 7. Air blank completed.
- 7) 8. Calibration check completed. Test results 0. 10' AC.
- 7) 9. Air blank completed.
- 7) 10. Remove and sign printed record. Attach the record to the completed checklist.

SIGNATURE Cecil Nolan

2 of 2 SC

TS4

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-002261
01/17/2008 21:27:39

Standard Lot#: 7309311
Last Changed By: C. NOLAN #36874

OPERATOR: C. NOLAN #36874
TUCSON PD

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | 0.000 | 21:28:07 |
| Cal Check | 0.101 | 21:28:26 |
| Air Blank | 0.000 | 21:28:55 |



Operator's Signature