



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory
1306 W. Miracle Mile, Tucson, AZ 85705

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-002261 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

*TS4
2-23-2012*

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 2/2/12 Cal 1 0.099 AC Cal 2 0.098 AC

Between the dates of 1/5/12 and 2/2/12 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on -.
The instrument was returned to service on -.

Sarah Cione

Sarah L. Cione


2/10/12

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 10th, day of February, 2012.

Elma Escarcega

Notary Public

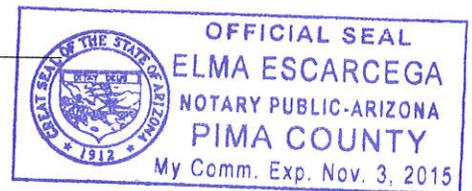

*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

1 of 7 sc

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

B4
2-23-2012

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 2/2/12 TIME 1328

INTOXILYZER SERIAL # 80-002261 LOCATION DUI HQ

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 098 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number : 80-002261
02/02/2012

13:32:27

2 of 7sc

Dh

Standard Lot#: 23511100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|------------------|--------|----------|
| Air Blank | 0.000 | 13:33:10 |
| Diagnostic Test | Pass | 13:33:37 |
| Air Blank | 0.000 | 13:34:05 |
| 0.100 Cal Check | 0.099 | 13:34:24 |
| Air Blank | 0.000 | 13:34:53 |
| Subject Test | 0.000 | 13:35:36 |
| Air Blank | 0.000 | 13:36:03 |
| Five Minute Wait | ABT* | |
| Air Blank | 0.000 | 13:36:55 |

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number : 80-002261
02/02/2012 13:39:31

3 of 7sc

Dh

Standard Lot#: 23511100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | RFI* | 13:39:57 |
| Air Blank | IPS** | 13:40:05 |

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number : 80-002261
02/02/2012

13:41:36

4 of 7 SC

TJ4

Standard Lot#: 23511100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 13:42:32 |
| Diagnostic Test | Pass | 13:42:59 |
| Air Blank | 0.000 | 13:43:27 |
| 0.100 Cal Check | 0.098 | 13:43:46 |
| Air Blank | 0.000 | 13:44:15 |
| Subject Test | 0.032* | 13:47:23 |
| Air Blank | 0.000 | 13:47:54 |
| Air Blank | 0.000 | 13:48:22 |
| Subject Test | XXX** | 13:48:47 |
| Air Blank | 0.000 | 13:49:21 |

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002261
02/02/2012 13:54:34


5 of 7_{sc}

Standard Lot#: 23511100A2
Last Changed By: S. CIONE #50616

TJ4

QAS: S. CIONE #50616
TUCSON PD

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | 0.000 | 13:55:01 |
| Cal Check | 0.098 | 13:55:20 |
| Air Blank | 0.000 | 13:55:50 |

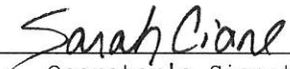

Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6 of 7 sc

ARIZONA DEPARTMENT OF PUBLIC SAFETY

T54

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

2.23.2012

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Sarah Cione AGENCY Tucson Police Department
DATE 2/2/12 TIME 1328
Intoxilyzer Serial # 80-002261 Location DUI HQ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002261
02/02/2012 13:28:40

7 of 75c

Standard Lot#: 23511100A2
Last Changed By: S. CIONE #50616

154

QAS: S. CIONE #50616
TUCSON PD

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | 0.000 | 13:29:07 |
| Cal Check | 0.099 | 13:29:26 |
| Air Blank | 0.000 | 13:29:55 |



Operator's Signature