



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory  
1306 W. Miracle Mile, Tucson, AZ 85705

TS4  
11-25-2015


I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-002261 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 11/13/15 Cal 1 0.103 AC Cal 2 0.102 AC

Between the dates of 10/15/15 and 11/13/15 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on                     .  
The instrument was returned to service on                     .

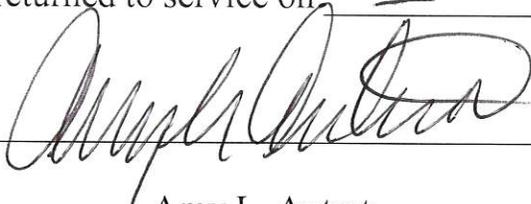  
Amy L. Autret


11/16/15

ADPS QAS Permit # 33718

SUBSCRIBED AND SWORN to before me this 16, day of November, 20 15.



Notary Public

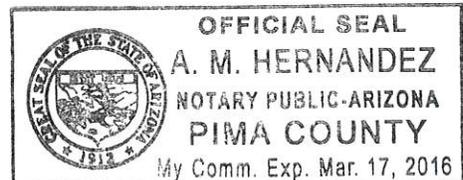

\*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

1 of 7  
*[Signature]*  
TJG  
11-25-2015

QA SPECIALIST Amy Autret 53546 AGENCY Tucson Police Department

DATE 11/13/15 TIME 0811

INTOXILYZER SERIAL # 80-002261 LOCATION OPE

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

(✓) 1. Clock time check.

(✓) 2. Date check.

OPERATIONAL TESTS

(✓) 1. Alcohol-free subject test result 0. 000 AC.

(✓) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed

(✓) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.

(✓) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 102 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE *[Signature]*

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-002261  
Core Version: 8105.48  
11/13/2015 08:13:00

*2087  
all  
TS4*

---

Standard Lot#: 25414100A4  
Last Changed By: A. AUTRET #53546

---

QAS: A. AUTRET #53546  
TUCSON PD

---

Operator: A. AUTRET #53546  
TUCSON PD

---

Subject: SQAP, SQAP  
DOB: 11/11/1987  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	08:15:01
Diagnostic Test	Pass	08:15:36
Air Blank	0.000	08:16:04
0.100 Cal Check	0.103	08:16:24
Air Blank	0.000	08:16:54
Subject Test	0.000	08:17:20
Air Blank	0.000	08:17:47
Wait	ABT*	
Air Blank	0.000	08:18:22

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-002261  
Core Version: 8105.48  
11/13/2015

08:18:28

3 of 7  
*[Signature]*  
TSh

Standard Lot#: 25414100A4  
Last Changed By: A. AUTRET #53546

QAS: A. AUTRET #53546  
TUCSON PD

Operator: A. AUTRET #53546  
TUCSON PD

Subject: SQAP, SQAP  
DOB: 11/11/1987  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:19:20
Diagnostic Test	Pass	08:19:55
Air Blank	0.000	08:20:22
0.100 Cal Check	RFI*	08:20:35
Air Blank	IPS**	08:20:43

\*RFI Detect  
\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-002261  
Core Version: 8105.48  
11/13/2015 08:20:48

4 of 7  
JLB  
TSY

---

Standard Lot#: 25414100A4  
Last Changed By: A. AUTRET #53546

---

QAS: A. AUTRET #53546  
TUCSON PD

---

Operator: A. AUTRET #53546  
TUCSON PD

---

Subject: SQAP, SQAP  
DOB: 11/11/1987  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	08:21:40
Diagnostic Test	Pass	08:22:15
Air Blank	0.000	08:22:42
0.100 Cal Check	0.103	08:23:03
Air Blank	0.000	08:23:33
Subject Test	0.148*	08:26:40
Air Blank	0.000	08:27:12
Air Blank	0.000	08:27:39
Subject Test	XXX**	08:28:03
Air Blank	0.000	08:28:36

\*Deficient Sample  
\*\*Invalid Sample  
Begin new deprivation period

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-002261  
Core Version: 8105.48  
11/13/2015 08:28:52

5 of 7  
[Signature]  
TS4

-----  
Standard Lot#: 25414100A4  
Last Changed By: A. AUTRET #53546  
-----

QAS: A. AUTRET #53546  
TUCSON PD  
-----

Operator: A. AUTRET #53546  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	08:29:23
0.100 Cal Check	0.102	08:29:44
Air Blank	0.000	08:30:13

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

6 of 7  
JG  
11.25.2015

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Amy Autret 53546 AGENCY Tucson Police Department  
DATE 11/13/15 TIME 0811  
Intoxilyzer Serial # 80-002261 Location ODE

- ( ✓ ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- ( ✓ ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ( ✓ ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ( ✓ ) 4. Air blank completed.
- ( ✓ ) 5. Calibration check completed. Test results 0. 103 AC.
- ( ✓ ) 6. Air blank completed.
- ( ✓ ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE



DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-002261  
Core Version: 8105.48  
11/13/2015 08:11:30

7 of 7  
dd  
TS4

-----  
Standard Lot#: 25414100A4  
Last Changed By: A. AUTRET #53546  
-----

QAS: A. AUTRET #53546  
TUCSON PD  
-----

Operator: A. AUTRET #53546  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	08:11:59
0.100 Cal Check	0.103	08:12:20
Air Blank	0.000	08:12:49

Preventative Maintenance  
Performed -- Timer Reset