



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory  
370 S. Stone Avenue Tucson, AZ 85701-1917

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-002262 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

TSY  
5.31.2011

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 5/26/11 Cal 1 0.100 AC Cal 2 0.099 AC

Between the dates of 4/28/11 and 5/26/11 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on -  
The instrument was returned to service on -

Sarah Cione


Sarah L. Cione

5/30/11

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 30, day of

MAY, 20 11

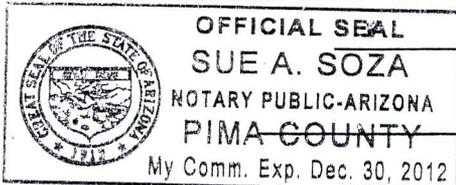

Sue A. Soza

Notary Public

\*Includes documents specified in A.A.C. R13-10-104.A.5

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

BY  
5.31.2011

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 5/26/11 TIME 1030

INTOXILYZER SERIAL # 80-002262 LOCATION DUI HQ

( ✓ ) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- ( ✓ ) 1. Clock time check.
- ( ✓ ) 2. Date check.

OPERATIONAL TESTS

- ( ✓ ) 1. Alcohol-free subject test result 0. 000 AC.
- ( ✓ ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- ( ✓ ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- ( ✓ ) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 099 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS

---



---



---



---

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

2 of 9sc

INTOXILYZER 8000  
Location: ARIZONA  
Serial Number : 80-002262  
05/26/2011 10:34:11

TSN

---

Standard Lot#: 642845  
Last Changed By: S. CIONE #50616

---

QAS: S. CIONE #50616  
TUCSON PD

---

Operator: S. CIONE #50616  
TUCSON PD

---

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	10:34:53
Diagnostic Test	Pass	10:35:20
Air Blank	0.000	10:35:48
0.100 Cal Check	0.100	10:36:06
Air Blank	0.000	10:36:36
Subject Test	0.000	10:37:09
Air Blank	0.000	10:37:36
Five Minute Wait	ABT*	
Air Blank	0.000	10:38:11

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

3 of 9 sc

TS

INTOXILYZER 8000  
Location: ARIZONA  
Serial Number : 80-002262  
05/26/2011 10:38:36

---

Standard Lot#: 642845  
Last Changed By: S. CIONE #50616

---

QAS: S. CIONE #50616  
TUCSON PD

---

Operator: S. CIONE #50616  
TUCSON PD

---

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	10:39:06
Air Blank	IPS**	10:39:13

\*RFI Detect  
\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

4 of 9 sc

184

INTOXILYZER 8000  
Location: ARIZONA  
Serial Number : 80-002262  
05/26/2011 10:40:22

Standard Lot#: 642845  
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616  
TUCSON PD

Operator: S. CIONE #50616  
TUCSON PD

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:41:18
Diagnostic Test	Pass	10:41:46
Air Blank	0.000	10:42:13
0.100 Cal Check	0.099	10:42:32
Air Blank	0.000	10:43:01
Subject Test	0.017*	10:46:10
Air Blank	0.000	10:46:42
Air Blank	0.000	10:47:09
Subject Test	XXX**	10:47:23
Air Blank	0.000	10:47:57

\*Deficient Sample  
\*\*Invalid Sample  
Begin new deprivation period

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: ARIZONA  
Serial Number: 80-002262  
05/26/2011 10:48:10

5 of 9 sc

-----  
Standard Lot#: 642845  
Last Changed By: S. CIONE #50616  
-----

BM

-----  
QAS: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	10:48:38
Cal Check	0.099	10:48:56
Air Blank	0.000	10:49:26

Sarah Cione  
Operator's Signature

6 of 9 sc

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

TBY  
5.31.2011

QA Specialist Sarah Cione AGENCY Tucson Police Department  
DATE 5/26/11 TIME 1030  
Intoxilyzer Serial # 80-002262 Location DUI HQ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000  
Location: ARIZONA  
Serial Number: 80-002262  
05/26/2011 10:30:26

7 of 9 sc

-----  
Standard Lot#: 642845  
Last Changed By: S. CIONE #50616  
-----

TBY

-----  
QAS: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	10:30:53
Cal Check	0.100	10:31:12
Air Blank	0.000	10:31:41

*Sarah Cione*

\_\_\_\_\_  
Operator's Signature



TUCSON POLICE DEPARTMENT  
CRIME LABORATORY

8 of 9 sc

INTOXILYZER SERVICE RECORD  
FOR  
CALIBRATION GAS TANK CHANGE

By  
5.31.2011

Date: 5/26/11

QAS: Sarah Cione

Intoxilyzer Serial Number: 80-002262

New Lot Number: 20910100A1

New Expiration Date: 8/1/12

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."  
Select "D" for dry gas and enter the following information:  
  - Enter Target Value: 0.100
  - Enter Control Lot #: From the tank label
  - Enter Exp Date: From the tank label (mm/dd/yyyy)
  - Edit info Y/N? Y or N
  - Update Std Info? Y ENTER
  - "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "Control Testing."  
Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 099 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

9 of 95c

TSY

INTOXILYZER 8000  
Location: ARIZONA  
Serial Number: 80-002262  
05/26/2011 11:02:49

-----  
Standard Lot#: 20910100A1  
Last Changed By: S. CIONE #50616  
-----

QAS: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	11:03:16
Cal Check	0.099	11:03:35
Air Blank	0.000	11:04:04

Sarah Cione  
Operator's Signature