



CERTIFICATION by CUSTODIAN of RECORDS  
Tucson Police Department Crime Laboratory

ll  
10-26-07

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-002264 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 9.19.2007 Cal 1 0.101 AC Cal 2 0.100 AC

Between the dates of 8.17.2007 and 9.19.2007 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on                     .  
The instrument was returned to service on                     .

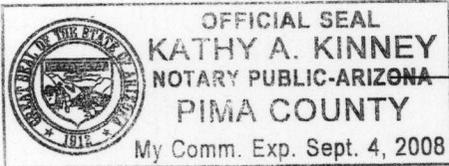
Terry S. Gallegos

Terry S. Gallegos

10.25.2007

ADPS QAS Permit # 33169

SUBSCRIBED AND SWORN to before me this 25, day of October, 2007.



Kathy A Kinney

Notary Public

\*Includes documents specified in A.A.C. R13-10-104.A.5

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

EXHIBIT G-4

CM  
10-26-07

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD QUALITY ASSURANCE PROCEDURE

1077  
TS4

QA SPECIALIST Terry Gallegos 28947 AGENCY Tucson Police Department  
DATE 9-19-2007 TIME 1755  
INTOXILYZER SERIAL # 80-002264 LOCATION DUI HQ

(  ) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

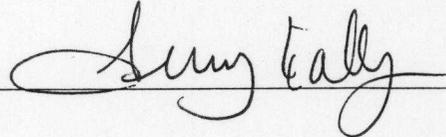
(  ) 1. Clock time check.  
(  ) 2. Date check.

OPERATIONAL TESTS

(  ) 1. Alcohol-free subject test result 0. 000 AC.  
(  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed ✓✓✓  
(  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
(  ) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 100 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS Ex G-2 performed at 33 days. TSG  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE 

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

CM  
2077  
TS4

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002264  
09/19/2007 18:04:15

-----  
Standard Lot#: 7198011  
Last Changed By: S. SULLIVAN #34971  
-----

QAS: T. GALLEGOS #28847  
TUCSON PD CRIME LAB  
-----

Operator: T. GALLEGOS #28847  
TUCSON PD CRIME LAB  
-----

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111  
-----

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:05:02
Diagnostic Test	Pass	18:05:31
Air Blank	0.000	18:05:58
0.100 Cal Check	0.101	18:06:17
Air Blank	0.000	18:06:46
Subject Test	0.000	18:07:42
Air Blank	0.000	18:08:09
Five Minute Wait	ABT*	
Air Blank	0.000	18:08:44

\*Sequence Aborted

Not a Successfully  
Completed  
Test Sequence

CM  
3077  
134

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002264  
09/19/2007 18:10:00

-----  
Standard Lot#: 7198011  
Last Changed By: S. SULLIVAN #34971  
-----

QAS: T. GALLEGOS #28847  
TUCSON PD CRIME LAB  
-----

Operator: T. GALLEGOS #28847  
TUCSON PD CRIME LAB  
-----

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111  
-----

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:10:46
Diagnostic Test	Pass	18:11:15
Air Blank	0.000	18:11:43
0.100 Cal Check	0.100	18:12:01
Air Blank	RFI*	18:12:16
Air Blank	IPS**	18:12:33

\*RFI Detect  
\*\*Improper Sample

Not a Successfully  
Completed  
Test Sequence

CU  
4057  
TS4

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002264  
09/19/2007 18:13:13

-----  
Standard Lot#: 7198011  
Last Changed By: S. SULLIVAN #34971  
-----

DAS: T. GALLEGOS #28847  
TUCSON PD CRIME LAB  
-----

Operator: T. GALLEGOS #28847  
TUCSON PD CRIME LAB  
-----

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111  
-----

15 Minute Deprivation Period? Yes  
-----

Test	g/210L	Time
Air Blank	0.000	18:14:01
Diagnostic Test	Pass	18:14:30
Air Blank	0.000	18:14:58
0.100 Cal Check	0.101	18:15:16
Air Blank	0.000	18:15:46
Subject Test	0.300*	18:18:54
Air Blank	0.000	18:19:27
Air Blank	0.000	18:19:54
Subject Test	XXX**	18:20:12
Air Blank	0.000	18:20:45

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

Not a Successfully  
Completed  
Test Sequence

CM

Soft  
TS4

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002264  
09/19/2007 18:21:40

-----  
Standard Lot#: 7198011  
Last Changed By: S. SULLIVAN #34971  
-----

QAS: T. GALLEGOS #28847  
TUCSON PD CRIME LAB  
-----

Test	g/210L	Time
Air Blank	0.000	18:22:07
Cal Check	0.100	18:22:26
Air Blank	0.000	18:22:56

  
-----  
Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

CU  
10-26-07

ARIZONA DEPARTMENT OF PUBLIC SAFETY

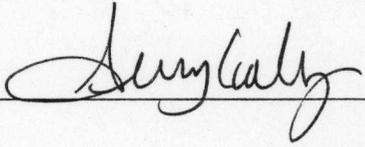
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

6077  
TSh

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Terry Gallegos 28842 AGENCY Tucson Police Department  
DATE 9-19-2007 TIME 1755  
Intoxilyzer Serial # 80-002264 Location DUI HQ

- ( ✓ ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C + 0.2°C
- ( ✓ ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ( ✓ ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ( ✓ ) 4. Air blank completed.
- ( ✓ ) 5. Calibration check completed. Test results 0. 101 AC.
- ( ✓ ) 6. Air blank completed.
- ( ✓ ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

DPS Form Exh G-2 (Rev. 05-01)

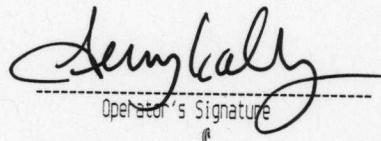
CM  
7 of 7  
BY

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002264  
09/19/2007 17:55:56

-----  
Standard Lot#: 7198011  
Last Changed By: S. SULLIVAN #34971  
-----

QAS: T. GALLEGOS #28847  
TUCSON PD CRIME LAB  
-----

Test	g/210L	Time
Air Blank	0.000	17:56:24
Cal Check	0.101	17:56:42
Air Blank	0.000	17:57:12

  
-----  
Operator's Signature  
↓



TUCSON POLICE DEPARTMENT  
CRIME LABORATORY

CM  
10-26-07

INTOXILYZER SERVICE RECORD  
FOR  
CALIBRATION GAS TANK CHANGE

10 of 2  
TS4

Date: 9/5/07  
 QAS: SULLIVAN # 33272  
 Intoxilyzer Serial Number: 80-002264  
 New Lot Number: 719-8011  
 New Expiration Date: 23 July, 2009

- (X) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 2. Remove the old gas tank and install the new one.
- (X) 3. Hit the "ESC" key twice and swipe the QAS card.
- (X) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."  
 Select "D" for dry gas and enter the following information:  
     Enter Target Value: 0.100  
     Enter Control Lot #: From the tank label  
     Enter Exp Date: From the tank label (mm/dd/yyyy)  
     Edit info Y/N? Y or N  
     Update Std Info? Y ENTER  
     "Please wait saving..."
- (X) 5. Hit the "ESC" key twice.
- (X) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."  
 Select "D" for dry control test. After selection is made press ENTER.
- (X) 7. Air blank completed.
- (X) 8. Calibration check completed. Test results 0. 099 AC.
- (X) 9. Air blank completed.
- (X) 10. Remove and sign printed record. Attach the record to the completed checklist.

SIGNATURE Sullivan QAS # 33272

CM  
TS4  
2012

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002264  
09/05/2007 18:24:19

-----  
Standard Lot#: 7198011  
Last Changed By: S. SULLIVAN #34971  
-----

QAS: S. SULLIVAN #34971  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	18:24:46
Cal Check	0.099	18:25:05
Air Blank	0.000	18:25:34

SULLIVAN #33212  
Operator's Signature