



CERTIFICATION by CUSTODIAN of RECORDS
Tucson Police Department Crime Laboratory

TSY
1.16.2008

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-002264 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 1/16/08 Cal 1 0.103 AC Cal 2 0.102 AC

Between the dates of 12/18/07 and 1/16/08 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on -
The instrument was returned to service on -


Sarah L. Cione

Sarah L. Cione

1/16/08

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 16, day of January, 2008.

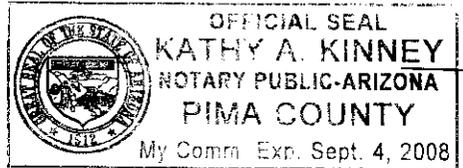

Kathy A Kinney
Notary Public

*Includes documents specified in A.A.C. R13-10-104.A.5

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

TS4

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 1/16/08 TIME 9:33 am

INTOXILYZER SERIAL # 80-002264 LOCATION DUI HQ

- (✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 102 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Sarah Cione

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-002264
01/16/2008 09:36:19

TS4

Standard Lot#: 7254021
Last Changed By: J. HAWKINS #42401

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP,TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:37:07
Diagnostic Test	Pass	09:37:35
Air Blank	0.000	09:38:02
0.100 Cal Check	0.102	09:38:21
Air Blank	0.000	09:38:51
Subject Test	0.000	09:39:12
Air Blank	0.000	09:39:39
Five Minute Wait	ABT*	
Air Blank	0.000	09:40:32

*Sequence Aborted

Not a Successfully Completed Test Sequence

3 of 7 SC

154

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-002264
01/16/2008 09:42:43

Standard Lot#: 7254021
Last Changed By: J. HAWKINS #42401

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:43:46
Diagnostic Test	Pass	09:44:14
Air Blank	RFI*	09:44:22
Air Blank	IPS**	09:44:36

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

4 of 750

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-002264
01/16/2008 09:47:02

104

Standard Lot#: 7254021
Last Changed By: J. HAWKINS #42401

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:47:55
Diagnostic Test	Pass	09:48:23
Air Blank	0.000	09:48:50
0.100 Cal Check	0.103	09:49:09
Air Blank	0.000	09:49:38
Subject Test	0.094*	09:52:47
Air Blank	0.000	09:53:20
Air Blank	0.000	09:53:47
Subject Test	XXX**	09:54:07
Air Blank	0.000	09:54:41

*Deficient Sample
 **Invalid Sample
 Begin new deprivation period

Not a Successfully Completed Test Sequence

5 of 75c

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-002264
01/16/2008 09:57:37

TS4

Standard Lot#: 7254021
Last Changed By: J. HAWKINS #42401

QAS: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	09:58:05
Cal Check	0.102	09:58:24
Air Blank	0.000	09:58:53

Sarah Cione

Operator's Signature

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6 of 7sc

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

TSC
1-16-2008

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Sarah Cione AGENCY Tucson Police Department
DATE 1/16/08 TIME 9:33am
Intoxilyzer Serial # 80-002264 Location DUI HQ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 103 AC.
- (✓) 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-002264
01/16/2008 09:33:05

7 of 75c
TS4

Standard Lot#: 7254021
Last Changed By: J. HAWKINS #42401

QAS: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	09:33:32
Cal Check	0.103	09:33:51
Air Blank	0.000	09:34:20

Sarah Cione

Operator's Signature



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

1 of 250

TS4

INTOXILYZER SERVICE RECORD
FOR
CALIBRATION GAS TANK CHANGE

Date: 12/29/07
QAS: HAWKINS 42401
Intoxilyzer Serial Number: 80-002264
New Lot Number: 7254021
New Expiration Date: 12 Sep 2009

- (X) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 2. Remove the old gas tank and install the new one.
- (X) 3. Hit the "ESC" key twice and swipe the QAS card.
- (X) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."
Select "D" for dry gas and enter the following information:
Enter Target Value: 0.100
Enter Control Lot #: From the tank label
Enter Exp Date: From the tank label (mm/dd/yyyy)
Edit info Y/N? Y or N
Update Std Info? Y ENTER
"Please wait saving..."
- (X) 5. Hit the "ESC" key twice.
- (X) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."
Select "D" for dry control test. After selection is made press ENTER.
- (X) 7. Air blank completed.
- (X) 8. Calibration check completed. Test results 0. 103 AC.
- (X) 9. Air blank completed.
- (X) 10. Remove and sign printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

BS4

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-602264
12/29/2007 19:48:58

Standard Lot#: 7254021
Last Changed By: J. HAWKINS #42401

DAS: J. HAWKINS #42401
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	19:49:25
Cal Check	0.103	19:49:44
Air Blank	0.000	19:50:14



Operator's Signature