



**CERTIFICATION by CUSTODIAN of RECORDS**  
*Tucson Police Department Crime Laboratory*  
 270 S. Stone Avenue Tucson, AZ 85701-1917

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-002264 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

*TSY*  
*2.4.2009*

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 1/15/09 Cal 1 0.101 AC Cal 2 0.101 AC

Between the dates of 12/18/08 and 1/15/09 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on —.  
 The instrument was returned to service on —.

Sarah Cione

Sarah L. Cione


1/21/09

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 21<sup>st</sup>, day of January, 2009.

Art Hernandez

Notary Public

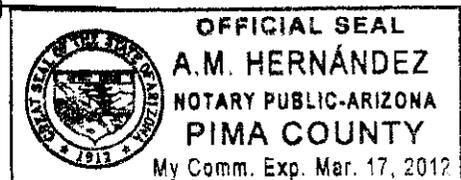

\*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

1 of 7 SC

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD QUALITY ASSURANCE PROCEDURE

TS4  
2.4.2009

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department  
DATE 1/15/09 TIME 9:00  
INTOXILYZER SERIAL # 80-002264 LOCATION DUI HQ

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 0.101 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

2 of 750

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-002264  
01/15/2009 09:01:57

AM

Standard Lot#: 811201I  
Last Changed By: J. HAWKINS #42401

QAS: S. CIONE #50616  
TUCSON PD

Operator: S. CIONE #50616  
TUCSON PD

Subject: SQAP, TEST  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:02:41
Diagnostic Test	Pass	09:03:08
Air Blank	0.000	09:03:36
0.100 Cal Check	0.102	09:03:54
Air Blank	0.000	09:04:24
Subject Test	0.000	09:04:44
Air Blank	0.000	09:05:12
Five Minute Wait	ABT*	
Air Blank	0.000	09:06:30

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

3 of 750

104

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-002264  
01/15/2009

09:11:48

---

Standard Lot#: 811201I  
Last Changed By: J. HAWKINS #42401

---

QAS: S. CIONE #50616  
TUCSON PD

---

Operator: S. CIONE #50616  
TUCSON PD

---

Subject: SQAP, TEST  
DOB: 11/11/1111  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	09:12:30
Air Blank	IPS**	09:12:39

\*RFI Detect  
\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

4 of 7 SC

BY

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-002264  
01/15/2009 09:13:03

Standard Lot#: 811201I  
Last Changed By: J. HAWKINS #42401

QAS: S. CIONE #50616  
TUCSON PD

Operator: S. CIONE #50616  
TUCSON PD

Subject: SQAP, TEST  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:13:47
Diagnostic Test	Pass	09:14:14
Air Blank	0.000	09:14:41
0.100 Cal Check	0.102	09:15:00
Air Blank	0.000	09:15:30
Subject Test	0.065*	09:18:38
Air Blank	0.000	09:19:09
Air Blank	0.000	09:19:37
Subject Test	XXX**	09:20:01
Air Blank	0.000	09:20:34

\*Deficient Sample  
\*\*Invalid Sample  
Begin new deprivation period

**Not a Successfully Completed Test Sequence**

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

5 of 7 SC

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002264  
01/15/2009 09:21:15

-----  
Standard Lot#: 811201I  
Last Changed By: J. HAWKINS #42401  
-----

BY

-----  
QAS: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	09:21:43
Cal Check	0.101	09:22:01
Air Blank	0.000	09:22:31

Sarah Cione  
Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

6 of 7 SC

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

TS4  
2.4.2009

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Sarah Cione AGENCY Tucson Police Department  
DATE 1/15/09 TIME 9:00  
Intoxilyzer Serial # 80-002264 Location DUI HQ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 101 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002264  
01/15/2009 09:00:15

7 of 7 sc

-----  
Standard Lot#: 811201I  
Last Changed By: J. HAWKINS #42401  
-----

154

-----  
QAS: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	09:00:43
Cal Check	0.101	09:01:01
Air Blank	0.000	09:01:31

Sarah Cione  
Operator's Signature



TUCSON POLICE DEPARTMENT  
CRIME LABORATORY

1 of 2 SC

INTOXILYZER SERVICE RECORD  
FOR  
CALIBRATION GAS TANK CHANGE

By  
2.4.2009

Date: 12/22/08  
 QAS: HAWKINS 42401  
 Intoxilyzer Serial Number: 80-002264  
 New Lot Number: 811201T  
 New Expiration Date: 31 Mar 2010

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."  
 Select "D" for dry gas and enter the following information:  
     Enter Target Value: 0.100  
     Enter Control Lot #: From the tank label  
     Enter Exp Date: From the tank label (mm/dd/yyyy)  
     Edit info Y/N? Y or N  
     Update Std Info? Y ENTER  
     "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."  
 Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 1102 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove and sign printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] 42401

2 of 2 sc

BSH

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002264  
12/22/2008 19:55:14

-----  
Standard Lot#: 8112011  
Last Changed By: J. HAWKINS #42401

-----  
OAS: J. HAWKINS #42401  
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	19:55:41
Cal Check	0.102	19:56:00
Air Blank	0.000	19:56:30

  
-----  
Operator's Signature