



CERTIFICATION by CUSTODIAN of RECORDS  
Tucson Police Department Crime Laboratory

T34  
3-17-08

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-002265 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 3/7/08 Cal 1 0.101 AC Cal 2 0.100 AC

Between the dates of 2/8/08 and 3/7/08 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on —.  
The instrument was returned to service on —.


Sarah Cione

Sarah L. Cione

3/14/08

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 14, day of March, 2008.

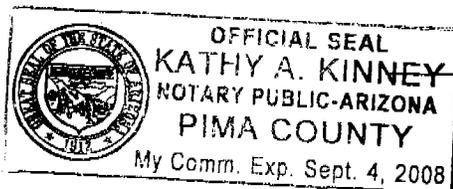

Kathy A Kinney  
Notary Public

\*Includes documents specified in A.A.C. R13-10-104.A.5

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

1 of 75c

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

B4

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 3/7/08 TIME 13:18

INTOXILYZER SERIAL # 80-002265 LOCATION Team 5

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 100 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS

---



---



---



---

SIGNATURE Sarah Cione

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-002265  
03/07/2008

13:19:56

2 of 7sc

184

---

Standard Lot#: 719801I  
Last Changed By: S. CIONE #50616

---

QAS: S. CIONE #50616  
TUCSON PD

---

Operator: S. CIONE #50616  
TUCSON PD

---

Subject: SQAP, TEST  
DOB: 11/11/1111  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	13:20:43
Diagnostic Test	Pass	13:21:11
Air Blank	0.000	13:21:38
0.100 Cal Check	0.101	13:21:57
Air Blank	0.000	13:22:26
Subject Test	0.000	13:22:42
Air Blank	0.000	13:23:09
Five Minute Wait	ABT*	
Air Blank	0.000	13:23:51

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-002265  
03/07/2008 13:24:06

3 of 7sc

BY

---

Standard Lot#: 719801I  
Last Changed By: S. CIONE #50616

---

QAS: S. CIONE #50616  
TUCSON PD

---

Operator: S. CIONE #50616  
TUCSON PD

---

Subject: SQAP, TEST  
DOB: 11/11/1111  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	13:24:54
Diagnostic Test	Pass	13:25:21
Air Blank	RFI*	13:25:29
Air Blank	IPS**	13:25:37

\*RFI Detect

\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

4 of 75c

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-002265  
03/07/2008 13:25:47

BY

Standard Lot#: 719801I  
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616  
TUCSON PD

Operator: S. CIONE #50616  
TUCSON PD

Subject: SQAP,TEST  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:26:35
Diagnostic Test	Pass	13:27:02
Air Blank	0.000	13:27:30
0.100 Cal Check	0.102	13:27:49
Air Blank	0.000	13:28:18
Subject Test	0.089*	13:31:26
Air Blank	0.000	13:31:59
Air Blank	0.000	13:32:26
Subject Test	XXX**	13:32:50
Air Blank	0.000	13:33:23

\*Deficient Sample  
\*\*Invalid Sample  
Begin new deprivation period

**Not a Successfully Completed Test Sequence**

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002265  
03/07/2008 13:33:41

5 of 7 sc

-----  
Standard Lot#: 719801I  
Last Changed By: S. CIONE #50616  
-----

BS4

-----  
QAS: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	13:34:08
Cal Check	0.100	13:34:27
Air Blank	0.000	13:34:57

*Sarah Cione*

Operator's Signature

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Sarah Cione AGENCY Tucson Police Department  
DATE 3/7/08 TIME 13:18  
Intoxilyzer Serial # 80-002265 Location Team 5

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 101 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE

Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002265  
03/07/2008 13:18:27

7 of 75C  
B4

-----  
Standard Lot#: 719801I  
Last Changed By: S. CIONE #50616  
-----

QAS: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	13:18:54
Cal Check	0.101	13:19:13
Air Blank	0.000	13:19:42

  
-----  
Operator's Signature



TUCSON POLICE DEPARTMENT  
CRIME LABORATORY

1 of 2 sc

INTOXILYZER SERVICE RECORD

BY

Date: 3/4/08

QAS: Sarah Cione

Intoxilyzer Serial Number: 80-002265

Reason For Service: I went to Team 5 to change gas tank, but it just needed to be screwed in. Updated standard lot #, + checked calibration

Date Out of Service: N/A

Calibration Check completed prior to removal from service

Date: N/A

SQAP completed prior to removal from service

Date: N/A

Corrective Action:

Sent to CMI, Inc.

N/A

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002265  
03/04/2008 09:11:01

2 of 25c  
B4

-----  
Standard Lot#: 719801I  
Last Changed By: S. CIONE #50616  
-----

QAS: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	09:11:28
Cal Check	0.101	09:11:47
Air Blank	0.000	09:12:16

  
-----  
Operator's Signature