



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory
270 S. Stone Avenue Tucson, AZ 85701-1917

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-002265 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

TSY
6.8.2009

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 5/28/09 Cal 1 0.101 AC Cal 2 0.101 AC

Between the dates of 5/1/09 and 5/28/09 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on -
The instrument was returned to service on -

Sarah Cione

Sarah L. Cione


5/28/09

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 28th day of May, 2009.

A.M. Hernandez

Notary Public

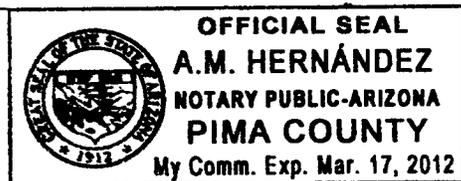

*Includes documents specified in A.A.C. R13-10-104.A.5

1 of 7 sc

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

TSH
6.8.2009

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 5/28/09 TIME 10:59

INTOXILYZER SERIAL # 80-002265 LOCATION Team 5

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 101 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

2 of 7 sc

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-002265
05/28/2009 11:01:11

104

Standard Lot#: 801002I
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:01:58
Diagnostic Test	Pass	11:02:25
Air Blank	0.000	11:02:53
0.100 Cal Check	0.101	11:03:12
Air Blank	0.000	11:03:41
Subject Test	0.000	11:04:04
Air Blank	0.000	11:04:32
Five Minute Wait	ABT*	
Air Blank	0.000	11:05:08

*Sequence Aborted

Not a Successfully Completed Test Sequence

3 of 7 sc

SM

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-002265
05/28/2009 11:09:05

Standard Lot#: 801002I
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	11:09:43
Air Blank	IPS**	11:09:51

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

4 of 7 sc

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-002265
05/28/2009

11:10:05

Standard Lot#: 801002I
Last Changed By: S. CIONE #50616

BY

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP,TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:10:50
Diagnostic Test	Pass	11:11:17
Air Blank	0.000	11:11:45
0.100 Cal Check	0.101	11:12:04
Air Blank	0.000	11:12:33
Subject Test	0.100*	11:15:41
Air Blank	0.000	11:16:13
Air Blank	0.000	11:16:41
Subject Test	XXX**	11:17:04
Air Blank	0.000	11:17:37

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-002265
05/28/2009 11:17:54

5 of 7 sc

Standard Lot#: 801002I
Last Changed By: S. CIONE #50616

py

QAS: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	11:18:21
Cal Check	0.101	11:18:40
Air Blank	0.000	11:19:10

Sarah Cione

Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6 of 7 sc

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

By
6.8.2009

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Sarah Cione AGENCY Tucson Police Department
DATE 5/28/09 TIME 10:59
Intoxilyzer Serial # 80-002265 Location Team 5

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 101 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000

Location: TUCSON POLICE DEPT

Serial Number: 80-002265

05/28/2009

10:59:45

7 of 7
5c

TS4

Standard Lot#: 801002I

Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616

TUCSON PD

Test	g/210L	Time
Air Blank	0.000	11:00:12
Cal Check	0.101	11:00:31
Air Blank	0.000	11:01:00

Sarah Cione

Operator's Signature