



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory  
270 S. Stone Avenue Tucson, AZ 85701-1917

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-002265 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

*TBJ  
10-25-2010*

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 10/21/10 Cal 1 0.099 AC Cal 2 0.099 AC

Between the dates of 9/23/10 and 10/21/10 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on N/A.  
The instrument was returned to service on N/A.

*Cheryl K. Straub*

Cheryl K. Straub


10/21/10

ADPS QAS Permit # 33408

SUBSCRIBED AND SWORN to before me this 21, day of October, 2010.

*A.M. Hernandez*

Notary Public

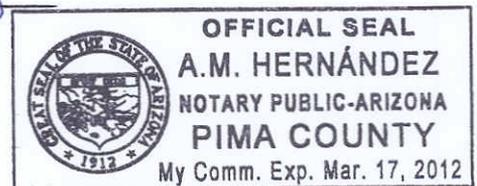

\*Includes documents specified in A.A.C. R13-10-104.A.5

187  
CS

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD QUALITY ASSURANCE PROCEDURE

T54  
10-25-2010

QA SPECIALIST Cheryl Straub AGENCY Tucson Police Department  
DATE 10/21/10 TIME 08:50  
INTOXILYZER SERIAL # 80-002265 LOCATION TEAM 5

( ✓ ) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- ( ✓ ) 1. Clock time check.
- ( ✓ ) 2. Date check.

OPERATIONAL TESTS

- ( ✓ ) 1. Alcohol-free subject test result 0. 000 AC.
- ( ✓ ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- ( ✓ ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- ( ✓ ) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 099 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE Cheryl Straub

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

277  
CS

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-002265  
10/21/2010

08:52:23

154

Standard Lot#: 642845  
Last Changed By: C. STRAUB #50772

QAS: C. STRAUB #50772  
TUCSON PD

Operator: C. STRAUB #50772  
TUCSON PD

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:53:13
Diagnostic Test	Pass	08:53:40
Air Blank	0.000	08:54:07
0.100 Cal Check	0.100	08:54:26
Air Blank	0.000	08:54:56
Subject Test	0.000	08:55:24
Air Blank	ABT*	08:55:31
Air Blank	0.000	08:56:00

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

377  
CS

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-002265  
10/21/2010

08:56:07

ph

Standard Lot#: 642845  
Last Changed By: C. STRAUB #50772

QAS: C. STRAUB #50772  
TUCSON PD

Operator: C. STRAUB #50772  
TUCSON PD

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	08:56:42
Air Blank	IPS**	08:56:51

\*RFI Detect  
\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

477  
CS

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-002265  
10/21/2010

08:57:01

TS4

Standard Lot#: 642845  
Last Changed By: C. STRAUB #50772

QAS: C. STRAUB #50772  
TUCSON PD

Operator: C. STRAUB #50772  
TUCSON PD

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:57:48
Diagnostic Test	Pass	08:58:16
Air Blank	0.000	08:58:43
0.100 Cal Check	0.100	08:59:02
Air Blank	0.000	08:59:32
Subject Test	0.026*	09:02:40
Air Blank	0.000	09:03:12
Air Blank	0.000	09:03:39
Subject Test	XXX**	09:04:06
Air Blank	0.000	09:04:40

\*Deficient Sample  
\*\*Invalid Sample  
Begin new deprivation period

**Not a Successfully Completed Test Sequence**

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

577  
CS

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002265  
10/21/2010 09:04:55

B4

-----  
Standard Lot#: 642845  
Last Changed By: C. STRAUB #50772  
-----

QAS: C. STRAUB #50772  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	09:05:23
Cal Check	0.099	09:05:42
Air Blank	0.000	09:06:11



\_\_\_\_\_  
Operator's Signature

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

By  
10-25-2010

QA Specialist Cheryl Straub AGENCY Tucson Police Department  
DATE 10/21/10 TIME 08:50  
Intoxilyzer Serial # 80-002265 Location TEAM 5

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution N/A AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE

Cheryl Straub

DPS Form Exh G-2 (Rev. 05-01)

797  
CS

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002265  
10/21/2010 08:50:48

BY

-----  
Standard Lot#: 642845  
Last Changed By: C. STRAUB #50772  
-----

QAS: C. STRAUB #50772  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	08:51:16
Cal Check	0.099	08:51:34
Air Blank	0.000	08:52:04



Operator's Signature