



CERTIFICATION by CUSTODIAN of RECORDS
 Tucson Police Department Crime Laboratory
 1306 W. Miracle Mile, Tucson, AZ 85705

SC
11/18/11

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-002265 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 11-10-11 Cal 1 0.100 AC Cal 2 0.099 AC

Between the dates of 10-19-11 and 11-10-11 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on N/A.
 The instrument was returned to service on N/A.

Clare Conley

Clare L. Conley


11-10-11

ADPS QAS Permit # 33314

SUBSCRIBED AND SWORN to before me this 10 day of November, 2011.

Diane Quiroz

Notary Public

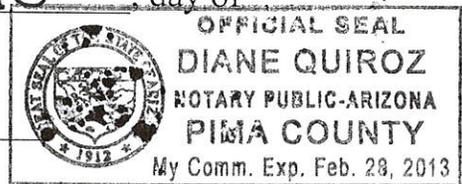

*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

1067
CW
SC
11/18/11

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Clare Conley AGENCY Tucson Police Department

DATE 11-10-11 TIME 0804

INTOXILYZER SERIAL # 80-002265 LOCATION ODW

() 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

() 1. Clock time check.

() 2. Date check.

OPERATIONAL TESTS

() 1. Alcohol-free subject test result 0. 000 AC.

() 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed

() 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.

() 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 099 AC

Instrument is operating properly and accurately. YES NO

COMMENTS None.

SIGNATURE Clare Conley

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-002265
11/10/2011

08:05:50

267
cu
SC

Standard Lot#: 20910100A1
Last Changed By: C. STRAUB #50772

QAS: C. CONLEY #34517
TUCSON PD

Operator: C. CONLEY #34517
TUCSON PD

Subject: TEST,SQAP
DOB: 11/11/1111
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:06:35
Diagnostic Test	Pass	08:07:02
Air Blank	0.000	08:07:30
0.100 Cal Check	0.100	08:07:49
Air Blank	0.000	08:08:18
Subject Test	0.000	08:08:43
Air Blank	ABT*	08:08:50
Air Blank	0.000	08:09:18

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-002265
11/10/2011 08:09:24

347
CW
SC

Standard Lot#: 20910100A1
Last Changed By: C. STRAUB #50772

QAS: C. CONLEY #34517
TUCSON PD

Operator: C. CONLEY #34517
TUCSON PD

Subject: TEST,SQAP
DOB: 11/11/1111
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	08:09:59
Air Blank	RFI**	08:10:07

*Improper Sample
**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-002265
11/10/2011

08:10:15

457
CW
SC

Standard Lot#: 20910100A1
Last Changed By: C. STRAUB #50772

QAS: C. CONLEY #34517
TUCSON PD

Operator: C. CONLEY #34517
TUCSON PD

Subject: TEST,SQAP
DOB: 11/11/1111
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:11:13
Diagnostic Test	Pass	08:11:40
Air Blank	0.000	08:12:08
0.100 Cal Check	0.100	08:12:26
Air Blank	0.000	08:12:56
Subject Test	0.097*	08:16:04
Air Blank	0.000	08:16:36
Air Blank	0.000	08:17:03
Subject Test	XXX**	08:17:28
Air Blank	0.000	08:18:02

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-002265
11/10/2011 08:18:39

5067
CW

Standard Lot#: 20910100A1
Last Changed By: C. STRAUB #50772

SC

QAS: C. CONLEY #34517
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	08:19:07
Cal Check	0.099	08:19:26
Air Blank	0.000	08:19:55

Clare Conley

Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

bob
CW

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

SC
11/18/11

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Clare Conley AGENCY Tucson Police Department
DATE 11-10-11 TIME 0804
Intoxilyzer Serial # 80-002265 Location ODW

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE

Clare Conley

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-002265
11/10/2011 08:04:26

7587
CW

Standard Lot#: 20910100A1
Last Changed By: C. STRAUB #50772

SC

QAS: C. CONLEY #34517
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	08:04:53
Cal Check	0.100	08:05:12
Air Blank	0.000	08:05:42

C. Conley

Operator's Signature