



CERTIFICATION by CUSTODIAN of RECORDS
 Tucson Police Department Crime Laboratory
 1306 W. Miracle Mile, Tucson, AZ 85705

TS4
 9.4.2013

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-002265 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 8/15/13 Cal 1 0.101 AC Cal 2 0.097 AC

Between the dates of 7/18/13 and 8/15/13 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on -.
 The instrument was returned to service on -.

Sarah Cione

Sarah L. Cione


8/29/13

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 29th, day of August, 2013.

Elma Escarcega

Notary Public

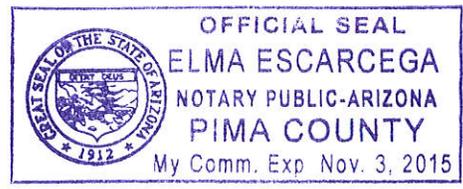

*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

1 of 9 sc
TS4
9.4.2013

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department
DATE 8/15/13 TIME 1509
INTOXILYZER SERIAL # 80-002265 LOCATION Crime Lab

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 097 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002265
Core Version: 8105.48
08/15/2013

15:12:12

279sc
Dy

Standard Lot#: 09312100A1
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:13:11
Diagnostic Test	Pass	15:13:46
Air Blank	0.000	15:14:13
0.100 Cal Check	0.100	15:14:34
Air Blank	0.000	15:15:04
Subject Test	0.000	15:15:36
Air Blank	0.000	15:16:03
Wait	ABT*	
Air Blank	0.000	15:16:38

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002265
Core Version: 8105.48
08/15/2013

15:21:03

3 of 9 sc
D4

Standard Lot#: 30412100A1
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:21:50
Diagnostic Test	Pass	15:22:25
Air Blank	0.000	15:22:53
0.100 Cal Check	0.098	15:23:14
Air Blank	0.000	15:23:43
Subject Test	RFI*	15:23:59
Air Blank	IPS**	15:24:07

*RFI Detect

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002265
Core Version: 8105.48
08/15/2013

15:26:33

4 of 9
sc
BY

Standard Lot#: 30412100A1
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:27:33
Diagnostic Test	Pass	15:28:09
Air Blank	0.000	15:28:36
0.100 Cal Check	0.098	15:28:57
Air Blank	0.000	15:29:27
Subject Test	0.324*	15:32:34
Air Blank	0.000	15:33:06
Air Blank	0.000	15:33:34
Subject Test	XXX**	15:33:45
Air Blank	0.000	15:34:18

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002265
Core Version: 8105.48
08/15/2013 15:34:43

5 of 9 sc
TSY

Standard Lot#: 30412100A1
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	15:35:12
0.100 Cal Check	0.097	15:35:33
Air Blank	0.000	15:36:03

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

6 of 9 sc
TS4
9.4.2013

QA Specialist Sarah Cione AGENCY Tucson Police Department
DATE 8/15/13 TIME 1509
Intoxilyzer Serial # 80-002265 Location Crime Lab

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 101 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002265
Core Version: 8105.48
08/15/2013 15:09:46

7 of 9 sec
TSh

Standard Lot#: 09312100A1
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	15:10:14
0.100 Cal Check	0.101	15:10:35
Air Blank	0.000	15:11:05

Preventative Maintenance
Performed -- Timer Reset



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

8 of 9
TS4
9.4.2013

INTOXILYZER SERVICE RECORD
FOR
CALIBRATION GAS TANK CHANGE

Date: 8/15/13
QAS: Sarah Cione
Intoxilyzer Serial Number: 80-002265
New Lot Number: 30412100A1
New Expiration Date: 12/1/14

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."
Select "D" for dry gas and enter the following information:
 - Enter Target Value: 0.100
 - Enter Control Lot #: From the tank label
 - Enter Exp Date: From the tank label (mm/dd/yyyy)
 - Edit info Y/N? Y or N
 - Update Std Info? Y ENTER
 - "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."
Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 098 AC.
- (✓) 9. Air blank completed.
- () 10. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002265
Core Version: 8105.48
08/15/2013 15:18:52

9 of 9
x
BY

Standard Lot#: 30412100A1
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	15:19:21
0.100 Cal Check	0.098	15:19:42
Air Blank	0.000	15:20:12

Preventative Maintenance NOT
Performed -- Timer NOT Reset