



CERTIFICATION by CUSTODIAN of RECORDS
Tucson Police Department Crime Laboratory

CM
10-26-07

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-002345 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 9.19.2007 Cal 1 0.101 AC Cal 2 0.100 AC

Between the dates of 8.17.2007 and 9.19.2007 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on .
The instrument was returned to service on .


Terry S. Gallegos

Terry S. Gallegos

10.25.2007

ADPS QAS Permit # 33169

SUBSCRIBED AND SWORN to before me this 25, day of October, 2007.

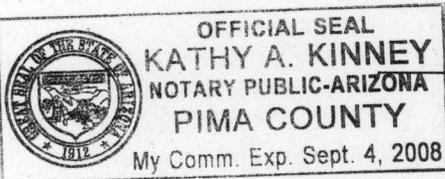

Kathy A Kinney

Notary Public

*Includes documents specified in A.A.C. R13-10-104.A.5

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

CM
10-26-07

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

1 of 7
TS6

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Terry Gallegos 28847 AGENCY Tucson Police Department
DATE 9.19.2007 TIME 1727
INTOXILYZER SERIAL # 80-002345 LOCATION DUI HQ

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS


(✓) 1. Clock time check.
(✓) 2. Date check.

OPERATIONAL TESTS

(✓) 1. Alcohol-free subject test result 0. 000 AC.
(✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed ✓✓✓
(✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 100 AC

Instrument is operating properly and accurately. YES NO

COMMENTS Ex G-2 performed at 33 days. TS6

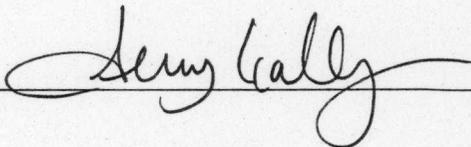SIGNATURE 

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

CM

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-002345
09/19/2007 17:30:15

2077
TS4

Standard Lot#: 706602I
Last Changed By: J. MORGAN III #42130

QAS: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Operator: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Subject: SQAP, SQAP
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:31:05
Diagnostic Test	Pass	17:31:33
Air Blank	0.000	17:32:00
0.100 Cal Check	0.100	17:32:19
Air Blank	0.000	17:32:48
Subject Test	0.000	17:33:24
Air Blank	0.000	17:33:51
Five Minute Wait	ABT*	
Air Blank	0.000	17:34:27

*Sequence Aborted

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000

Location: TUCSON POLICE DEPT

Serial Number : 80-002345

09/19/2007

17:34:33

CM

3 of 7

TG

Standard Lot#: 706602I

Last Changed By: J. MORGAN III #42130

OAS: T GALLEGOS #28847
TUCSON PD CRIME LAB

Operator: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Subject: SQAP, SQAP

DOB: 11/11/1111

Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:35:25
Diagnostic Test	Pass	17:35:52
Air Blank	RFI*	17:36:13
Air Blank	IPS**	17:36:23

*RFI Detect

**Improper Sample

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-002345
09/19/2007 17:36:34

CM

Standard Lot#: 706602I
Last Changed By: J. MORGAN III #42130

4057
TB4

QAS: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Operator: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Subject: SQAP, SQAP
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:37:24
Diagnostic Test	Pass	17:37:51
Air Blank	0.000	17:38:18
0.100 Cal Check	0.100	17:38:37
Air Blank	0.000	17:39:07
Subject Test	0.218*	17:42:15
Air Blank	0.000	17:42:48
Air Blank	0.000	17:43:15
Subject Test	XXX**	17:43:37
Air Blank	0.000	17:44:11

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-002345
09/19/2007 17:45:23

CM

Standard Lot#: 706602I
Last Changed By: J. MORGAN III #42130

5087
TB4

QAS: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Test	g/210L	Time
Air Blank	0.000	17:45:51
Cal Check	0.100	17:46:09
Air Blank	0.000	17:46:39


Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

CM
10-26-07


ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

6 of 7
TS4

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Terry Gallegos 29847 AGENCY Tucson Police Department
DATE 9.19.2007 TIME 1727
Intoxilyzer Serial # 80-002345 Location DUI HQ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 101 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

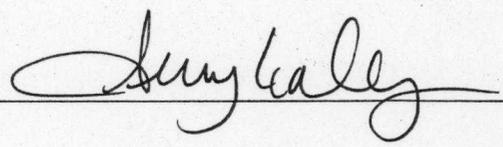SIGNATURE 

INTOXILYZER 8000

Location: TUCSON POLICE DEPT

Serial Number: 80-002345

09/19/2007

17:27:42

CM

Standard Lot#: 706602I

Last Changed By: J. MORGAN III #42130


7077
134

QAS: T. GALLEGOS #28847

TUCSON PD CRIME LAB

Test g/210L Time

Air Blank 0.000 17:28:09
Cal Check 0.101 17:28:28
Air Blank 0.000 17:28:57

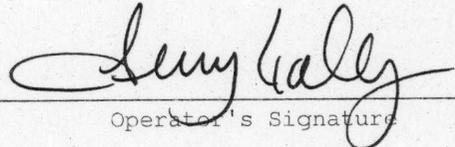

Operator's Signature



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

CM
10-26-07

INTOXILYZER SERVICE RECORD
FOR
CALIBRATION GAS TANK CHANGE

10f 2
TS6

Date: 9.3.07
 QAS: John Morgan
 Intoxilyzer Serial Number: 80-002345
 New Lot Number: 706602I
 New Expiration Date: 11 MAR 2009

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."
 Select "D" for dry gas and enter the following information:
 Enter Target Value: 0.100
 Enter Control Lot #: From the tank label
 Enter Exp Date: From the tank label (mm/dd/yyyy)
 Edit info Y/N? Y or N
 Update Std Info? Y ENTER
 "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."
 Select "D" for dry control test. After selection is made press
 ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 100 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove and sign printed record. Attach the record to the completed checklist.

SIGNATURE John Morgan #42130

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-002345
09/03/2007 22:23:49

CM
20f2
TS4

Standard Lot#: 7066021
Last Changed By: J. MORGAN III #42130

QAS: J. MORGAN III #42130
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	22:24:16
Cal Check	0.100	22:24:35
Air Blank	0.000	22:25:04

J. Morgan #42130
Operator's Signature