



CERTIFICATION by CUSTODIAN of RECORDS
 Tucson Police Department Crime Laboratory
 270 S. Stone Avenue Tucson, AZ 85701-1917

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-002345 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

TS4
3-17-09

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 3/6/09 Cal 1 0.103 AC Cal 2 0.103 AC

Between the dates of 2/6/09 and 3/6/09 the instrument was operating properly and accurately.


The instrument was taken out of service for repair on -.
 The instrument was returned to service on -.

Sarah Cione
 Sarah L. Cione

3/17/09
 ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 17, day of March, 2009.

A.M. Hernandez
 Notary Public

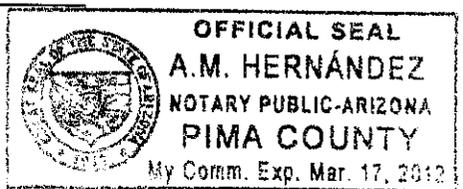

*Includes documents specified in A.A.C. R13-10-104.A.5

1 of 7sc

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

TS4
3/17/09

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 3/6/09 TIME 11:06

INTOXILYZER SERIAL # 80-002345 LOCATION Team 3

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 103 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Sarah Cione

2 of 7sc
TS4

INTOXILYZER 8000
Location: ARIZONA
Serial Number : 80-002345
03/06/2009 11:08:13

Standard Lot#: 826001I
Last Changed By: C. CONLEY #34517

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SOAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:08:57
Diagnostic Test	Pass	11:09:24
Air Blank	0.000	11:09:52
0.100 Cal Check	0.103	11:10:11
Air Blank	0.000	11:10:40
Subject Test	0.000	11:10:56
Air Blank	0.000	11:11:24
Five Minute Wait	ABT*	
Air Blank	0.000	11:11:57

*Sequence Aborted

Not a Successfully Completed Test Sequence

Pursuant to section 28-1527, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department

INTOXILYZER 8000
Location: ARIZONA
Serial Number : 80-002345
03/06/2009

11:12:04

3 of 7_{sc}

Standard Lot#: 826001I
Last Changed By: C. CONLEY #34517

BY

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	11:12:31
Air Blank	IPS**	11:12:39

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

4 of 7 SC

154

INTOXILYZER 8000
Location: ARIZONA
Serial Number : 80-002345
03/06/2009 11:12:44

Standard Lot#: 826001I
Last Changed By: C. CONLEY #34517

OAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SOAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:13:27
Diagnostic Test	Pass	11:13:54
Air Blank	0.000	11:14:22
0.100 Cal Check	0.103	11:14:40
Air Blank	0.000	11:15:10
Subject Test	0.026*	11:18:18
Air Blank	0.000	11:18:50
Air Blank	0.000	11:19:18
Subject Test	XXX**	11:19:34
Air Blank	0.000	11:20:07

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: ARIZONA
Serial Number: 80-002345
03/06/2009 11:20:20

5 of 7_{sc}

Standard Lot#: 826001I
Last Changed By: C. CONLEY #34517

TS4

QAS: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	11:20:48
Cal Check	0.103	11:21:07
Air Blank	0.000	11:21:36

Sarah Cione
Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6 of 7 SC

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

TS4
3-17-09

QA Specialist Sarah Cione AGENCY Tucson Police Department
DATE 3/6/09 TIME 11:06
Intoxilyzer Serial # 80-002345 Location Team 3

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 103 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE

Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000
Location: ARIZONA
Serial Number: 80-002345
03/06/2009 11:06:48

7 of 7 sc

Standard Lot#: 826001I
Last Changed By: C. CONLEY #34517

154

QAS: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	11:07:15
Cal Check	0.103	11:07:34
Air Blank	0.000	11:08:04

Sarah Cione

Operator's Signature