



CERTIFICATION by CUSTODIAN of RECORDS
 Tucson Police Department Crime Laboratory
 270 S. Stone Avenue Tucson, AZ 85701-1917

134
7-1-09

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-002345 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 6-26-09 Cal 1 0.102 AC Cal 2 0.099 AC

Between the dates of 5-28-09 and 6-26-09 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on N/A.
 The instrument was returned to service on N/A.

Clare Conley

Clare L. Conley


6-30-09

ADPS QAS Permit # 33314

SUBSCRIBED AND SWORN to before me this 30th day of June, 2009.

A.M. Hernandez

Notary Public

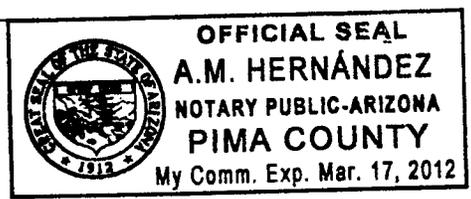

*Includes documents specified in A.A.C. R13-10-104.A.5

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

1587
cu
Ply 7-1-09

QA SPECIALIST Clare Conley AGENCY Tucson Police Department

DATE 6-26-09 TIME 0837

INTOXILYZER SERIAL # 80-002345 LOCATION ODM

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 099 AC

Instrument is operating properly and accurately. YES NO

COMMENTS None.

SIGNATURE Clare Conley

INTOXILYZER 8000
Location: ARIZONA
Serial Number : 80-002345
06/26/2009

08:38:49

2 of 7
On
104

Standard Lot#: 826001I
Last Changed By: C. CONLEY #34517

QAS: C. CONLEY #34517
TUCSON PD

Operator: C. CONLEY #34517
TUCSON PD

Subject: TEST,SQAP
DOB: 11/11/1111
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:39:33
Diagnostic Test	Pass	08:40:01
Air Blank	0.000	08:40:28
0.100 Cal Check	0.102	08:40:47
Air Blank	0.000	08:41:16
Subject Test	0.000	08:41:32
Air Blank	ABT*	08:41:38
Air Blank	0.000	08:42:07

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: ARIZONA
Serial Number : 80-002345
06/26/2009

08:42:11

387
cu
py

Standard Lot#: 826001I
Last Changed By: C. CONLEY #34517

QAS: C. CONLEY #34517
TUCSON PD

Operator: C. CONLEY #34517
TUCSON PD

Subject: TEST,SQAP
DOB: 11/11/1111
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	08:42:38
Air Blank	IPS**	08:42:45

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: ARIZONA
Serial Number : 80-002345
06/26/2009

08:42:50

487
Cm
104

Standard Lot#: 826001I
Last Changed By: C. CONLEY #34517

QAS: C. CONLEY #34517
TUCSON PD

Operator: C. CONLEY #34517
TUCSON PD

Subject: TEST,SQAP
DOB: 11/11/1111
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:43:34
Diagnostic Test	Pass	08:44:01
Air Blank	0.000	08:44:28
0.100 Cal Check	0.102	08:44:47
Air Blank	0.000	08:45:16
Subject Test	0.419*	08:48:25
Air Blank	0.000	08:48:59
Air Blank	0.000	08:49:27
Subject Test	XXX**	08:49:47
Air Blank	0.000	08:50:22

*Deficient Sample
**Invalid Sample
Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: ARIZONA
Serial Number: 80-002345
06/26/2009 08:50:38

Standard Lot#: 826001I
Last Changed By: C. CONLEY #34517

QAS: C. CONLEY #34517
TUCSON PD

5867
Cn
754

Test	g/210L	Time
Air Blank	0.000	08:51:06
Cal Check	0.099	08:51:24
Air Blank	0.000	08:51:54

Clare Conley
Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

6507
134
7-1-09
Cm

QA Specialist Clare Conley AGENCY Tucson Police Department
DATE 6-26-09 TIME 0837
Intoxilyzer Serial # 80-002345 Location DDM

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C + 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 102 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Clare Conley

DPS Form Exh G-2 (Rev. 05-01)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: ARIZONA
Serial Number: 80-002345
06/26/2009 08:37:26

Standard Lot#: 826001I
Last Changed By: C. CONLEY #34517

QAS: C. CONLEY #34517
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	08:37:53
Cal Check	0.102	08:38:12
Air Blank	0.000	08:38:41

Clare Conley

Operator's Signature

7567
Cm

164