



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory
1306 W. Miracle Mile, Tucson, AZ 85705

TS4
3-20-2012

I hereby certify that these documents are a true and correct copy of the
periodic maintenance\* for the Intoxilyzer 8000 Serial Number
80-002345 maintained by the Tucson Police Department
Crime Laboratory pursuant to the requirements of the Arizona Department
of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following
dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2,
for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 3/9/12 Cal 1 0.102 AC Cal 2 0.101 AC

Between the dates of 2/9/12 and 3/9/12
the instrument was operating properly and accurately.


The instrument was taken out of service for repair on -
The instrument was returned to service on -

Sarah Cione

Sarah L. Cione

3/9/12


ADPS QAS Permit # 33355

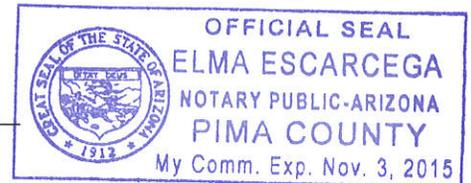

SUBSCRIBED AND SWORN to before me this 9th, day of
March, 2012.

Elma Escarcega

Notary Public

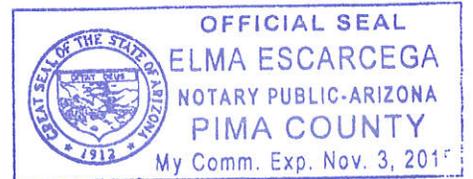

\*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

1 of 7  
SC

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

TSK  
3-20-2012

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 3/9/12 TIME 1133

INTOXILYZER SERIAL # 80-002345 LOCATION Tm 5-0DD

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 101 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS

---



---



---



---

SIGNATURE Sarah Cione

2 of 7  
SC

TSH

INTOXILYZER 8000  
Location: ARIZONA  
Serial Number : 80-002345  
03/09/2012 11:35:13

---

Standard Lot#: 20910100A1  
Last Changed By: C. STRAUB #50772

---

QAS: S. CIONE #50616  
TUCSON PD

---

Operator: S. CIONE #50616  
TUCSON PD

---

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	11:36:07
Diagnostic Test	Pass	11:36:35
Air Blank	0.000	11:37:02
0.100 Cal Check	0.101	11:37:21
Air Blank	0.000	11:37:50
Subject Test	0.000	11:38:18
Air Blank	0.000	11:38:45
Five Minute Wait	ABT*	
Air Blank	0.000	11:39:27

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

3 of 7  
sc

INTOXILYZER 8000  
Location: ARIZONA  
Serial Number : 80-002345  
03/09/2012 11:39:31

TS4

---

Standard Lot#: 20910100A1  
Last Changed By: C. STRAUB #50772

---

QAS: S. CIONE #50616  
TUCSON PD

---

Operator: S. CIONE #50616  
TUCSON PD

---

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	11:39:55
Air Blank	IPS**	11:40:04

\*RFI Detect  
\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

4 of 7  
SC

INTOXILYZER 8000  
Location: ARIZONA  
Serial Number : 80-002345  
03/09/2012

11:40:22

TS4

Standard Lot#: 20910100A1  
Last Changed By: C. STRAUB #50772

QAS: S. CIONE #50616  
TUCSON PD

Operator: S. CIONE #50616  
TUCSON PD

Subject: SQAP, SQAP  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:41:05
Diagnostic Test	Pass	11:41:32
Air Blank	0.000	11:42:00
0.100 Cal Check	0.102	11:42:19
Air Blank	0.000	11:42:48
Subject Test	0.078*	11:45:56
Air Blank	0.000	11:46:28
Air Blank	0.000	11:46:55
Subject Test	XXX**	11:47:28
Air Blank	0.000	11:48:02

\*Deficient Sample  
\*\*Invalid Sample  
Begin new deprivation period

**Not a Successfully Completed Test Sequence**

5 of 7  
sc

INTOXILYZER 8000  
Location: ARIZONA  
Serial Number: 80-002345  
03/09/2012 11:48:16

Tsh

-----  
Standard Lot#: 20910100A1  
Last Changed By: C. STRAUB #50772  
-----

QAS: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	11:48:44
Cal Check	0.101	11:49:02
Air Blank	0.000	11:49:32

*Sarah Cione*

\_\_\_\_\_  
Operator's Signature

EXHIBIT G-2

6 of 7  
JC

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

TS4  
3.20.2012

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Sarah Cione AGENCY Tucson Police Department  
DATE 3/9/12 TIME 1133  
Intoxilyzer Serial # 80-002345 Location Tm 5-0DD

- ( ✓ ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- ( ✓ ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ( ✓ ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ( ✓ ) 4. Air blank completed.
- ( ✓ ) 5. Calibration check completed. Test results 0. 102 AC.
- ( ✓ ) 6. Air blank completed.
- ( ✓ ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

7 of 7  
F SC

INTOXILYZER 8000  
Location: ARIZONA  
Serial Number: 80-002345  
03/09/2012 11:33:48

TS

-----  
Standard Lot#: 20910100A1  
Last Changed By: C. STRAUB #50772  
-----

QAS: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	11:34:15
Cal Check	0.102	11:34:34
Air Blank	0.000	11:35:04

Sarah Cione  
Operator's Signature