

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

1 of 7
JLS
TSh
8.6.2015

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Amy Astret 53546 AGENCY Tucson Police Department
DATE 7/24/15 TIME 1236
INTOXILYZER SERIAL # 80-002345 LOCATION 000

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 098 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE



2067
JLB
T34

INTOXILYZER 8000
Location:
Serial Number: 80-002345
Core Version: 8105.48
07/24/2015

12:38:11

Standard Lot#: 25414100A4
Last Changed By: A. AUTRET #53546

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	12:39:37
Diagnostic Test	Pass	12:40:12
Air Blank	0.000	12:40:40
0.100 Cal Check	0.100	12:41:01
Air Blank	0.000	12:41:30
Subject Test	0.000	12:41:49
Air Blank	0.000	12:42:17
Wait	ABT*	
Air Blank	0.000	12:42:52

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location:
Serial Number: 80-002345
Core Version: 8105.48
07/24/2015

12:42:56

3 of 7
JL
TS4

Standard Lot#: 25414100A4
Last Changed By: A. AUTRET #53546

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	12:43:54
Diagnostic Test	Pass	12:44:29
Air Blank	0.000	12:44:57
0.100 Cal Check	RFI*	12:45:09
Air Blank	IPS**	12:45:17

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

4 of 7
JL
Bh

INTOXILYZER 8000
Location:
Serial Number: 80-002345
Core Version: 8105.48
07/24/2015

12:45:38

Standard Lot#: 25414100A4
Last Changed By: A. AUTRET #53546

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	12:46:35
Diagnostic Test	Pass	12:47:10
Air Blank	0.000	12:47:37
0.100 Cal Check	0.100	12:47:58
Air Blank	0.000	12:48:28
Subject Test	0.202*	12:51:35
Air Blank	0.000	12:52:08
Air Blank	0.000	12:52:35
Subject Test	XXX**	12:52:58
Air Blank	0.000	12:53:32

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location:

Serial Number: 80-002345

Core Version: 8105.48

07/24/2015

12:54:02

5 of 7
Ab
Bh

Standard Lot#: 25414100A4

Last Changed By: A. AUTRET #53546

QAS: A. AUTRET #53546

TUCSON PD

Operator: A. AUTRET #53546

TUCSON PD

Test g/210L Time

Air Blank 0.000 12:54:31
0.100 Cal Check 0.098 12:54:52
Air Blank 0.000 12:55:21

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6 of 7
old
TS4
8-6-2015

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Amy Autret 53546 AGENCY Tucson Police Department
DATE 7/24/15 TIME 1236
Intoxilyzer Serial # 80-002345 Location 000

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000
Location:
Serial Number: 80-002345
Core Version: 8105.48
07/24/2015 12:36:33

7 of 7
dd
TS

Standard Lot#: 25414100A4
Last Changed By: A. AUTRET #53546

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	12:37:01
0.100 Cal Check	0.100	12:37:22
Air Blank	0.000	12:37:52

Preventative Maintenance
Performed -- Timer Reset