



CERTIFICATION by CUSTODIAN of RECORDS  
 Tucson Police Department Crime Laboratory  
 270 S. Stone Avenue Tucson, AZ 85701-1917

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-202759 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

TS4  
3-17-09

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 3-6-09 Cal 1 0.096 AC Cal 2 0.095 AC

Between the dates of 2-6-09 and 3-6-09 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on N/A.  
 The instrument was returned to service on N/A.

Clare Conley

Clare L. Conley


3-17-09

ADPS QAS Permit # 33314

SUBSCRIBED AND SWORN to before me this 17, day of March, 2009.

A.M. Hernandez

Notary Public

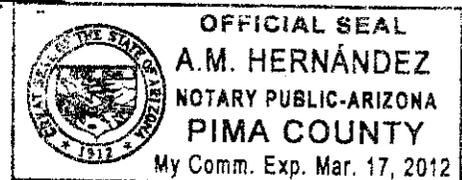

\*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD QUALITY ASSURANCE PROCEDURE

189  
cu  
154  
3-17-09

QA SPECIALIST Clare Conley AGENCY Tucson Police Department  
DATE 3-6-09 TIME 0732  
INTOXILYZER SERIAL # 80-002759 LOCATION Crime Lab

( ✓ ) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- ( ✓ ) 1. Clock time check.
- ( ✓ ) 2. Date check.

OPERATIONAL TESTS

- ( ✓ ) 1. Alcohol-free subject test result 0. 020 AC.
- ( ✓ ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- ( ✓ ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- ( ✓ ) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 095 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS None.  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE Clare Conley

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number : 80-002759  
03/06/2009

07:33:59

Standard Lot#: 725401I  
Last Changed By: S. CIONE #50616

QAS: C. CONLEY #34517  
TUCSON PD

Operator: C. CONLEY #34517  
TUCSON PD

Subject: TEST, SQAP  
DOB: 11/11/1111  
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:34:44
Diagnostic Test	Pass	07:35:12
Air Blank	0.000	07:35:39
0.100 Cal Check	0.096	07:35:58
Air Blank	0.000	07:36:28
Subject Test	0.000	07:36:45
Air Blank	ABT*	07:36:52
Air Blank	0.000	07:37:20

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

2869  
cu

TS4

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number : 80-002759  
03/06/2009

07:37:29

Standard Lot#: 725401I  
Last Changed By: S. CIONE #50616

QAS: C. CONLEY #34517  
TUCSON PD

Operator: C. CONLEY #34517  
TUCSON PD

Subject: TEST, SQAP  
DOB: 11/11/1111  
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	07:37:58
Air Blank	IPS**	07:38:09

\*RFI Detect  
\*\*Improper Sample

349  
en

134

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number : 80-002759  
03/06/2009

07:38:19

Standard Lot#: 725401I  
Last Changed By: S. CIONE #50616

QAS: C. CONLEY #34517  
TUCSON PD

Operator: C. CONLEY #34517  
TUCSON PD

Subject: TEST, SQAP  
DOB: 11/11/1111  
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:39:06
Diagnostic Test	Pass	07:39:34
Air Blank	0.000	07:40:01
0.100 Cal Check	0.096	07:40:20
Air Blank	0.000	07:40:49
Subject Test	0.210*	07:43:58
Air Blank	0.000	07:44:31
Air Blank	0.000	07:44:59
Subject Test	XXX**	07:45:23
Air Blank	0.000	07:45:57

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

4569  
ew

154

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-002759  
03/06/2009

07:46:34

-----  
Standard Lot#: 725401I  
Last Changed By: S. CIONE #50616  
-----

-----  
QAS: C. CONLEY #34517  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	07:47:01
Cal Check	0.095	07:47:20
Air Blank	0.000	07:47:50

*C. Conley*

\_\_\_\_\_  
Operator's Signature

5069  
cu  
154

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

689  
Cu  
TSG  
3-17-09

QA Specialist Clare Conley AGENCY Tucson Police Department  
DATE 3-6-09 TIME 0732  
Intoxilyzer Serial # 80-00 2759 Location Crime Lab

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C + 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 096 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Clare Conley

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000  
 Location: TUCSON PD  
 Serial Number: 80-002759  
 03/06/2009 07:32:38

7569  
 CW

-----  
 Standard Lot#: 725401I  
 Last Changed By: S. CIONE #50616  
 -----

QAS: C. CONLEY #34517  
 TUCSON PD  
 -----

154

Test	g/210L	Time
Air Blank	0.000	07:33:05
Cal Check	0.096	07:33:24
Air Blank	0.000	07:33:54

*Clare Conley*

Operator's Signature



TUCSON POLICE DEPARTMENT  
CRIME LABORATORY

8-6-09  
CW

INTOXILYZER SERVICE RECORD  
FOR  
CALIBRATION GAS TANK CHANGE

Date: 3.6.09  
 QAS: Clare Conley  
 Intoxilyzer Serial Number: 80-002759  
 New Lot Number: 826001 I  
 New Expiration Date: 9/24/2010

TSY  
3-17-09

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change." Select "D" for dry gas and enter the following information:  
 Enter Target Value: 0.100  
 Enter Control Lot #: From the tank label  
 Enter Exp Date: From the tank label (mm/dd/yyyy)  
 Edit info Y/N? Y or N  
 Update Std Info? Y ENTER  
 "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing." Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 101 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove and sign printed record. Attach the record to the completed checklist.

SIGNATURE Clare Conley

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-002759  
03/06/2009

07:53:09

-----  
Standard Lot#: 826001I  
Last Changed By: C. CONLEY #34517  
-----

QAS: C. CONLEY #34517  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	07:53:36
Cal Check	0.101	07:53:55
Air Blank	0.000	07:54:25

*Clare Conley*

\_\_\_\_\_  
Operator's Signature

989  
Cn

154



TUCSON POLICE DEPARTMENT  
CRIME LABORATORY

1 of 2<sub>sc</sub>  
TSY  
3/17/09

INTOXILYZER SERVICE RECORD  
FOR  
CALIBRATION GAS TANK CHANGE

Date: 3/6/09  
QAS: Sarah Cione  
Intoxilyzer Serial Number: 80-002759  
New Lot Number: 808101G  
New Expiration Date: 3/31/2010

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."  
Select "D" for dry gas and enter the following information:  
Enter Target Value: 0.100  
Enter Control Lot #: From the tank label  
Enter Exp Date: From the tank label (mm/dd/yyyy)  
Edit info Y/N? Y or N  
Update Std Info? Y ENTER  
"Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."  
Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 100 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove and sign printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-002759  
03/06/2009

12:55:29

-----  
Standard Lot#: 808101G  
Last Changed By: S. CIONE #50616  
-----

QAS: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	12:55:57
Cal Check	0.100	12:56:15
Air Blank	0.000	12:56:45

2 of 250

154

*Sarah Cione*

Operator's Signature