



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory
1306 W. Miracle Mile, Tucson, AZ 85705

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-002759 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

SC
9/25/12

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 9.20.2012 Cal 1 0.098 AC Cal 2 0.097 AC

Between the dates of 8.23.2012 and 9.20.2012 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on .
The instrument was returned to service on .

Terry S. Gallegos

Terry S. Gallegos


9.24.2012

ADPS QAS Permit # 33169

SUBSCRIBED AND SWORN to before me this 24th, day of September, 2012.

Adina Escarcega

Notary Public

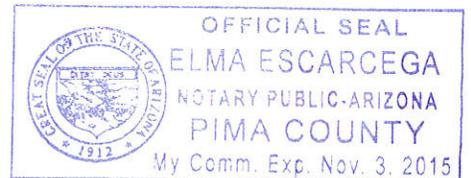

*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

10 of 7
Dg

SC
a/25/12

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Terry Gallegos 28847 AGENCY Tucson Police Department

DATE 9-20-2012 TIME 0907

INTOXILYZER SERIAL # 80-002759 LOCATION ODE

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 097 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE 

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location:
Serial Number : 80-002759
09/20/2012

09:09:25

2 of 7
TS4
JC

Standard Lot#: 23511100A2
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|------------------|--------|----------|
| Air Blank | 0.000 | 09:10:18 |
| Diagnostic Test | Pass | 09:10:45 |
| Air Blank | 0.000 | 09:11:12 |
| 0.100 Cal Check | 0.097 | 09:11:31 |
| Air Blank | 0.000 | 09:12:01 |
| Subject Test | 0.000 | 09:12:28 |
| Air Blank | 0.000 | 09:12:56 |
| Five Minute Wait | ABT* | |
| Air Blank | 0.000 | 09:13:40 |

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location:
Serial Number : 80-002759
09/20/2012

09:13:52

30 of 7
TS4
SC

Standard Lot#: 23511100A2
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 09:14:45 |
| Diagnostic Test | Pass | 09:15:12 |
| Air Blank | 0.000 | 09:15:40 |
| 0.100 Cal Check | RFI* | 09:15:57 |
| Air Blank | IPS** | 09:16:08 |

*RFI Detect

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location:
Serial Number : 80-002759
09/20/2012

09:16:19

4087
TS4
SC

Standard Lot#: 23511100A2
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SQAP

DOB: 11/11/1987

Sex: F

Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 09:17:12 |
| Diagnostic Test | Pass | 09:17:39 |
| Air Blank | 0.000 | 09:18:06 |
| 0.100 Cal Check | 0.098 | 09:18:25 |
| Air Blank | 0.000 | 09:18:55 |
| Subject Test | 0.092* | 09:22:03 |
| Air Blank | 0.000 | 09:22:35 |
| Air Blank | 0.000 | 09:23:02 |
| Subject Test | XXX** | 09:23:24 |
| Air Blank | 0.000 | 09:23:57 |

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location:

Serial Number: 80-002759

09/20/2012

09:24:19

50F7

TB4

SC


Standard Lot#: 23511100A2

Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847

TUCSON PD

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | 0.000 | 09:24:46 |
| Cal Check | 0.097 | 09:25:05 |
| Air Blank | 0.000 | 09:25:35 |

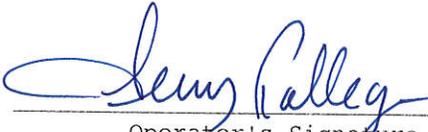

Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

6077
D4
SC
9/25/12

QA Specialist Terry Gallegos 28847 AGENCY Tucson Police Department
DATE 9.20.2012 TIME 0907
Intoxilyzer Serial # 80002759 Location ODE

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 098 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Terry Gallegos

DPS Form Exh G-2 (Rev. 05-01)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000

Location:

Serial Number: 80-002759

09/20/2012

09:07:27

Standard Lot#: 23511100A2


Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847

TUCSON PD

7077
TS4
SC

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | 0.000 | 09:07:54 |
| Cal Check | 0.098 | 09:08:13 |
| Air Blank | 0.000 | 09:08:42 |

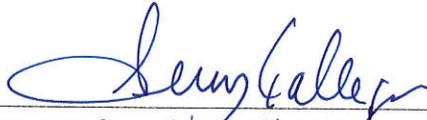

Operator's Signature