

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

1017
alt
TS4
6.1.2015

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Amy Autret 53546 AGENCY Tucson Police Department

DATE 5-28-15 TIME 1252

INTOXILYZER SERIAL # 80-002759 LOCATION OPW

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.


OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 097 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE

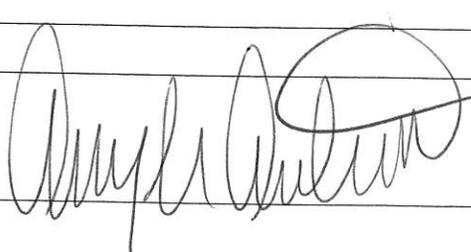

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002759
Core Version: 8105.48
05/28/2015

12:55:14

2087
JLB
754

Standard Lot#: 11514100A2
Last Changed By: S. CIONE #50616

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	12:56:22
Diagnostic Test	Pass	12:56:57
Air Blank	0.000	12:57:24
0.100 Cal Check	0.098	12:57:45
Air Blank	0.000	12:58:14
Subject Test	0.000	12:58:33
Air Blank	0.000	12:59:00
Wait	ABT*	
Air Blank	0.000	12:59:35

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002759
Core Version: 8105.48
05/28/2015

13:00:18

3 of 7
JLB
754

Standard Lot#: 11514100A2
Last Changed By: S. CIONE #50616

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:01:18
Diagnostic Test	Pass	13:01:53
Air Blank	0.000	13:02:21
0.100 Cal Check	RFI*	13:02:36
Air Blank	IPS**	13:02:44

*RFI Detect

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002759
Core Version: 8105.48
05/28/2015

13:03:46

4087
Dk
TS4

Standard Lot#: 11514100A2
Last Changed By: S. CIONE #50616

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:04:44
Diagnostic Test	Pass	13:05:19
Air Blank	0.000	13:05:47
0.100 Cal Check	0.098	13:06:08
Air Blank	0.000	13:06:37
Subject Test	0.095*	13:09:45
Air Blank	0.000	13:10:17
Air Blank	0.000	13:10:44
Subject Test	XXX**	13:11:13
Air Blank	0.000	13:11:46

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002759
Core Version: 8105.48
05/28/2015 13:12:44

5 of 7
JL
184

Standard Lot#: 11514100A2
Last Changed By: S. CIONE #50616

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	13:13:14
0.100 Cal Check	0.097	13:13:35
Air Blank	0.000	13:14:05

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6087
JLB
TS4

6-1-2015

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Amy Autret 53546 AGENCY Tucson Police Department

DATE 5-28-15 TIME 1252

Intoxilyzer Serial # 80-002759 Location OPW

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002759
Core Version: 8105.48
05/28/2015 12:52:28

7 of 7
JLB
TSG

Standard Lot#: 11514100A2
Last Changed By: S. CIONE #50616

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	12:52:59
0.100 Cal Check	0.099	12:53:20
Air Blank	0.000	12:53:50

Preventative Maintenance
Performed -- Timer Reset