



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory
1306 W. Miracle Mile, Tucson, AZ 85705

TSG
6-5-2013

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-002760 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 5/23/13 Cal 1 0.101 AC Cal 2 0.100 AC

Between the dates of 4/25/13 and 5/23/13 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on -.
The instrument was returned to service on -.

Sarah Cione

Sarah L. Cione


5/30/13

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 30th, day of May, 2013.

Elma Escarcega

Notary Public

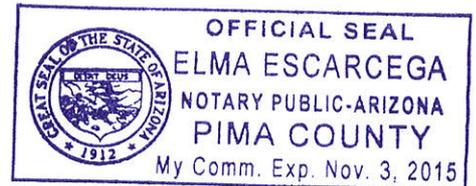

*Includes documents specified in A.A.C. R13-10-104.A.5

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

1 of 9 sc
T84
6.5.2013

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 5/23/13 TIME 817

INTOXILYZER SERIAL # 80-002760 LOCATION DUI HQ

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 100 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Sarah Cione

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002760
Core Version: 8105.48
05/23/2013

08:21:15

2 of 9 sc
BH

Standard Lot#: 23511100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:22:17
Diagnostic Test	Pass	08:22:52
Air Blank	0.000	08:23:19
0.100 Cal Check	0.102	08:23:40
Air Blank	0.000	08:24:10
Subject Test	0.000	08:24:34
Air Blank	0.000	08:25:02
Wait	ABT*	
Air Blank	0.000	08:25:44

*Sequence Aborted

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002760
Core Version: 8105.48
05/23/2013

08:29:18

3 of 9 sc
Bh

Standard Lot#: 23511100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:30:13
Diagnostic Test	Pass	08:30:48
Air Blank	0.000	08:31:16
0.100 Cal Check	0.102	08:31:36
Air Blank	0.000	08:32:06
Subject Test	RFI*	08:32:27
Air Blank	IPS**	08:32:39

*RFI Detect

**Improper Sample

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002760
Core Version: 8105.48
05/23/2013

08:32:45

4 of 9 sc
TSh

Standard Lot#: 23511100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:33:38
Diagnostic Test	Pass	08:34:13
Air Blank	0.000	08:34:41
0.100 Cal Check	0.101	08:35:01
Air Blank	0.000	08:35:31
Subject Test	0.318*	08:38:39
Air Blank	0.000	08:39:13
Air Blank	0.000	08:39:40
Subject Test	XXX**	08:40:01
Air Blank	0.000	08:40:36

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002760
Core Version: 8105.48
05/23/2013 08:41:05

5 of 9 sc
TS4

Standard Lot#: 23511100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	08:41:34
0.100 Cal Check	0.100	08:41:55
Air Blank	0.000	08:42:24

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

6 of 9 sc
TS4
6.5.2013

QA Specialist Sarah Cione AGENCY Tucson Police Department
DATE 5/23/13 TIME 817
Intoxilyzer Serial # 80-002760 Location DUI HQ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C + 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 101 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002760
Core Version: 8105.48
05/23/2013 08:17:48

7 of 9 SC
TSG

Standard Lot#: 23511100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	08:18:16
0.100 Cal Check	0.101	08:18:37
Air Blank	0.000	08:19:07

Preventative Maintenance
Performed -- Timer Reset



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

8 of 9 sc

INTOXILYZER SERVICE RECORD
FOR
CALIBRATION GAS TANK CHANGE

TS4
6-5-2013

Date: 5/23/13
 QAS: Sarah Cione
 Intoxilyzer Serial Number: 80-002760
 New Lot Number: 30412100A1
 New Expiration Date: 12/1/14

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change." Select "D" for dry gas and enter the following information:
 Enter Target Value: 0.100
 Enter Control Lot #: From the tank label
 Enter Exp Date: From the tank label (mm/dd/yyyy)
 Edit info Y/N? Y or N
 Update Std Info? Y ENTER
 "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing." Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 097 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002760
Core Version: 8105.48
05/23/2013 08:45:05

9 of 9 sc
TS4

Standard Lot#: 30412100A1
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	08:45:34
0.100 Cal Check	0.097	08:45:55
Air Blank	0.000	08:46:24

Preventative Maintenance
Performed -- Timer Reset