



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory
270 S. Stone Avenue Tucson, AZ 85701-1917

TS4
2-4-2009

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-002803 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 1-15-09 Cal 1 0.100 AC Cal 2 0.098 AC

Between the dates of 12-18-08 and 1-15-09 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on N/A.
The instrument was returned to service on N/A.

Clare Conley

Clare L. Conley


2-3-09

ADPS QAS Permit # 33314

SUBSCRIBED AND SWORN to before me this 3rd, day of February, 2009.

A.M. Hernandez

Notary Public

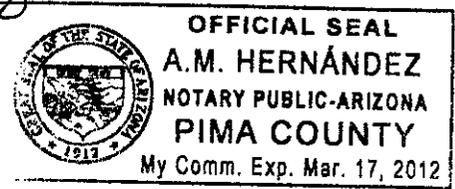

*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

1007
cu
B4
2-4-09

QA SPECIALIST Clare Conley AGENCY Tucson Police Department

DATE 1-15-09 TIME 0842

INTOXILYZER SERIAL # 80-002803 LOCATION ODW

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 098 AC

Instrument is operating properly and accurately. YES NO

COMMENTS None.

SIGNATURE Clare Conley

INTOXILYZER 8000
Location: ARIZONA
Serial Number : 80-002803
01/15/2009

08:43:29

287
cu
134

Standard Lot#: 804302I
Last Changed By: C.-CONLEY #34517

QAS: C. CONLEY #34517
TUCSON PD

Operator: C. CONLEY #34517
TUCSON PD

Subject: TEST,SQAP
DOB: 11/11/1111
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:44:44
Diagnostic Test	Pass	08:45:11
Air Blank	0.000	08:45:38
0.100 Cal Check	0.100	08:45:57
Air Blank	0.000	08:46:27
Subject Test	0.000	08:46:53
Air Blank	ABT*	08:47:01
Air Blank	0.000	08:47:30

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: ARIZONA
Serial Number : 80-002803
01/15/2009

08:47:36

3567
cu

Standard Lot#: 804302I
Last Changed By: C.-CONLEY #34517

164

QAS: C. CONLEY #34517
TUCSON PD

Operator: C. CONLEY #34517
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1111
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	08:48:09
Air Blank	IPS**	08:48:18

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: ARIZONA
Serial Number : 80-002803
01/15/2009

08:48:34

4567
cu

Standard Lot#: 804302I
Last Changed By: C.-CONLEY #34517

BY

QAS: C. CONLEY #34517
TUCSON PD

Operator: C. CONLEY #34517
TUCSON PD

Subject: TEST, SQAP

DOB: 11/11/1111

Sex: M

Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:49:26
Diagnostic Test	Pass	08:49:53
Air Blank	0.000	08:50:21
0.100 Cal Check	0.100	08:50:39
Air Blank	0.000	08:51:09
Subject Test	0.379*	08:54:17
Air Blank	0.000	08:54:57
Air Blank	0.000	08:55:24
Subject Test	XXX**	08:55:52
Air Blank	0.000	08:56:27

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: ARIZONA
Serial Number: 80-002803
01/15/2009 08:57:01

5 of 7
Cu
154

Standard Lot#: 804302I
Last Changed By: C. CONLEY #34517

QAS: C. CONLEY #34517
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	08:57:29
Cal Check	0.098	08:57:48
Air Blank	0.000	08:58:17

Clare Conley

Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

687
cu

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

TSY
2-4-2009

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Clare Conley AGENCY Tucson Police Department
DATE 1-15-09 TIME 0842
Intoxilyzer Serial # 80-002803 Location ODW

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Clare Conley

DPS Form Exh G-2 (Rev. 05-01)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: ARIZONA
Serial Number: 80-002803
01/15/2009 08:42:03

7567
en

Standard Lot#: 804302I
Last Changed By: C. CONLEY #34517

754

QAS: C. CONLEY #34517
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	08:42:30
Cal Check	0.100	08:42:49
Air Blank	0.000	08:43:19

Clare Conley

Operator's Signature