

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

1 of 7
Dy
5.7.2015

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Amy Autret 53546 AGENCY Tucson Police Department

DATE 4/30/2015 TIME 1431

INTOXILYZER SERIAL # 80-002803 LOCATION Crime Lab

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.


OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 099 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE

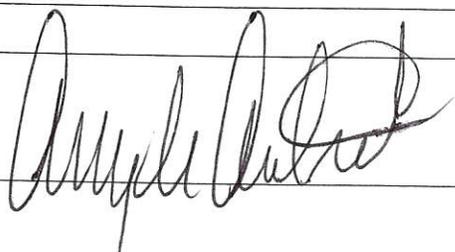

20F7
[Signature]
TS4

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002803
Core Version: 8105.48
04/30/2015 14:34:22

Standard Lot#: 25414100A4
Last Changed By: T. GALLEGOS #28847

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Subject: SOAP, SOAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	14:35:27
Diagnostic Test	Pass	14:36:05
Air Blank	0.000	14:36:32
0.100 Cal Check	0.100	14:36:53
Air Blank	0.000	14:37:23
Subject Test	0.000	14:37:39
Air Blank	0.000	14:38:07
Wait	ABT*	
Air Blank	0.000	14:38:43

*Sequence Aborted

Not a Successfully
Completed
Test Sequence

3 of 7
[Handwritten initials]

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002803
Core Version: 8105.48
04/30/2015 14:40:01

Standard Lot#: 25414100A4
Last Changed By: T. GALLEGOS #28847

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Subject: SOAP, SOAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	14:41:18
Diagnostic Test	Pass	14:41:55
Air Blank	0.000	14:42:22
0.100 Cal Check	RFI*	14:42:34
Air Blank	IPS**	14:42:43

*RFI Detect
**Improper Sample

Not a Successfully
Completed
Test Sequence

4 of 7
JLB
134

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002803
Core Version: 8105.48
04/30/2015 14:44:14

Standard Lot#: 25414100A4
Last Changed By: T. GALLEGOS #28847

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Subject: SOAP, SOAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	14:45:48
Diagnostic Test	Pass	14:46:26
Air Blank	0.000	14:46:53
0.100 Cal Check	0.101	14:47:14
Air Blank	0.000	14:47:43
Subject Test	0.235*	14:50:51
Air Blank	0.000	14:51:24
Air Blank	0.000	14:51:51
Subject Test	XXX**	14:52:21
Air Blank	0.000	14:52:55

*Deficient Sample
**Invalid Sample
Begin new deprivation period

Not a Successfully
Completed
Test Sequence

5 of 7
AA
Dy

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002803
Core Version: 8105.48
04/30/2015 14:54:13

Standard Lot#: 25414100A4
Last Changed By: T. GALLEGOS #28847

OAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	14:54:44
0.100 Cal Check	0.099	14:55:05
Air Blank	0.000	14:55:34

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6 of 7
JBY
5.7.2015

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Amy Autret 53546 AGENCY Tucson Police Department
DATE 4/30/2015 TIME 1431
Intoxilyzer Serial # 80-002803 Location Crime Lab

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C + 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

DPS Form Exh G-2 (Rev. 05-01)

7 of 7
AA
TSL

INTOXILYZER 8000
Instrument Initialization
13:37:20 04/30/2015

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-002803
Core Version: 8105.48
04/30/2015 14:31:12

Standard Lot#: 25414100A4
Last Changed By: T. GALLEGOS #28847

QAS: A. AUTRET #53546
TUCSON PD

Operator: A. AUTRET #53546
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	14:31:42
0.100 Cal Check	0.100	14:32:03
Air Blank	0.000	14:32:32

Preventative Maintenance
Performed -- Timer Reset