



CERTIFICATION by CUSTODIAN of RECORDS
 Tucson Police Department Crime Laboratory
 1306 W. Miracle Mile, Tucson, AZ 85705

Ab
 7/30/15

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-006048 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 7.16.2015 Cal 1 0.099 AC Cal 2 0.099 AC

Between the dates of 6.18.2015 and 7.16.2015 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on —.
 The instrument was returned to service on —.

Terry S. Gallegos

Terry S. Gallegos


7.30.2015

ADPS QAS Permit # 33169

SUBSCRIBED AND SWORN to before me this 30th, day of July, 2015.

Elma Escarcega

Notary Public

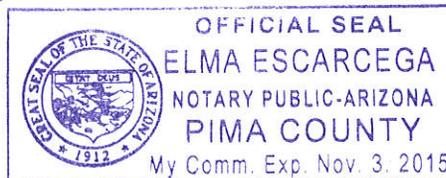

*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

1 of 9
TS4
7/30/15

QA SPECIALIST Terry Gallegos 28847 AGENCY Tucson Police Department

DATE 7-16-2015 TIME 1110

INTOXILYZER SERIAL # 80-006048 LOCATION DUI-HQ

() 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 099 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Terry Gallegos

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

2 of 9
754
dd

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006048
Core Version: 8105.48
07/16/2015 11:12:14

Standard Lot#: 25414100A4
Last Changed By: J. KNEPPER #46585

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 11:13:08 |
| Diagnostic Test | Pass | 11:13:43 |
| Air Blank | 0.000 | 11:14:10 |
| 0.100 Cal Check | 0.099 | 11:14:31 |
| Air Blank | 0.000 | 11:15:00 |
| Subject Test | 0.000 | 11:15:19 |
| Air Blank | 0.000 | 11:15:46 |
| Wait | ABT* | |
| Air Blank | 0.000 | 11:16:22 |

*Sequence Aborted

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

3 of 9
TS4
[Signature]

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006048
Core Version: 8105.48
07/16/2015 11:24:19

Standard Lot#: 25414100A4
Last Changed By: J. KNEPPER #46585

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 11:25:14 |
| Diagnostic Test | Pass | 11:25:49 |
| Air Blank | 0.000 | 11:26:16 |
| 0.100 Cal Check | RFI* | 11:26:31 |
| Air Blank | IPS** | 11:26:46 |

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006048
Core Version: 8105.48
07/16/2015

11:26:58

4 of 9
754
[Signature]

Standard Lot#: 25414100A4
Last Changed By: J. KNEPPER #46585

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 11:27:43 |
| Diagnostic Test | Pass | 11:28:18 |
| Air Blank | 0.000 | 11:28:46 |
| 0.100 Cal Check | 0.099 | 11:29:07 |
| Air Blank | 0.000 | 11:29:36 |
| Subject Test | 0.061* | 11:32:43 |
| Air Blank | 0.000 | 11:33:15 |
| Air Blank | 0.000 | 11:33:43 |
| Subject Test | XXX** | 11:33:58 |
| Air Blank | 0.000 | 11:34:31 |

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006048
Core Version: 8105.48
07/16/2015 11:35:24

50f9
TS4
JL

Standard Lot#: 25414100A4
Last Changed By: J. KNEPPER #46585

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 11:35:53 |
| 0.100 Cal Check | 0.099 | 11:36:14 |
| Air Blank | 0.000 | 11:36:43 |

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

60 of 9
TS4
JL
7/20/15

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Terry Gallegos 28847 AGENCY Tucson Police Department
DATE 7-16-2015 TIME 1110
Intoxilyzer Serial # 80-006048 Location DUI-HQ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Terry Gallegos

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006048
Core Version: 8105.48
07/16/2015 11:10:41

TS4
7 of 9
TS4
JL

Standard Lot#: 25414100A4
Last Changed By: J. KNEPPER #46585

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 11:11:11 |
| 0.100 Cal Check | 0.099 | 11:11:31 |
| Air Blank | 0.000 | 11:12:01 |

Preventative Maintenance
Performed -- Timer Reset



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

8 of 9
TS4
7/30/15

INTOXILYZER SERVICE RECORD
FOR
CALIBRATION GAS TANK CHANGE

Date: 7-16-2015
QAS: Terry Gallegos 28847
Intoxilyzer Serial Number: 80-006048
New Lot Number: 25414100A4
New Expiration Date: 10-5-2016

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."
Select "D" for dry gas and enter the following information:
Enter Target Value: 0.100
Enter Control Lot #: From the tank label
Enter Exp Date: From the tank label (mm/dd/yyyy)
Edit info Y/N? Y or N
Update Std Info? Y ENTER
"Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."
Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 098 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Terry Gallegos

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006048
Core Version: 8105.48
07/16/2015 12:08:17

90f9
TSK
dd

Standard Lot#: 25414100A4
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 12:08:51 |
| 0.100 Cal Check | 0.098 | 12:09:11 |
| Air Blank | 0.000 | 12:09:41 |

Preventative Maintenance
Performed -- Timer Reset