



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory
1306 W. Miracle Mile, Tucson, AZ 85705

By
4.29.2014

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-006051 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 4/24/14 Cal 1 0.102 AC Cal 2 0.099 AC

Between the dates of 3/27/14 and 4/24/14 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on -.
The instrument was returned to service on -.

Sarah Cione

Sarah L. Cione


4/28/14

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 28th, day of April, 2014.

Elma Escarcega

Notary Public

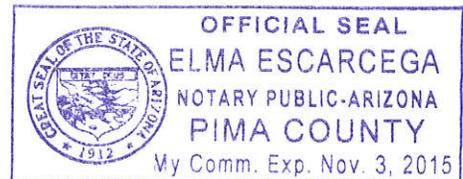

*Includes documents specified in A.A.C. R13-10-104.A.5

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

1 of 9 sc
TS4
4.29.2014

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 4/24/14 TIME 759

INTOXILYZER SERIAL # 80-006051 LOCATION DUI HQ

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 099 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006051
Core Version: 8105.48
04/24/2014

08:10:52

2 of 9
x
TS4

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:11:46
Diagnostic Test	Pass	08:12:21
Air Blank	0.000	08:12:48
0.100 Cal Check	0.100	08:13:09
Air Blank	0.000	08:13:39
Subject Test	0.000	08:14:12
Air Blank	0.000	08:14:39
Wait	ABT*	
Air Blank	0.000	08:15:42

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006051
Core Version: 8105.48
04/24/2014 08:18:24

3 of 9 SK
BH

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:19:18
Diagnostic Test	Pass	08:19:53
Air Blank	0.000	08:20:21
0.100 Cal Check	0.099	08:20:41
Air Blank	0.000	08:21:11
Subject Test	RFI*	08:21:26
Air Blank	IPS**	08:21:33

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006051
Core Version: 8105.48
04/24/2014

08:25:15

4 of 9
SC
TBH

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, SQAP
DOB: 11/11/1980
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:26:06
Diagnostic Test	Pass	08:26:41
Air Blank	0.000	08:27:08
0.100 Cal Check	0.100	08:27:29
Air Blank	0.000	08:27:59
Subject Test	0.331*	08:31:06
Air Blank	0.000	08:31:39
Air Blank	0.000	08:32:06
Subject Test	XXX**	08:32:30
Air Blank	0.000	08:33:04

*Deficient Sample
**Invalid Sample
Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006051
Core Version: 8105.48
04/24/2014 08:33:44

5 of 9 sc
TS4

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	08:34:12
0.100 Cal Check	0.099	08:34:33
Air Blank	0.000	08:35:03

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

6 of 9
x
154

QA Specialist Sarah Cione AGENCY Tucson Police Department
DATE 4/24/14 TIME 759
Intoxilyzer Serial # 80-006051 Location DUI HQ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 102 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006051
Core Version: 8105.48
04/24/2014 07:59:33

7 of 9
SC
154

Standard Lot#: 19613100A4
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	08:00:02
0.100 Cal Check	0.102	08:00:22
Air Blank	0.000	08:00:52

Preventative Maintenance
Performed -- Timer Reset



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

8 of 9
TS4
4.29.2014

INTOXILYZER SERVICE RECORD
FOR
CALIBRATION GAS TANK CHANGE

Date: 4/24/14
QAS: Sarah Cione
Intoxilyzer Serial Number: 80-006051
New Lot Number: 25913100A2
New Expiration Date: 10/1/15

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change." Select "D" for dry gas and enter the following information:
 Enter Target Value: 0.100
 Enter Control Lot #: From the tank label
 Enter Exp Date: From the tank label (mm/dd/yyyy)
 Edit info Y/N? Y or N
 Update Std Info? Y ENTER
 "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing." Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 099 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006051
Core Version: 8105.48
04/24/2014 08:09:03

9 of 9
SC
154

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	08:09:32
0.100 Cal Check	0.099	08:09:52
Air Blank	0.000	08:10:22

Preventative Maintenance
Performed -- Timer Reset