



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory
1306 W. Miracle Mile, Tucson, AZ 85705

SC
8/12/14

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-006052 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 7-24-2014 Cal 1 0.099 AC Cal 2 0.097 AC

Between the dates of 6-25-2014 and 7-24-2014 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on -.
The instrument was returned to service on -.


Terry S. Gallegos

8-12-2014

ADPS QAS Permit # 33169

SUBSCRIBED AND SWORN to before me this 12, day of August, 2014.

Notary Public

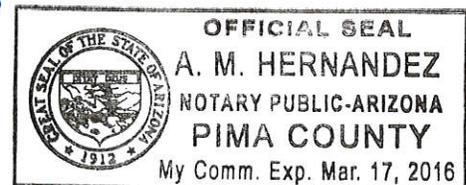

*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

1 of 7
TS4

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

SC
8/12/14

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Terry Gallegos 28847 AGENCY Tucson Police Department

DATE 7.24.2014 TIME 1000

INTOXILYZER SERIAL # 80-006052 LOCATION ODE Sgt's Office

() 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 097 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE

Terry Gallegos

2 of 7
TS4

5C

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006052
Core Version: 8105.48
07/24/2014 10:01:55

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:02:43
Diagnostic Test	Pass	10:03:21
Air Blank	0.000	10:03:48
0.100 Cal Check	0.098	10:04:09
Air Blank	0.000	10:04:38
Subject Test	0.000	10:05:14
Air Blank	0.000	10:05:41
Wait	ABT*	
Air Blank	0.000	10:06:20

*Sequence Aborted

Not a Successfully
Completed
Test Sequence

3087
TS4

50

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006052
Core Version: 8105.48
07/24/2014 10:07:45

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST,SOAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:08:33
Diagnostic Test	Pass	10:09:10
Air Blank	0.000	10:09:38
0.100 Cal Check	RFI*	10:09:52
Air Blank	IPS**	10:09:59

*RFI Detect
**Improper Sample

Not a Successfully
Completed
Test Sequence

40f7
154
SC

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006052
Core Version: 8105.48
07/24/2014 10:10:07

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

DAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST_SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:11:02
Diagnostic Test	Pass	10:11:39
Air Blank	0.000	10:12:06
0.100 Cal Check	0.098	10:12:27
Air Blank	0.000	10:12:57
Subject Test	0.233*	10:16:04
Air Blank	0.000	10:16:41
Air Blank	0.000	10:17:09
Subject Test	XXX**	10:17:33
Air Blank	0.000	10:18:07

*Deficient Sample
**Invalid Sample
Begin new deprivation period

Not a Successfully
Completed
Test Sequence

5077
T54

SC

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006052
Core Version: 8105.48
07/24/2014 10:18:25

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	10:18:54
0.100 Cal Check	0.097	10:19:15
Air Blank	0.000	10:19:45

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6 of 7
TS4
SC
8/12/14

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Terry Gallegos 28847 AGENCY Tucson Police Department
DATE 7-24-2014 TIME 1000
Intoxilyzer Serial # 80-006052 Location ODE Sgt's Office

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE

Terry Gallegos

DPS Form Exh G-2 (Rev. 05-01)

70F7
TS4
SC

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006052
Core Version: 8105.48
07/24/2014 10:00:22

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Test	g/210L	Time
Air Blank	0.000	10:00:49
0.100 Cal Check	0.099	10:01:10
Air Blank	0.000	10:01:39

1