



EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

10F7  
TJ4

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

sc  
11/12/14

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Terry Gallegos 28847 AGENCY Tucson Police Department

DATE 10-15-2014 TIME 1009

INTOXILYZER SERIAL # 80-006052 LOCATION ODE Sgt's Office

(  ) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.
- (  ) 2. Date check.

OPERATIONAL TESTS

- (  ) 1. Alcohol-free subject test result 0. 000 AC.
- (  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (  ) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 100 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS

---



---



---



---

SIGNATURE Terry Gallegos

2087  
Tdy  
SL

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-006052  
Core Version: 8105.48  
10/15/2014 10:12:13

Standard Lot#: 25913100A2  
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847  
TUCSON PD

Operator: T. GALLEGOS #28847  
TUCSON PD

Subject: TEST, SOAP  
DOB: 11/11/1987  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:13:08
Diagnostic Test	Pass	10:13:45
Air Blank	0.000	10:14:13
0.100 Cal Check	0.099	10:14:33
Air Blank	0.000	10:15:03
Subject Test	0.000	10:15:26
Air Blank	0.000	10:15:54
Wait	ABT*	
Air Blank	0.000	10:16:28

\*Sequence Aborted

Not a Successfully  
Completed  
Test Sequence

30F7  
TSM

SC

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-006052  
Core Version: 8105.48  
10/15/2014 10:16:36

Standard Lot#: 25913100A2  
Last Changed By: S. CIONE #50616

OAS: T. GALLEGOS #28847  
TUCSON PD

Operator: T. GALLEGOS #28847  
TUCSON PD

Subject: TEST, SOAP  
DOB: 11/11/1987  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:17:28
Diagnostic Test	Pass	10:18:06
Air Blank	0.000	10:18:33
0.100 Cal Check	RFI*	10:18:49
Air Blank	IPS**	10:18:57

\*RFI Detect  
\*\*Improper Sample

Not a Successfully  
Completed  
Test Sequence

4 of 7  
TS4

5C

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-006052  
Core Version: 8105.48  
10/15/2014 10:19:04

-----  
Standard Lot#: 25913100A2  
Last Changed By: S. CLONE #50616  
-----

DAS: T. GALLEGOS #28847  
TUCSON PD  
-----

Operator: T. GALLEGOS #28847  
TUCSON PD  
-----

Subject: TEST,SOAP  
DOB: 11/11/1987  
Sex: F Weight: 111  
-----

15 Minute Deprivation Period? Yes  
-----

Test	g/210L	Time
Air Blank	0.000	10:19:52
Diagnostic Test	Pass	10:20:30
Air Blank	0.000	10:20:57
0.100 Cal Check	0.100	10:21:18
Air Blank	0.000	10:21:47
Subject Test	0.207*	10:24:55
Air Blank	0.000	10:25:27
Air Blank	0.000	10:25:55
Subject Test	XXX**	10:26:12
Air Blank	0.000	10:26:46

\*Deficient Sample  
\*\*Invalid Sample  
Begin new deprivation period

Not a Successfully  
Completed  
Test Sequence

50f7  
784

5c

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-006052  
Core Version: 8105.48  
10/15/2014 10:27:05

-----  
Standard Lot#: 25913100A2  
Last Changed By: S. CIONE #50616  
-----

DAS: T. GALLEGOS #28847  
TUCSON PD  
-----

Operator: T. GALLEGOS #28847  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	10:27:34
0.100 Cal Check	0.100	10:27:55
Air Blank	0.000	10:28:24

Preventative Maintenance  
Performed -- Timer Reset

|

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

607  
TS4

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

5C  
11/12/14

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Terry Gallegos 28847 AGENCY Tucson Police Department  
DATE 10-15-2014 TIME 1009  
Intoxilyzer Serial # 80-006052 Location ODE Sgt's Office

- ( ✓ ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- ( ✓ ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ( ✓ ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ( ✓ ) 4. Air blank completed.
- ( ✓ ) 5. Calibration check completed. Test results 0. 099 AC.
- ( ✓ ) 6. Air blank completed.
- ( ✓ ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Terry Gallegos

DPS Form Exh G-2 (Rev. 05-01)

7 of 7 754

SC

INTOXILYZER 8000  
Instrument Initialization  
09:36:36 10/15/2014

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-006052  
Core Version: 8105.48  
10/15/2014 10:09:40

-----  
Standard Lot#: 25913100A2  
Last Changed By: S. CLONE #50616  
-----

QAS: T. GALLEGOS #28847  
TUCSON PD  
-----

Operator: T. GALLEGOS #28847  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	10:10:11
0.100 Cal Check	0.099	10:10:32
Air Blank	0.000	10:11:02

Preventative Maintenance  
Performed -- Timer Reset