



CERTIFICATION by CUSTODIAN of RECORDS
 Tucson Police Department Crime Laboratory
 1306 W. Miracle Mile, Tucson, AZ 85705

SC
 12/19/14

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-006052 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 12.11.2014 Cal 1 0.100 AC Cal 2 0.099 AC

Between the dates of 11.13.2014 and 12.11.2014 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on —.
 The instrument was returned to service on —.

Terry S. Gallegos

Terry S. Gallegos


12.17.2014

ADPS QAS Permit # 33169

SUBSCRIBED AND SWORN to before me this 17th, day of December, 2014.

Elma Escarcega

Notary Public

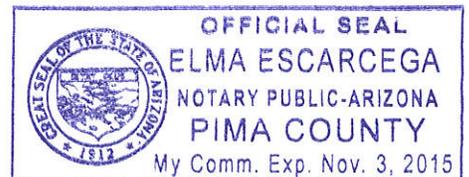

*Includes documents specified in A.A.C. R13-10-104.A.5

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

10f7
TS4

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

SC
12/19/14

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Terry Gallegos 28847 AGENCY Tucson Police Department

DATE 12-11-2014 TIME 1141

INTOXILYZER SERIAL # 80-006052 LOCATION ODE Sgt's Office

() 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 099 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Terry Gallegos

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

2 of 7
TS4

SC

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006052
Core Version: 8105.48
12/11/2014 11:43:24

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SOAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:44:19
Diagnostic Test	Pass	11:44:56
Air Blank	0.000	11:45:24
0.100 Cal Check	0.100	11:45:45
Air Blank	0.000	11:46:14
Subject Test	0.000	11:46:36
Air Blank	0.000	11:47:03
Wait	ABT*	
Air Blank	0.000	11:47:38

*Sequence Aborted

Not a Successfully
Completed
Test Sequence

30f7
TS4

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006052
Core Version: 8105.48
12/11/2014 11:48:09

SC

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST,SOAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:48:59
Diagnostic Test	Pass	11:49:36
Air Blank	0.000	11:50:03
0.100 Cal Check	RFI*	11:50:20
Air Blank	IPS**	11:50:27

*RFI Detect
**Improper Sample

Not a Successfully
Completed
Test Sequence

4 of 7
B6

SC

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006052
Core Version: 8105.48
12/11/2014 11:50:36

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST_SOAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:51:24
Diagnostic Test	Pass	11:52:01
Air Blank	0.000	11:52:28
0.100 Cal Check	0.100	11:52:49
Air Blank	0.000	11:53:19
Subject Test	0.134*	11:56:26
Air Blank	0.000	11:56:58
Air Blank	0.000	11:57:26
Subject Test	XXX**	11:57:48
Air Blank	0.000	11:58:22

*Deficient Sample
**Invalid Sample
Begin new deprivation period

Not a Successfully
Completed
Test Sequence

Sof 7
Dh

SC

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006052
Core Version: 8105.48
12/11/2014 11:58:56

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

DAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	11:59:26
0.100 Cal Check	0.099	11:59:46
Air Blank	0.000	12:00:16

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6077
754

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

SC

12/19/14

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Terry Gallegos 28842 AGENCY Tucson Police Department
DATE 12-11-2014 TIME 1141
Intoxilyzer Serial # 80-006052 Location ODE Sgt's Office

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Terry Gallegos

DPS Form Exh G-2 (Rev. 05-01)

7 of 7
TS4

SC

INTOXILYZER 8000
Instrument Initialization
10:54:28 12/11/2014

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006052
Core Version: 8105.48
12/11/2014 11:41:44

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	11:42:14
0.100 Cal Check	0.100	11:42:35
Air Blank	0.000	11:43:05

Preventative Maintenance
Performed -- Timer Reset



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

1 of 3
SC

TS4
12.17.2014

INTOXILYZER SERVICE RECORD

Date: 12/1/14

QAS: Sarah Lionel

Intoxilyzer Serial Number: 80-006052

Reason For Service: Received call on 11/28/14 from Officer Bonnano. He notified me that intox had a "Cal Check Out of Tolerance". Tank appeared to have become loose. Tightened tank and ran a cal check. Cal check within acceptable limits.

Date Out of Service: N/A

Calibration Check completed prior to removal from service

Date: -

SQAP completed prior to removal from service

Date: -

Corrective Action:

Sent to CMI, Inc.

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

2 of 3
SC
B4

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Sarah Cione AGENCY Tucson Police Department
DATE 12/1/14 TIME 1340
Intoxilyzer Serial # 80-006052 Location Crime Lab

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

3 of 3 sc
TSy

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006052
Core Version: 8105.48
12/01/2014 13:40:28

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	13:40:58
0.100 Cal Check	0.100	13:41:19
Air Blank	0.000	13:41:48

Preventative Maintenance
Performed -- Timer Reset