



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory  
1306 W. Miracle Mile, Tucson, AZ 85705

*Mo*  
10/5/15

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-006052 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 9-17-2015 Cal 1 0.098 AC Cal 2 0.098 AC

Between the dates of 8-21-2015 and 9-17-2015 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on                     .  
The instrument was returned to service on                     .

*Terry S. Gallegos*

Terry S. Gallegos


9-29-2015

ADPS QAS Permit # 33169

SUBSCRIBED AND SWORN to before me this 29<sup>th</sup>, day of September, 20 15.

*Elma Escarcega*

Notary Public

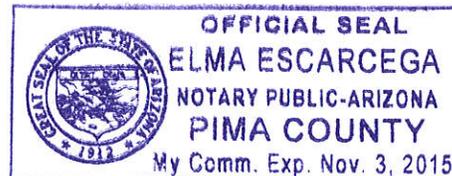

\*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

1 of 7  
TS4  
DAB  
10/5/15

QA SPECIALIST Terry Gallegos 28847 AGENCY Tucson Police Department

DATE 9.17.2015 TIME 0953

INTOXILYZER SERIAL # 80-006052 LOCATION ODE - Sgt's Office

(  ) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.
- (  ) 2. Date check.

OPERATIONAL TESTS

- (  ) 1. Alcohol-free subject test result 0. 000 AC.
- (  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (  ) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 098 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS

---



---



---



---

SIGNATURE 

2 of 7  
TS4  
[Signature]

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-006052  
Core Version: 8105.48  
09/17/2015 09:55:26

-----  
Standard Lot#: 25913100A2  
Last Changed By: S. CIONE #50616  
-----

OAS: T. GALLEGOS #28847  
TUCSON PD  
-----

Operator: T. GALLEGOS #28847  
TUCSON PD  
-----

Subject: TEST,SOAP  
DOB: 11/11/1987  
Sex: F Weight: 111  
-----

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:56:18
Diagnostic Test	Pass	09:56:55
Air Blank	0.000	09:57:22
0.100 Cal Check	0.098	09:57:43
Air Blank	0.000	09:58:13
Subject Test	0.000	09:59:03
Air Blank	0.000	09:59:31
Wait	ABT*	
Air Blank	0.000	10:00:04

\*Sequence Aborted

Not a Successfully  
Completed  
Test Sequence

3 of 7  
TS6  
[Signature]

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-006052  
Core Version: 8105.48  
09/17/2015 10:01:06

-----  
Standard Lot#: 25913100A2  
Last Changed By: S. CIONE #50616  
-----

OAS: T. GALLEGOS #28847  
TUCSON PD  
-----

Operator: T. GALLEGOS #28847  
TUCSON PD  
-----

Subject: TEST,SOAP  
DOB: 11/11/1987  
Sex: F Weight: 111  
-----

15 Minute Deprivation Period? Yes  
-----

Test	g/210L	Time
Air Blank	0.000	10:02:21
Diagnostic Test	Pass	10:02:58
Air Blank	0.000	10:03:26
0.100 Cal Check	RFI*	10:03:39
Air Blank	IPS**	10:03:49

\*RFI Detect  
\*\*Improper Sample

Not a Successfully  
Completed  
Test Sequence

4 of 7  
Tdy  
[Signature]

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-006052  
Core Version: 8105.48  
09/17/2015 10:03:59

-----  
Standard Lot#: 25913100A2  
Last Changed By: S. CIONE #50616  
-----

QAS: T. GALLEGOS #28847  
TUCSON PD  
-----

Operator: T. GALLEGOS #28847  
TUCSON PD  
-----

Subject: TEST, SQAP  
DOB: 11/11/1987  
Sex: F Weight: 111  
-----

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:04:52
Diagnostic Test	Pass	10:05:29
Air Blank	0.000	10:05:56
0.100 Cal Check	0.098	10:06:17
Air Blank	0.000	10:06:46
Subject Test	0.387*	10:09:54
Air Blank	0.000	10:10:27
Air Blank	0.000	10:10:54
Subject Test	XXX**	10:11:14
Air Blank	0.000	10:11:48

\*Deficient Sample  
\*\*Invalid Sample  
Begin new deprivation period

Not a Successfully  
Completed  
Test Sequence

5 of 7  
TSY  
AD

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-006052  
Core Version: 8105.48  
09/17/2015 10:12:13

-----  
Standard Lot#: 25913100A2  
Last Changed By: S. CIONE #50616  
-----

DAS: T. GALLEGOS #28847  
TUCSON PD  
-----

Operator: T. GALLEGOS #28847  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	10:12:42
0.100 Cal Check	0.098	10:13:03
Air Blank	0.000	10:13:33

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

60F7  
B4  
10/5/15

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Terry Gallegos 28847 AGENCY Tucson Police Department  
DATE 9.17.2015 TIME 0953  
Intoxilyzer Serial # 80-006052 Location ODE - Sgt's Office

- ( ✓ ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- ( ✓ ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ( ✓ ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ( ✓ ) 4. Air blank completed.
- ( ✓ ) 5. Calibration check completed. Test results 0. 098 AC.
- ( ✓ ) 6. Air blank completed.
- ( ✓ ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

7077  
TS4  
*[Signature]*

INTOXILYZER 8000  
Instrument Initialization  
09:24:56 09/17/2015

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-006052  
Core Version: 8105.48  
09/17/2015 09:53:33

-----  
Standard Lot#: 25913100A2  
Last Changed By: S. CIONE #50616  
-----

DAS: T. GALLEGOS #28847  
TUCSON PD  
-----

Operator: T. GALLEGOS #28847  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	09:54:03
0.100 Cal Check	0.098	09:54:24
Air Blank	0.000	09:54:53

Preventative Maintenance  
Performed -- Timer Reset