



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory  
1306 W. Miracle Mile, Tucson, AZ 85705

SC  
12/10/14

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-006053 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 11.7.2014 Cal 1 0.099 AC Cal 2 0.098 AC

Between the dates of        and        the instrument was operating properly and accurately.

The instrument was taken out of service for repair on       .  
The instrument was returned to service on 11.7.2014.

Terry S. Gallegos

Terry S. Gallegos


12.4.2014

ADPS QAS Permit # 33169

SUBSCRIBED AND SWORN to before me this 4th, day of December, 2014.

Elma Escarcega

Notary Public

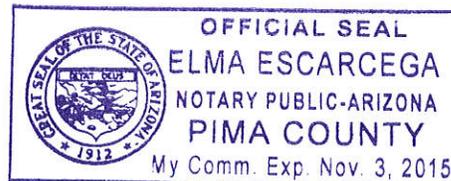

\*Includes documents specified in A.A.C. R13-10-104.A.5



TUCSON POLICE DEPARTMENT  
CRIME LABORATORY

INTOXILYZER SERVICE RECORD

10/12  
TDH

50

QAS: Terry Gallegos 28847

Intoxilyzer Serial Number: 80-006053

Date Returned to Service: 11-7-2014

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

20f12  
TS4

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

5C  
12/10/14

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Terry Gallegos 28844 AGENCY Tucson Police Department

DATE 11-7-2014 TIME 1609

INTOXILYZER SERIAL # 80-006053 LOCATION Crime Lab

(  ) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.
- (  ) 2. Date check.

OPERATIONAL TESTS

- (  ) 1. Alcohol-free subject test result 0. 000 AC.
- (  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (  ) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 098 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE Terry Gallegos

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-006053  
Core Version: 8105.48  
11/07/2014

16:21:30

30 of 12  
TS4  
SC

---

Standard Lot#: 25913100A2  
Last Changed By: T. GALLEGOS #28847

---

QAS: T. GALLEGOS #28847  
TUCSON PD

---

Operator: T. GALLEGOS #28847  
TUCSON PD

---

Subject: TEST, SQAP  
DOB: 11/11/1987  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	16:22:15
Diagnostic Test	Pass	16:22:50
Air Blank	0.000	16:23:18
0.100 Cal Check	0.100	16:23:38
Air Blank	0.000	16:24:08
Subject Test	0.000	16:24:24
Air Blank	0.000	16:24:51
Wait	ABT*	
Air Blank	0.000	16:25:25

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-006053  
Core Version: 8105.48  
11/07/2014

16:25:30

40 of 12  
TS4  
SC

---

Standard Lot#: 25913100A2  
Last Changed By: T. GALLEGOS #28847

---

QAS: T. GALLEGOS #28847  
TUCSON PD

---

Operator: T. GALLEGOS #28847  
TUCSON PD

---

Subject: TEST, SQAP  
DOB: 11/11/1987  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	16:26:18
Diagnostic Test	Pass	16:26:53
Air Blank	0.000	16:27:20
0.100 Cal Check	RFI*	16:27:36
Air Blank	IPS**	16:27:47

\*RFI Detect  
\*\*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-006053  
Core Version: 8105.48  
11/07/2014

16:27:52

5 of 12  
734

50

Standard Lot#: 25913100A2  
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847  
TUCSON PD

Operator: T. GALLEGOS #28847  
TUCSON PD

Subject: TEST, SQAP  
DOB: 11/11/1987  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	16:28:40
Diagnostic Test	Pass	16:29:15
Air Blank	0.000	16:29:43
0.100 Cal Check	0.099	16:30:03
Air Blank	0.000	16:30:33
Subject Test	0.075*	16:33:40
Air Blank	0.000	16:34:12
Air Blank	0.000	16:34:40
Subject Test	XXX**	16:34:56
Air Blank	0.000	16:35:30

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-006053  
Core Version: 8105.48  
11/07/2014 16:35:59

60 of 12  
TS4  
5C

-----  
Standard Lot#: 25913100A2  
Last Changed By: T. GALLEGOS #28847  
-----

QAS: T. GALLEGOS #28847  
TUCSON PD  
-----

Operator: T. GALLEGOS #28847  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	16:36:28
0.100 Cal Check	0.098	16:36:49
Air Blank	0.000	16:37:19

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

7 of 12  
TS4

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

SC  
12/10/14

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Terry Gallegos 28847 AGENCY Tucson Police Department  
DATE 11-7-2014 TIME 1609  
Intoxilyzer Serial # 80-006053 Location Crime Lab

- ( ✓ ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- ( ✓ ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ( ✓ ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ( ✓ ) 4. Air blank completed.
- ( ✓ ) 5. Calibration check completed. Test results 0. 099 AC.
- ( ✓ ) 6. Air blank completed.
- ( ✓ ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Terry Gallegos

DPS Form Exh G-2 (Rev. 05-01)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: TUCSON PD  
Serial Number: 80-006053  
Core Version: 8105.48  
11/07/2014 16:09:02

80812  
TS4  
SC

-----  
Standard Lot#: 25913100A2  
Last Changed By: T. GALLEGOS #28847  
-----

QAS: T. GALLEGOS #28847  
TUCSON PD  
-----

Operator: T. GALLEGOS #28847  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	16:09:32
0.100 Cal Check	0.099	16:09:53
Air Blank	0.000	16:10:23

Preventative Maintenance  
Performed -- Timer Reset



TUCSON POLICE DEPARTMENT  
CRIME LABORATORY

90812  
TS4

INTOXILYZER SERVICE RECORD  
FOR  
CALIBRATION GAS TANK CHANGE

SC  
12/10/14

Date: 11.7.2014  
QAS: Terry Gallegos 28847  
Intoxilyzer Serial Number: 80-006053  
New Lot Number: 25913100AZ  
New Expiration Date: 10.1.2015

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."  
Select "D" for dry gas and enter the following information:  
  - Enter Target Value: 0.100
  - Enter Control Lot #: From the tank label
  - Enter Exp Date: From the tank label (mm/dd/yyyy)
  - Edit info Y/N? Y or N
  - Update Std Info? Y ENTER
  - "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."  
Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 099 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Terry Gallegos

INTOXILYZER 8000  
Location: TUCSON PD  
Core Version: 8105.48  
Serial Number: 80-006053  
11/07/2014 16:00:40

*10 of 12  
TS4*

-----  
Standard Lot#: 25913100A2  
Last Changed By: T. GALLEGOS #28847  
-----  
QAS: S. CIONE #50616  
TUCSON PD  
-----

*SC*

Test	g/210L	Time
Air Blank	0.000	16:03:54
0.100 Cal Check	0.099	16:04:15
Air Blank	0.000	16:04:44
Air Blank	0.000	16:07:13
End of test	*	

\*Input Timed Out

N/A

Reprinted From Database

*The printer mangled the original  
TS4*

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

110f12  
TS4

SC

# Certificate of Calibration

This is to certify the calibration of **Intoxilyzer**® serial number 80-00053, manufactured by CMI Inc., a subsidiary of MPD, Inc. of Owensboro, Kentucky, was tested and found to conform to the National Highway Traffic Safety Administration (NHTSA) Standard for Devices to Measure Breath Alcohol (Federal Register, Vol.58 No.179, pp 48705-48710, Sept. 17, 1993) for accuracy and precision. Reference materials are traceable through the National Institute of Standards and Technology (NIST) to the International System of Units (SI).

Date October 20, 2014 Signed [Signature]

Technician



316 East 9<sup>th</sup> Street  
Owensboro, KY 42303  
USA

Part No. 650517 Rev.A

Pursuant to Section 28.1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.



09/2014 7:50:39 AM

Pursuant to Section 28.1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

1 of 2  
TS4


80-006053

Photo of Circuit Board from CMI

5c



1/19/2014 7:51:47 AM

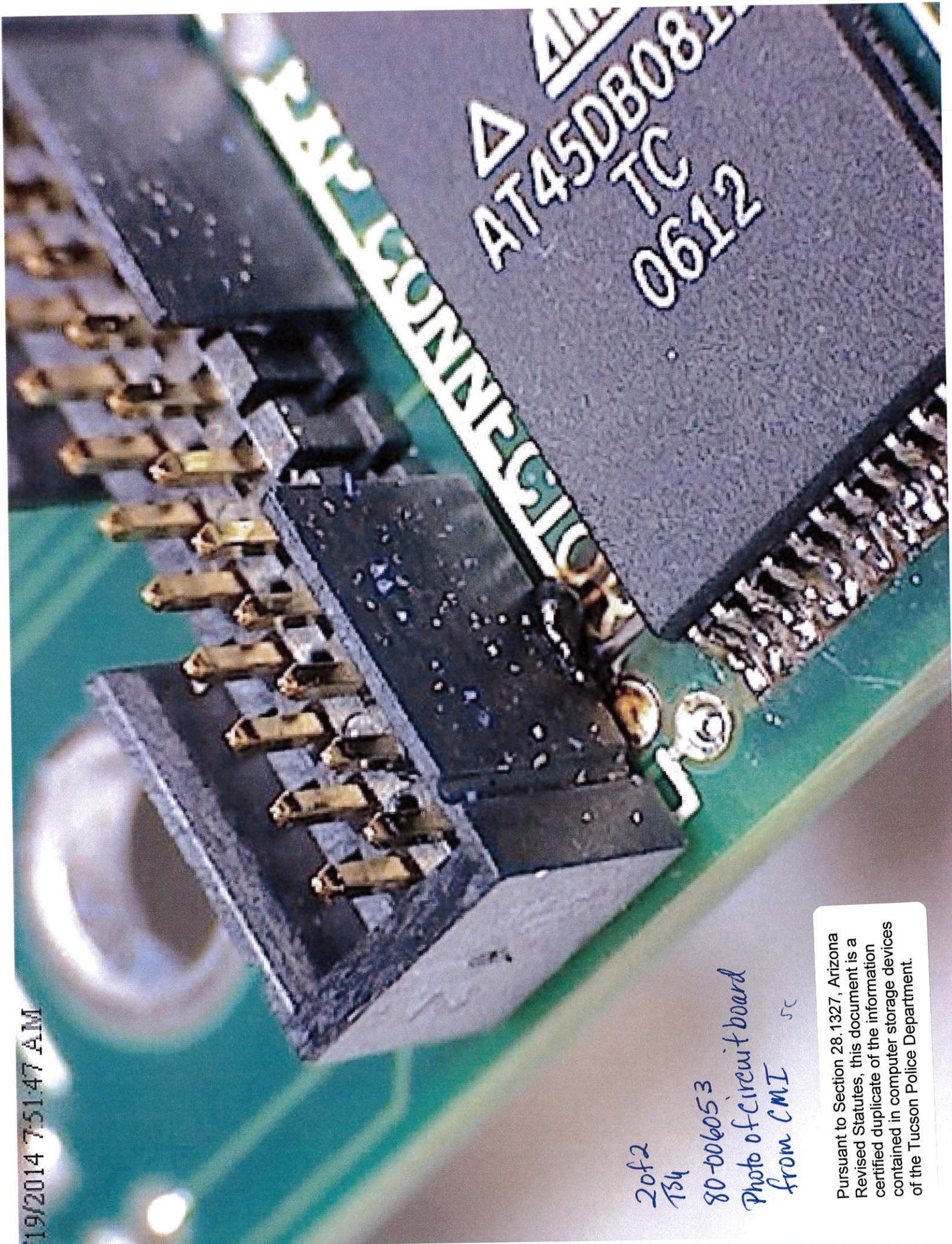

2 of 2  
TS4  
80-006053  
Photo of circuit board  
from CMI 5c

Pursuant to Section 28.1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.