



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory
1306 W. Miracle Mile, Tucson, AZ 85705

SC
12/10/14

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-006054 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 11.6.2014 Cal 1 0.099 AC Cal 2 0.098 AC

Between the dates of 10.9.2014 and 11.6.2014 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on —.
The instrument was returned to service on —.

Terry S. Gallegos

Terry S. Gallegos


12.4.2014

ADPS QAS Permit # 33169

SUBSCRIBED AND SWORN to before me this 21st, day of December, 2014.

Elma Escarcega

Notary Public

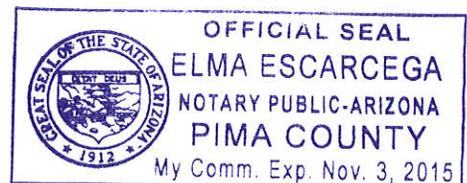

*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

10f9
TS4
SC
12/10/14

QA SPECIALIST Terry Gallegos 28847 AGENCY Tucson Police Department

DATE 11-6-2014 TIME 1211

INTOXILYZER SERIAL # 80-006054 LOCATION DDW-DUI-HQ

() 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 098 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Terry Gallegos

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006054
Core Version: 8105.48
11/06/2014

12:12:51

20f9
TS4
sc

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	12:13:43
Diagnostic Test	Pass	12:14:18
Air Blank	0.000	12:14:46
0.100 Cal Check	0.099	12:15:07
Air Blank	0.000	12:15:36
Subject Test	0.000	12:16:08
Air Blank	ABT*	12:16:29
Air Blank	0.000	12:16:58

*Sequence Aborted

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006054
Core Version: 8105.48
11/06/2014

12:17:12

3059
TS4

SC

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST,SQAP

DOB: 11/11/1987

Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	12:18:14
Diagnostic Test	Pass	12:18:49
Air Blank	0.000	12:19:16
0.100 Cal Check	0.099	12:19:37
Air Blank	RFI*	12:19:46
Air Blank	IPS**	12:19:54

*RFI Detect

**Improper Sample

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006054
Core Version: 8105.48
11/06/2014

12:29:09

4 of 9
TS4
5c

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1987
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	12:29:56
Diagnostic Test	Pass	12:30:31
Air Blank	0.000	12:30:58
0.100 Cal Check	0.099	12:31:19
Air Blank	0.000	12:31:49
Subject Test	DEF*	12:34:56
Air Blank	0.000	12:35:23
Air Blank	0.000	12:35:51
Subject Test	XXX**	12:36:29
Air Blank	0.000	12:37:04

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006054
Core Version: 8105.48
11/06/2014 12:37:37

50f9
TJ

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

SC

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	12:38:06
0.100 Cal Check	0.098	12:38:27
Air Blank	0.000	12:38:57

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

60 f9
TJ4

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

SC
12/10/14

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Terry Gallegos 28847 AGENCY Tucson Police Department
DATE 11-6-2014 TIME 1211
Intoxilyzer Serial # 80-006054 Location DDW-DUI-HQ

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006054
Core Version: 8105.48
11/06/2014 12:11:22

Zof9
TS4

Standard Lot#: 25913100A2
Last Changed By: S. CIONE #50616

SC

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	12:11:51
0.100 Cal Check	0.099	12:12:12
Air Blank	0.000	12:12:42

Preventative Maintenance
Performed -- Timer Reset



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

80f9
TJ4

INTOXILYZER SERVICE RECORD
FOR
CALIBRATION GAS TANK CHANGE

SC
12/10/14

Date: 11-6-2014
QAS: Terry Gallegos 28847
Intoxilyzer Serial Number: 80-006054
New Lot Number: 11514100 A2
New Expiration Date: 05-01-2016

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."
Select "D" for dry gas and enter the following information:
Enter Target Value: 0.100
Enter Control Lot #: From the tank label
Enter Exp Date: From the tank label (mm/dd/yyyy)
Edit info Y/N? Y or N
Update Std Info? Y ENTER
"Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."
Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 101 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Terry Gallegos

INTOXILYZER 8000
Location: TUCSON PD
Serial Number: 80-006054
Core Version: 8105.48
11/06/2014 13:08:20

90f9
T26

Standard Lot#: 11514100A2
Last Changed By: T. GALLEGOS #28847

SC

QAS: T. GALLEGOS #28847
TUCSON PD

Operator: T. GALLEGOS #28847
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	13:08:49
0.100 Cal Check	0.101	13:09:10
Air Blank	0.000	13:09:40

Preventative Maintenance
Performed -- Timer Reset