



316 E. 9th St.  
Owensboro, KY 42303  
1-800-835-0690  
Fax: 502-685-6678

April 30, 2007

Tucson Police Department Crime Laboratory  
Blood Alcohol Unit  
270 S. Stone Avenue  
Tucson, AZ 85701

To Whom It May Concern:

This is a list of software changes from version 8105.44 to 8105.46.

Changes from 8105.44 to 8105.45 (03/23/2004)

- New mouth alcohol detect routine based on raw-raw comparison.
- Removal of static delay which was causing improper samples.
- Password added to L3 access when card swipe is off.
- Prevent keyboard fault associated with "symbol" lockup.
- All incoming keyboard characters to upper case.
- QAS info update moved to precede wet/dry cal check from L2 ONLY.
- PM menu option changed to reset 31 day counter ONLY.
- Results for invalid sample and "no blow" recorded as .XXX and .DEF to match printout.
- Reformatted external auto cal printout to include barometric readings matching internal printout.
- Fix printer initialization faults.

Changes from 8105.45 to 8105.46 (05/05/2005)

- Prevent processor from entering low power mode
- Added a reference diagnostic tool
- Increased wet control test time to 30 seconds.
- Adjusted 15 V diagnostic limits.

If you have any questions, please give me a call.

Toby Hall  
President  
CMI, Inc.

**INTOXILYZER®** ...so you can breathe easier

